VERMEIREN

Charme

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: F, September 2011

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2011

Contents

Preface	2
1 Productdescription	3
1.1 Intended Use.....	3
1.2 Technical specifications	4
1.3 Drawing.....	5
1.4 Explanation of symbols.....	5
1.5 Accessories.....	5
1.6 For your safety	5
2 Use	6
2.1 Carrying the shell chair	6
2.2 Recommendations to use the shell chair Charme.....	6
2.3 Inclination of shell chair.....	7
2.4 Running wheels / Parking brakes	8
2.5 Mounting or removing of tray	8
2.6 Mounting or removing of head rest.....	9
2.7 Use of safety harness.....	9
2.8 Taking curbs by using ramps.....	10
2.9 Transport in the car.....	10
3 Installation and adjustment.....	10
3.1 Tools	11
3.2 Manner of delivery	11
3.3 Adjusting the seat height	11
3.4 Mounting or Removing the hand grips	12
3.5 Mounting the fifth wheel.....	12
4 Maintenance	12
4.1 Regular Maintenance.....	12
4.2 Shipping and storage	13
4.3 Care	13
4.4 Inspection	14
4.5 Disinfection	14
5 Guarantee	16
6 Disposal.....	16
7 Declaration of conformity	16
8 Maintenance plan	17
9 Disinfection book	17

Preface

First of all we want to thank you for putting your trust in us by selecting one of our products.

Vermeiren products are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the product.

The lifetime for the product is strongly influenced by the care and maintenance of the product.

This manual will help you get acquainted with the operation of your product.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

1 Productdescription

1.1 Intended Use

The Charme shell chair is developed entirely for your comfort.

The shell chair is designed for sitting of 1 person.

The shell chair has been designed exclusively for people with limited mobility and weak muscles so be careful with the use of the shell chair.

The shell chair is suited for indoor use.

You will need a helper for indoor transporting of the Charme, and the chair might also need its own storage place.

The different types of accessories, and the modular construction allow full use by persons disabled by:

- paralysis
- weak/malformed limbs
- injuries of the joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)
- and also for aged persons.

The indication to use this product must take into account the following elements:

- body size and weight (**max. load 120 kg**)
- physical and psychological condition
- home environment

The shell chair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

The shell chair should not be used as a ladder, nor is it a transport for heavy or hot objects.

When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged.

It is necessary to choose the best fitted model for the size/build of the patient.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

1.2 Technical specifications

Technical terms below are valid for the product in standard settings. If other accessories are used, the tabulated values will change.

Make	Vermeiren							
Address	Vermeirenplein 1/15, B-2920 Kalmthout							
Type	Shell chair							
Model	Charme							
Maximum occupant mass	120 kg							
Description	Min.		Max.					
Overall length with legrest	770 mm							
Effective seat width	380 mm	400 mm	420 mm	440 mm	480 mm			
Overall width	780 mm							
Overall height (with backrest height 750 mm)	1220 mm		1245 mm					
Overall height (with backrest height 770 mm)	1240 mm		1265 mm					
Overall height (with backrest height 790 mm)	1260 mm		1285 mm					
Total mass	Approx. 25 kg							
Mass of heaviest part	Not applicable							
Maximum mass of tray	1 kg							
Static stability downhill	6% (in standard configuration)							
Static stability uphill	6% (in standard configuration)							
Obstacle climbing	Not applicable							
Seat plane angle	-10°		+30°					
Effective seat depth	460 mm							
Seat surface height at front edge	430 / 450 / 470 mm		475 / 495 / 515 mm					
Thickness seat cushions	60 / 80 / 100 mm		60 / 80 / 100 mm					
Backrest angle	-10°		+30°					
Backrest height	750 mm		790 mm					
Distance between footrest and seat	Not applicable							
Angle between seat and footrest	Not applicable							
Armrests height	230 mm		270 mm					
Color of upholstery	Khaki beige	Bordeaux gray		Blue ecru				
Storage and use temperature	+ 5 °C		+ 41 °C					
Storage and use humidity	30%		70%					
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / kg / °								

Table 1: Technical specifications


The shell chair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

EN 1021-1, EN1021-2: Resistance to ignition of upholstered parts

1.3 Drawing

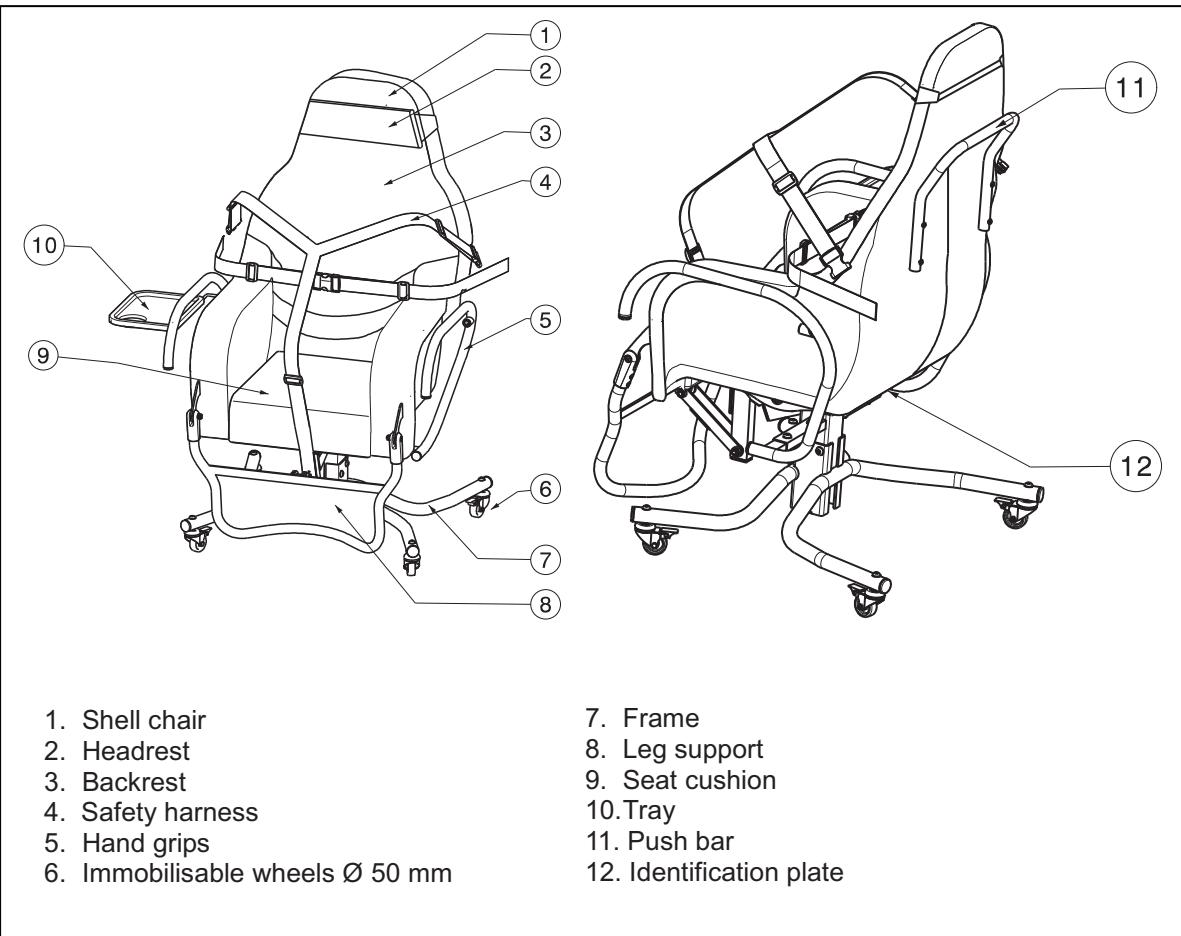1.4 Explanation of symbols

Maximum mass

Indoor use

CE conformity

Not crash-tested

1.5 Accessories

⚠ WARNING: Precautions shall be taken if accessories not listed below are used.

- Electrical seat and back inclination system

1.6 For your safety

Take note of the following safety hints:

- ⚠ To prevent injury of the hands and damage to the shell chair, make sure that no objects and or body parts are caught between the adjustment systems (seat height, inclination of shell chair, ...)
- ⚠ Regularly check that the wheel immobilizers function correctly.
- ⚠ The parking brakes should be applied before getting into and out the shell chair.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the shell chair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so there is no damage to the shell chair.

- ⚠ Only use your shell chair according to regulations. For example, avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.**
- ⚠ Never cross steps, ledges or other obstacles with a patient seated on the chair. If alternatives like ramps or lifts are available, use them.**
- ⚠ When transporting the shell chair, never pick it up by grasping movable parts (hand grips)**
- ⚠ Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight.**
- ⚠ Never exceed the **maximum load (120 kg)**.**
- ⚠ Use only genuine and original parts for maintenance work.**

2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The product is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the shell chair are given in § 3.

2.1 Carrying the shell chair

The best way to carry the shell chair is to make use of the wheels and roll the shell chair.

If this is not possible, firmly grasp the frame with two persons on the fixed parts either on the front, on the back or on the side under the arm supports. Do not use the wheels, hand grips to grasp the shell chair.

2.2 Recommendations to use the shell chair Charme

2.2.1 Transfer in and out the chair

- ⚠ WARNING: Risk of injury – Adjust the safety harness that no strangulation of the patient can appear and the patient can sit comfortable in the chair.**
- ⚠ WARNING: Risk of injury – Adjust the safety harness as close as possible to the body of the patient so the patient cannot slide out the safety harness.**
- ⚠ WARNING: Risk of tipping over – Be sure that the shell chair do not tip over backwards when the shell chair is placed in his most backwards position.**
- ⚠ CAUTION: In case you cannot perform the transfer in a safe manner, ask someone to assist you.**

The transfer from the patient in the shell chair shall be according the instructions below:

1. Position the shell chair as close as possible to the chair, couch or bed from you wish to transfer.
2. Check brakes from the shell chair are engaged and that the safety belt is good fixated to the shell chair.
3. Place the shell chair in his most position to the front (minimum inclination: -10°).
4. Transfer to the chair.
5. Mount the safety harness.
6. Check that the safety harness is placed correct (as close as possible to the body of the patient). If needed adjust the safety harness.
7. Place the chair in his most backwards position (maximum inclination: +30°).

The transfer from the patient out the shell chair shall be according the instructions below:

1. Position the shell chair as close as possible to the chair, couch or bed to you wish to transfer.

2. Check brakes from the shell chair are engaged.
3. Place the shell chair in his most position to the front (minimum inclination: -10°).
4. Unfasten the safety harness.
5. Transfer out the chair.

2.2.2 Correct position in the chair

Some recommendations for a comfortable use of the chair:

- Position your backside as close as possible to the back rest.
- Make sure your upper legs are horizontal – If needed adjust the seat height.


2.2.3 Moving/transporting the chair

- ⚠ WARNING: Risk of clamping – Be careful passing through restricted passages (e.g. doors).**
- ⚠ WARNING: Risk of burns – Be careful when using in hot or cold environments for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.**
- ⚠ WARNING: Risk of injury – Never use the shell chair to transport a patient.**
- ⚠ WARNING: Risk of injury – There should be no people or objects under the shell chair during transportation.**


1. Engage the brakes.
2. Transfer the patient out the chair.
3. Disengage the brakes.
4. Transport the chair to the desired place on an horizontal underground.
5. Transfer the patient in the chair.
6. Engage the brakes.

2.3 Inclination of shell chair

- ⚠ WARNING: Risk of tipping over – Be sure that the shell chair do not tip over backwards when the shell chair is placed in his most backwards position.**
- ⚠ WARNING: Risk of injury – Check the brakes are engaged before changing the inclination of the shell chair.**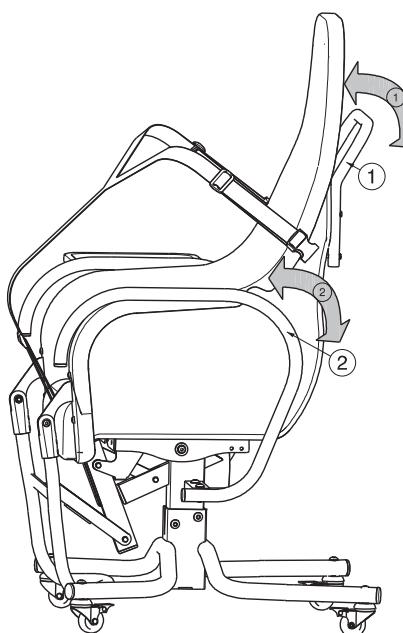

Minimum inclination: -10°

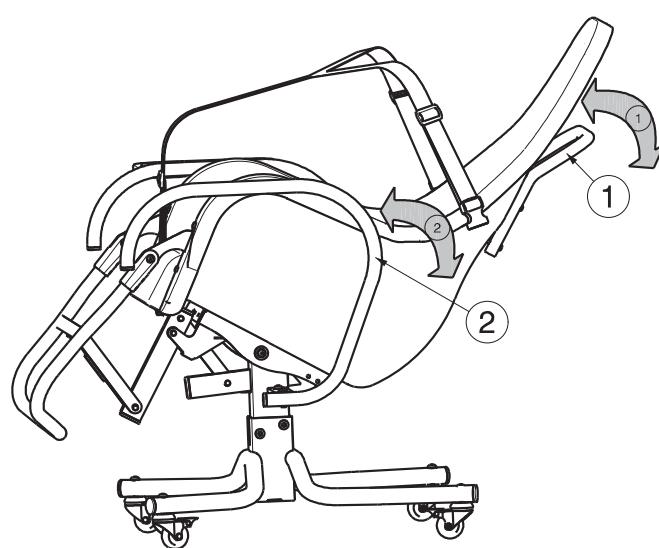Maximum inclination: +30°

Inclination set by an attendant ①:


1. Engage the brakes.
2. Pull/Push the push bar ① in the desired direction to incline the shell chair backwards or set in the horizontal position (+30° to -10°).
3. The leg support shall also inclined.

Inclination set by the user ②:

1. Engage the brakes.
2. Pull/Push the on the hand grips ② in the desired direction to incline the shell chair backwards or set in the horizontal position (+30° to -10°).
3. The leg support shall also inclined.


2.4 Running wheels / Parking brakes

- ⚠ **WARNING:** Risk of injury – Lock all four wheels in place to put the shell chair in a parking position.
- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.
- ⚠ **WARNING:** The brakes are adjustable and can wear – Check the operation of the brakes before each use.
- ⚠ **CAUTION:** Risk of unintended movement – Make sure the shell chair is on a flat horizontal surface before releasing the brakes. Never release all the brakes simultaneously.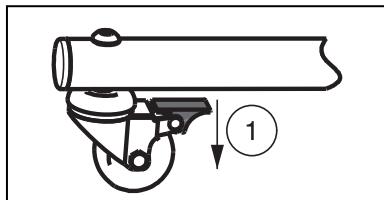

1 Engaging the parking brakes

With your foot, press the black brake plate of the four wheels positioned on the side downwards till it locks in place. The wheels cannot roll.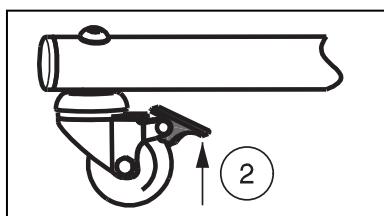


2 Disengaging the parking brakes

With your foot, press the black brake plate of the four wheels positioned on the side in the direction of the frame. The wheels have now been released and can both roll and swivel.


2.5 Mounting or removing of tray

- ⚠ **WARNING:** Risk of tipping over – Do not overload the tray. The maximum weight shall be 1 kg.

Mounting the tray: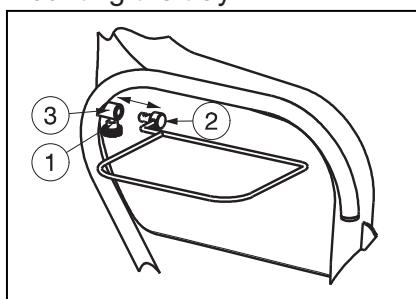

1. Loosen screw ①.
2. Mount the pen of the frame of tray ② to the bushing of hand grip ③.
3. Retighten screw ① firmly by hand.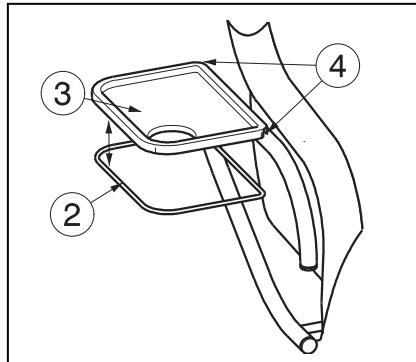

4. Mount the tray ③ to the frame of the tray ②.
5. Put the two holes ④ into direction of the chair.

To remove the tray you can repeat the steps in opposite direction.

2.6 Mounting or removing of head rest

Mounting the head rest:


1. Take the head rest.
2. Check that the large edge is placed to the bottom.
3. Mount the head rest to the back of the shell chair by the elastic.

Remove the head rest from the back of the shell chair by the elastic.


2.7 Use of safety harness

- ⚠ WARNING: Risk of injury – Adjust the safety harness that no strangulation of the patient can appear and the patient can sit comfortable in the chair.
- ⚠ WARNING: Risk of injury – Adjust the safety harness as close as possible to the body of the patient so the patient cannot slide out the safety harness.
- ⚠ WARNING: Risk of injury – Use the safety harness for patient that make extreme movements. These can lead to falling out the chair.

When the patient has spasm or make restless movements the safety harness shall be used.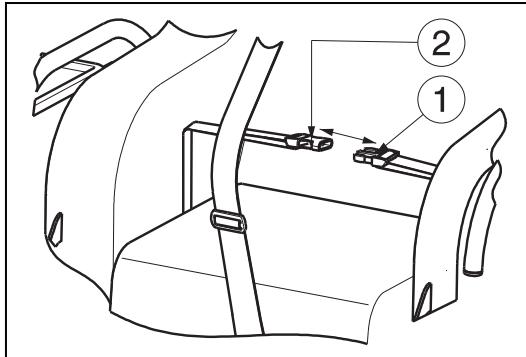


1. Mount the waist belt around the waist of the patient.
2. Click the buckle of belt ① into buckle of belt ②.
3. If needed adjust the waist belt.

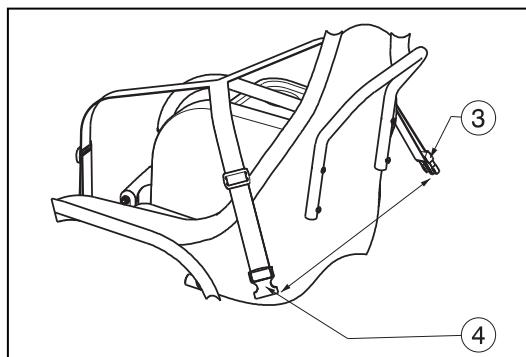4. Mount the safety harness around the body of the patient.
5. Cross the harness around the back of the shell chair.
6. Click the buckle of belt ③ into buckle of belt ④.
7. If needed adjust the safety harness.

The adjustment of the belt shall be with the plastic adjustment system ① mounted on the belt.

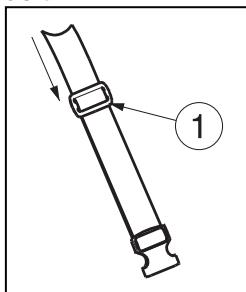1. Pull the belt through the adjustment system ①.

2.8 Taking curbs by using ramps

⚠ WARNING: Risk of tipping over – Never go up or down ramps without the aid of a helper.

If you are considering the installation of ramps to overcome obstacles, kindly note the following:

The chair is intended firstly and mostly to be used inside on level and solid surfaces. Never go up or down ramps that are rough or have bumpy edges. Limit yourself to ascents or descents with a maximum angle of 6%.

2.9 Transport in the car

- ⚠ DANGER:** Risk of injury – The shell chair is not suited for use as a seat in a motor vehicle.
- ⚠ WARNING:** Risk of injury – See that the chair is attached properly in the car or the truck.

Never use your shell chair as a seat in an automobile or other vehicle. The shell chair shall be marked with following symbol.

To transport the shell chair in the car use following steps:

1. Remove accessories.
2. Store accessories safely.
3. Place the chair in the luggage place.
4. If the chair and the passenger compartment is NOT separated, attach the frame of the chair securely to the vehicle. You can use the available safety belts in the vehicle.

3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

- ⚠ WARNING:** Risk of unsafe settings - Use only the settings described in this manual.
- ⚠ WARNING:** Variation of allowed adjustments can still change the stability of your product (tilt back or sideways).

3.1 Tools

To set up the chair the following tools are needed.

- Wrench set n° 17
- Allen key set n° 6

3.2 Manner of delivery

The Vermeiren product shall be delivered with:

- 1 frame with shell chair and 4 wheels
- 1 wheel not mounted on the shell chair
- 1 legsupport
- Tools
- Manual
- Accessories

Before use check if everything is included and that no products are damaged (example by transport, ...).


3.3 Adjusting the seat height

- ⚠ WARNING: Risk of injury – Make sure that no objects and or body parts are caught between the adjustment systems.**
- ⚠ WARNING: Risk of injury – Adjusting can only be done by authorized people.**
- ⚠ WARNING: Risk of injury – Check that all screws have been properly fastened.**

To adjust the seat height you can either make use of seat cushions of different diameters, or set the shell chair itself at different heights on the frame. The shell chair must not be removed during this process. The thickness of the seat cushions must be decided by the purchase.

Thickness seat cushions	Seat height	Position
60 mm	450 mm	Hole 1
80 mm (Standard)	470 mm (Standard)	Hole 1
100 mm	490 mm	Hole 1
60 mm	475 mm	Hole 2
80 mm	495 mm	Hole 2
100 mm	515 mm	Hole 2

Table 2: Seat heights

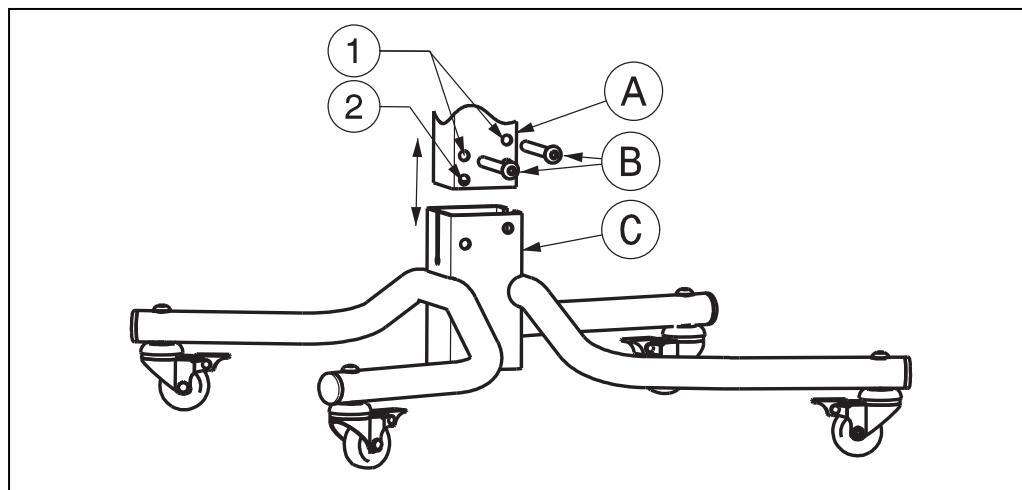
1. Place the shell chair on his side.
2. Loosen the screws ②.
3. Move part ① out part ③.
4. Place the shell chair in the desired position (refer to table 2).
5. Retighten the screws ② firmly by hand.

3.4 Mounting or Removing the hand grips

- ⚠ **WARNING:** Risk of injury – Make sure that no objects and or body parts are caught between the adjustment systems.
- ⚠ **WARNING:** Risk of injury – Adjusting can only be done by authorized people.
- ⚠ **WARNING:** Risk of injury – Check that all screws have been properly fastened.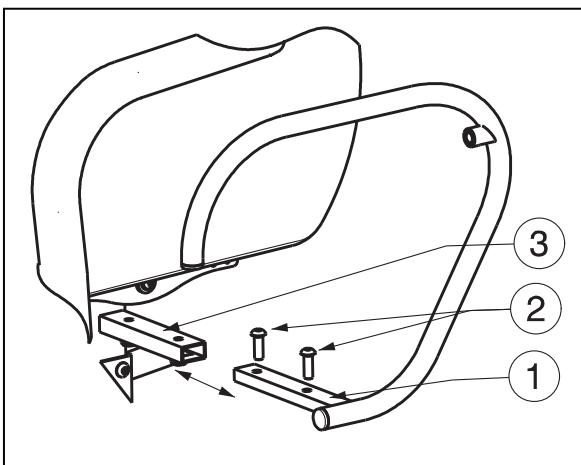


Mounting the hand grips:

1. Place the shell chair on his side.
2. Mount part ① in part ③.
3. Retighten the screws ② firmly by hand.

Removing the hand grips:

1. Place the shell chair on his side.
2. Loosen the screws ②.
3. Move part ① out part ③.

3.5 Mounting the fifth wheel

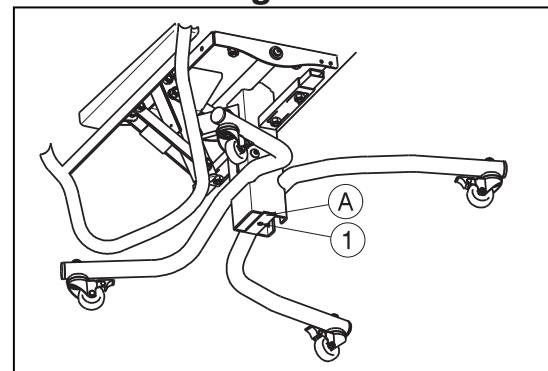With your shell chair there is 1 wheel delivered that is not mounted on the shell chair. There is the possibility to mount this wheel on the shell chair.

The location for the fifth wheel is in the middle of the shell chair. Mounted on the plate ① in hole ②.

4 Maintenance

Lifetime of the chair is influenced by its use, storage, regular maintenance, servicing and cleaning.

4.1 Regular Maintenance

The maintenance activities to ensure a good condition for your Vermeiren chair are described below:

- Before every use
 - Check the wheels and make them clean.
 - Check the brakes.
 - Check screw fastenings are secured.
 - Check the chair (cleanliness, cracks, damage on structural parts ...) and make it clean. If necessary change the coating.

- Every 8 weeks: inspected and lubricating or adjusting from
 - Tilt mechanism
 - Adjusting mechanism (seat height, hand grips)
- Every 6 months or for every new user
 - General review
 - Cleanliness, Disinfection
 - Function of the wheels: If the rotation resistance is too large, clean the bearings of the steering wheels. If this is insufficient, please consult your specialist dealer.

For your convenience, on the back of this manual is a maintenance plan.

Repairing and assembling of spare parts for your chair may only be performed by the specialist dealer.

Only authorized Vermeiren spare parts may be installed.

4.2 Shipping and storage

The shipping and storage of the product shall be according following instructions:

- Store in a dry place (between +5 °C and +41 °C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the product from rust and foreign bodies. (e.g. salt water, sea air, sand, dust).
- Store all removed parts together in one place (or mark them if necessary) to avoid mixing up with other products when re-assembling.
- Components must be stored without being subjected to strains (Do not put too heavy parts on the chair, not clamping between something, ...).

4.3 Care

4.3.1 Covers

The cleaning of the cover shall be according:

- Clean covers with a cloth moistened with hot water. Be aware that you do not soak the cover.
- Use a mild commercial detergent for removing stubborn dirt.
- Stains can be removed by using a sponge or a soft brush.
- Do not use strong cleaning liquids like solvents, nor use hard brushes.
- Never clean with steam and/or pressure washers.

4.3.2 Plastic parts

Clean plastic parts of your chair with commercial plastic cleaners. Only use a soft brush or soft sponge. Please comply with special product information.

Examples of plastic parts are tyres, protection on the attachment leg support.

4.3.3 Coating

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

When cleaning, only use warm water and normal household detergents and soft brushes and cloths. Ensure that no wetness gets into the tubes.

Initially the zinc parts only require rubbing with a dry cloth. Stubborn dirt can best be removed by using a suitable commercial zinc polish.

4.4 Inspection

In principle we recommend one inspection every year, and a minimum of one before usage is resumed. All of the following checks must be performed and documented by authorized persons:

- Check the frame parts for plastic deformation, cracks and impaired functioning.
- Visually check for damage to the paintwork (danger of corrosion)
- Check the operation of the wheels (free running, level rolling, axle play, wheel immobilizers)
- Check the solidity and tightness of all screws, nuts and bolts.
- Verify the amount of grease on the metal joints of movable parts
- Visually check all plastic parts for cracks and brittle spots
- Check the functioning of the leg support (locking, loading, deformation, wear and tear due to loading).
- Check the operation of other detachable parts (example: multi-point safety belt)
- Completeness of the delivery condition, instruction manual available?

The service must only be signed off in the maintenance plan if a minimum of all the above-mentioned aspects have been checked.

4.5 Disinfection

- ⚠ WARNING: Dangerous Products - The use of disinfectants is restricted to authorized personnel.**
- ⚠ WARNING: Dangerous Products, change on irritate your skin - You should wear suitable protective clothing because the disinfectants could irritate your skin. For this purpose you should also take note of the product information of the solutions concerned.**

All parts of the chair can be treated by scrubbing with a disinfectant.

All steps taken to disinfect rehabilitation equipment, their components or other accessory parts are to be recorded in a disinfection report containing a minimum of the following information (with product documentation appended):

Date of the disinfection	Reason	Specification	Substance and concentration	Signature
--------------------------	--------	---------------	-----------------------------	-----------

Table 3: Example of a disinfection book

Abbreviations used in column 2 (reason):

V = Suspected infection IF = Infection case W = Repetition I = Inspection

For a blank sheet of a disinfection book see § 9.

The recommended disinfectants for scrubbing (based on the list provided by the Robert Koch Institute, RKI) are standing in the table below. The current state of the disinfectants included in the RKI list can be obtained from the Robert Koch Institute (RKI) (homepage: www.rki.de).

Active substance	Product name	Laundry disinfection		Surface disinfection (scrubbing-/wiping disinfection)		Disinfection of excretions 1 part sputum or stools + 2 parts diluted solution or 1 part urine + 1 part diluted solution						Area of effectiveness	Manufacturer or Supplier		
						Sputum		Stools		Urine					
		Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect				
		%	Hr.	%	Hr.	%	Hr.	%	Hr.	%	Hr.				
Phenol or phenol derivative	Amocid	1	12	5	6	5	4	5	6	5	2	A	Lysoform		
	Gevisol	0,5	12	5	4	5	4	5	6	5	2	A	Schülke & Mayr		
	Helipur			6	4	6	4	6	6	6	2	A	B. Braun		
	m-cresylic soap solution (DAB 6)	1	12	5	4							A			
	Phenol	1	12	3	2							A			
Chlorine, organic or inorganic substances with active chlorine	Chloramin-T DAB 9	1,5	12	2,5	2	5	4					A ¹ B			
	Clorina	1,5	12	2,5	2	5	4					A ¹ B	Lysoform		
	Trichlorol	2	12	3	2	6	4					A ¹ B	Lysoform		
Per combinations	Apesin AP100 ²			4	4							AB	Tana PROFESSIONAL		
	Dismozon pur ²			4	1							AB	Bode Chemie		
	Perform ²			3	4							AB	Schülke & Mayr		
	Wofesteril ²			2	4							AB	Kesla Pharma		
Formaldehyde and/or other aldehydes or derivatives	Aldasan 2000			4	4							AB	Lysoform		
	Antifect FD 10			3	4							AB	Schülke & Mayr		
	Antiseptica surface disinfection 7			3	6							AB	Antiseptica		
	Apesin AP30			5	4							A	Tana PROFESSIONAL		
	Bacillocid special			6	4							AB	Bode Chemie		
	Buraton 10F			3	4							AB	Schülke & Mayr		
	Desomed A 2000			3	6							AB	Desomed		
	Hospital disinfectant cleaner			8	6							AB	Dreiturm		
	Desomed Perfekt			7	4							AB*	Desomed		
	Formaldehyde-solution (DAB 10), (formaline)	1,5	12	3	4							AB			
	Incidin Perfekt	1	12	3	4							AB	Ecolab		
	Incidin Plus			8	6							A	Ecolab		
	Kohrsolin	2	12	3	4							AB	Bode Chemie		
	Lysoform	4	12	5	6							AB	Lysoform		
	Lysoformin	3	12	5	6							AB	Lysoform		
	Lysoformin 2000			4	6							AB	Lysoform		
	Melsept	2	12	4	6							AB	B. Braun		
	Melsitt	4	12	10	4							AB	B. Braun		
	Minutil	2	12	6	4							AB	Ecolab		
	Multidor			3	6							AB	Ecolab		
	Nûscosept			5	4							AB	Dr. Nüsken Chemie		
	Optisept			7	4							AB*	Dr. Schumacher		
	Pursept-FD			7	4							AB*	Merz		
	Ultrasol F	3	12	5	4							AB	Fresenius Kabi		
Amphoteric surfactants (amfotensiden)	Tensodur 103	2	12									A	MFH Marienfelde		
Lye	Lime-milk ³							20	6			A ³ B			

1 Not effective against myco-bacteria when service disinfecting, especially in the presence of blood.

2 Not suitable for disinfecting blood-contaminated or porous surfaces (e.g. raw wood).

3 Useless for tuberculosis; preparation of Lime-milk: 1 part dissolved lime (calcium hydroxide) + 3 parts water.

* Checked for effectiveness on viruses in accordance with checking methods of the RKI (Federal Health Reporting 38 (1995) 242).

A: Suitable for killing vegetative bacterial germs including myco-bacteria as well as fungi, including fungal spores.

B: Suitable for deactivating viruses.

Table 4: Disinfectants

Kindly consult your specialist dealer if you have queries on matters related to disinfection; he will gladly assist you.

5 Guarantee

Excerpt from the "General Business Conditions":

(...)

5. The guarantee period for warranty claims is 24 months for the Charme frame.

(...)

The guarantee excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the guarantee excludes parts or working parts subject to natural wear and tear.

(...)

6 Disposal

When disposing of the product, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

7 Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked medical devices :

Productgroup: Shell chairs

Brand: Vermeiren

Type: Charme

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

EN 12182: 1999, EN 12183: 2009

8 Maintenance plan

Date	Maintenance	Remarks	Paragraph
1/1/2011	<i>Greasing and general servicing</i>	<i>non</i>	

9 Disinfection book

Date of the disinfection	Reason	Specification	Substance and concentration	Signature

Abbreviations used in column 2 (reason):

V = Suspected infection IF = Infection case

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, light-weight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at
our website, <http://www.vermeiren.be/registration>

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Vermeiren Nederland B.V.