


T4J Parts List (2010/10/27)

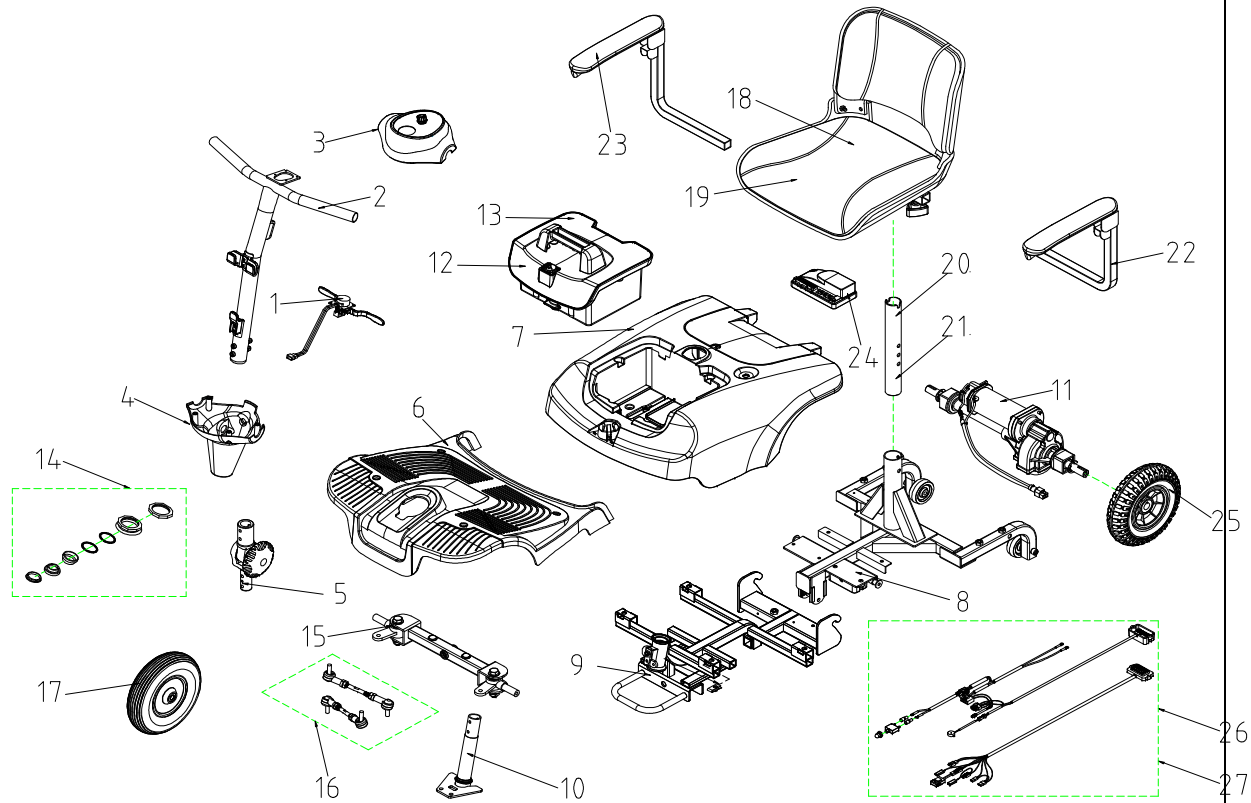


## WT-T4J 售服零件表

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
1	S12-060-00100	SPEED DIAL ASS'Y	1	
2	S02-060-00300	T TYPE HANDLE BAR ASS'Y	1	
3	S08-060-00100	UPPER COVER ASS'Y,HAND BAR	1SET	
4	S08-060-00253	COVER ASS'Y , HANDLE BAR , LOWER , BLUE 53	1SET	
5	S02-060-00100	TILLER SWIVEL ASS'Y	1SET	
6	S08-060-00500	FRONT SHROUD ASS'Y , BLACK	1	
7	S08-060-00453	REAR SHROUD ASS'Y , BLUE 53	1	
8	S01-060-00100	FRAME,REAR SECTION ASS'Y	1SET	
9	S01-060-00400	FRAME,FRONT SECTION ASS'Y	1	
10	C03-060-00200	L TYPE STEERING AXLE ASS'Y	1	
11	S09-050-00100	REAR DRIVE ASS'Y(W/I BOLT,NUT ETC.),7μM 4700RPM	1	
12	S17-060-00100	BATTERY PACK ASS'Y , W/I BATTERY	1	
13	S17-060-00200	BATTERY PACK ASS'Y , W/O BATTERY	1	
14	S02-060-00200	REGULAR BEARING BALL SET	1SET	
16	S03-060-00200	CONNECTING ROD ASS'Y	1	
17	C05-034-00600	FRONT PU FOAMED TIRE , BLACK	1	
19	S07-060-00200	SEAT ASS'Y	1	
20	C07-060-00500	SEAT POST	1	
22	C07-027-00601	ARMREST,L	1	
23	C07-027-00701	ARMREST, R	1	
24	C10-060-00100	CONTROLLER , CURTIS 45A	1	
25	C06-034-00401	200x50 PU FOAMED TIRE , REAR	1	
26	S18-060-00100	WIRE HARNESS ASS'Y W/O BUZZER	1	


T4J Parts List (2010/10/27)

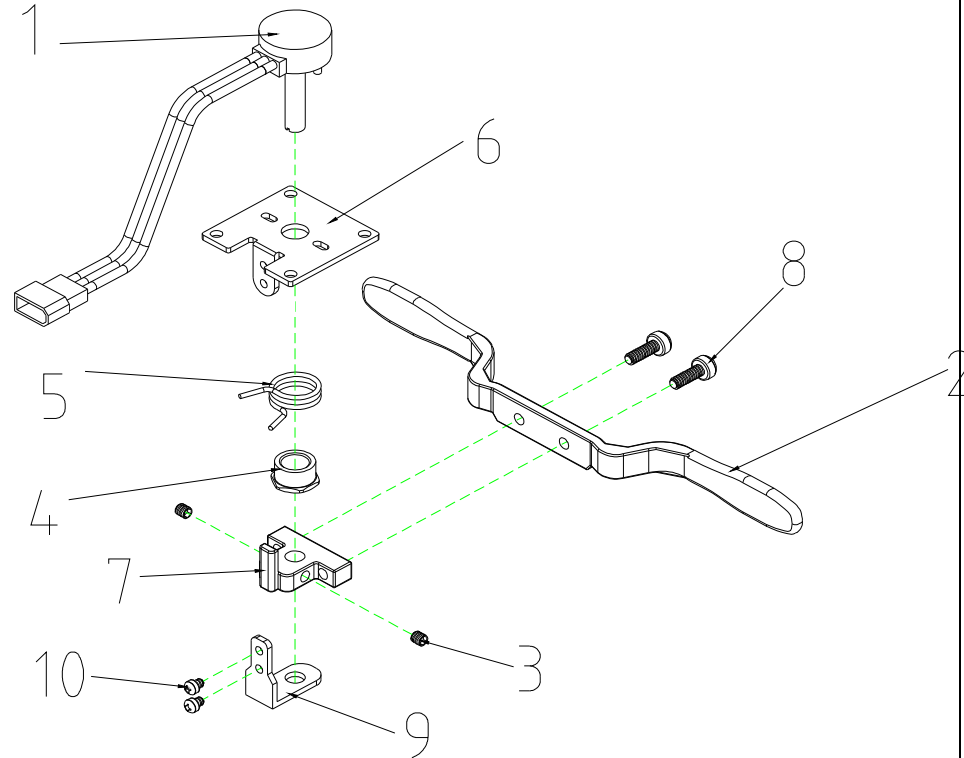

**1. SPEED DIAL ASS'Y**

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>1</b>	<b>S12-060-00100</b>	<b>SPEED DIAL ASS'Y</b>	<b>1</b>	
1-1	C12-060-00200	SPEED LEVER COMP.	1	
1-2	C12-009-00600	SPEED LEVER	1	
1-3	9358040040753	LOCKING SCREW,4x4	2	
1-4	C12-001-00101	NUT	1	
1-5	C12-001-00300	TORSION SPRING,SPEED LEVER	1	
1-6	C12-001-00402	FIX PLATE,SPEED LEVER	1	
1-7	C12-009-00700	HOLDER,SPEED LEVER	1	
1-8	9004040120704	BOLT,M4x12L	2	
1-9	C12-001-01500	SPEED LEVER COMP POSITION	1	
1-10	9004030040504	BOLT,M3x4	2	

T4J Parts List (2010/10/27)

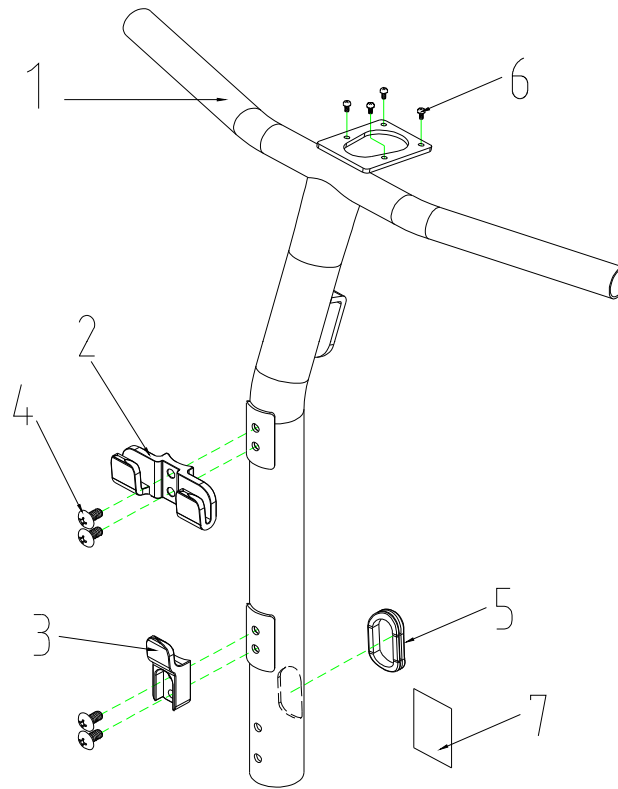


2. T TYPE HAND BAR ASS'Y

## WT-T4J 售服零件表

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>2</b>	<b>S02-060-00300</b>	<b>T TYPE HANDLE BAR ASS'Y</b>	<b>1</b>	
2-1	C02-060-00100	T TYPE HANDLE BAR	1	
2-2	C26-009-01601	BASKET HOLDER,PAINTED,UPPER	1	
2-3	C26-009-01700	BASKET HOLDER,PAINTED,LOWER	1	
2-4	9005060121002	BOLT , M6x12L	4	
2-5	C02-032-00202	PLUG,WIRE	1	
2-6	9004030080504	BOLT,M3X8	4	
2-7	C27-009-00200	LABEL,HANDLE BAR WARNING	1	

T4J Parts List (2010/10/27)

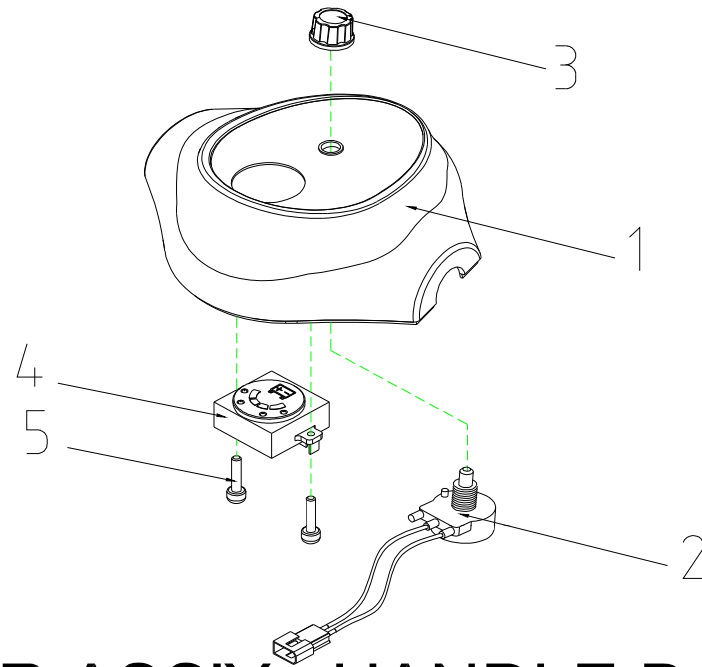


**3.COVER ASS'Y , HANDLE BAR  
, UPPER**

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>3</b>	<b>S08-060-00100</b>	<b>UPPER COVER ASS'Y,HAND BAR</b>	<b>1SET</b>	
3-1	C08-009-00362	UPPER COVER,BLACK	1	
3-2	C12-060-00100	SPEED DIAL COMP.	1	
3-3	C26-009-00200	SPEED ADJUST,JK-901C	1	
3-4	C11-045-00102	BATTERY INDICATOR	1	
3-5	9011030080502	BOLT , M3x8L	2	



T4J Parts List (2010/10/27)

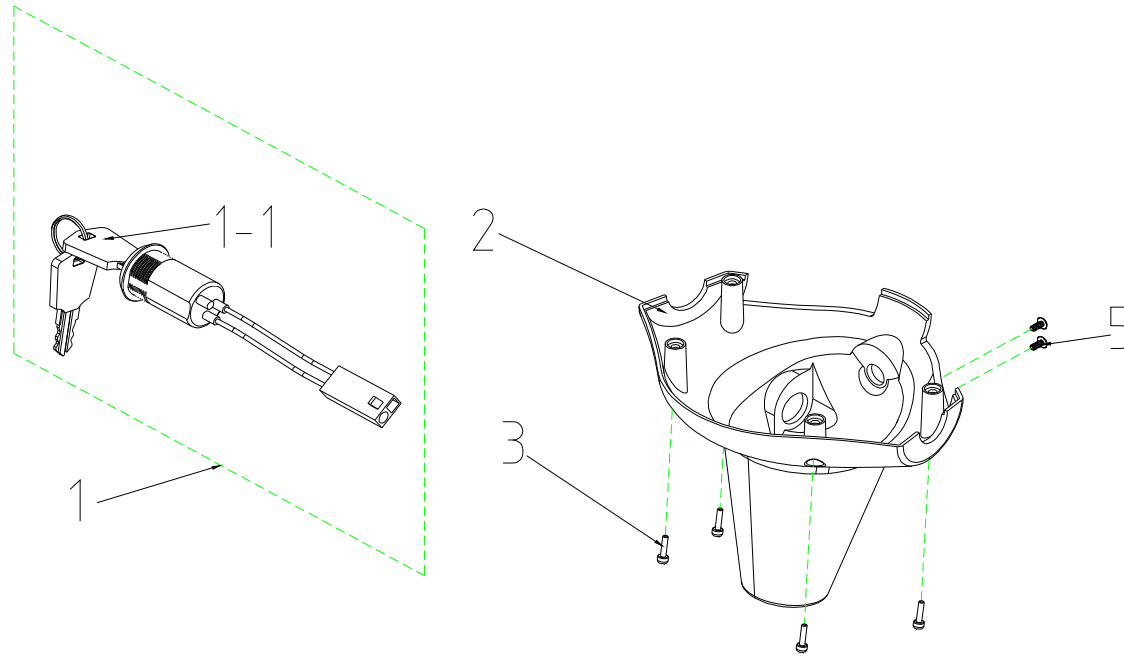


4. COVER ASS'Y HAND BAR , LOWER

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>4</b>	<b>S08-060-00253</b>	<b>COVER ASS'Y , HANDLE BAR , LOWER , BLUE 53</b>	<b>1SET</b>	
4-1	C18-050-00100	IGNITION SWITCH ASS'Y	1	
4-1-1	C18-050-00600	KEY	1	
4-2	C08-009-00453	COVER , HAND BAR , LOWER , BLUE 53	1	
4-3	9011350150004	BOLT , M3.5x15	4	
4-4	C25-001-00301	PLUG	1	
4-5	9005060101002	CR ROUND HD SCREW,M6X10L	2	

T4J Parts List (2010/10/27)

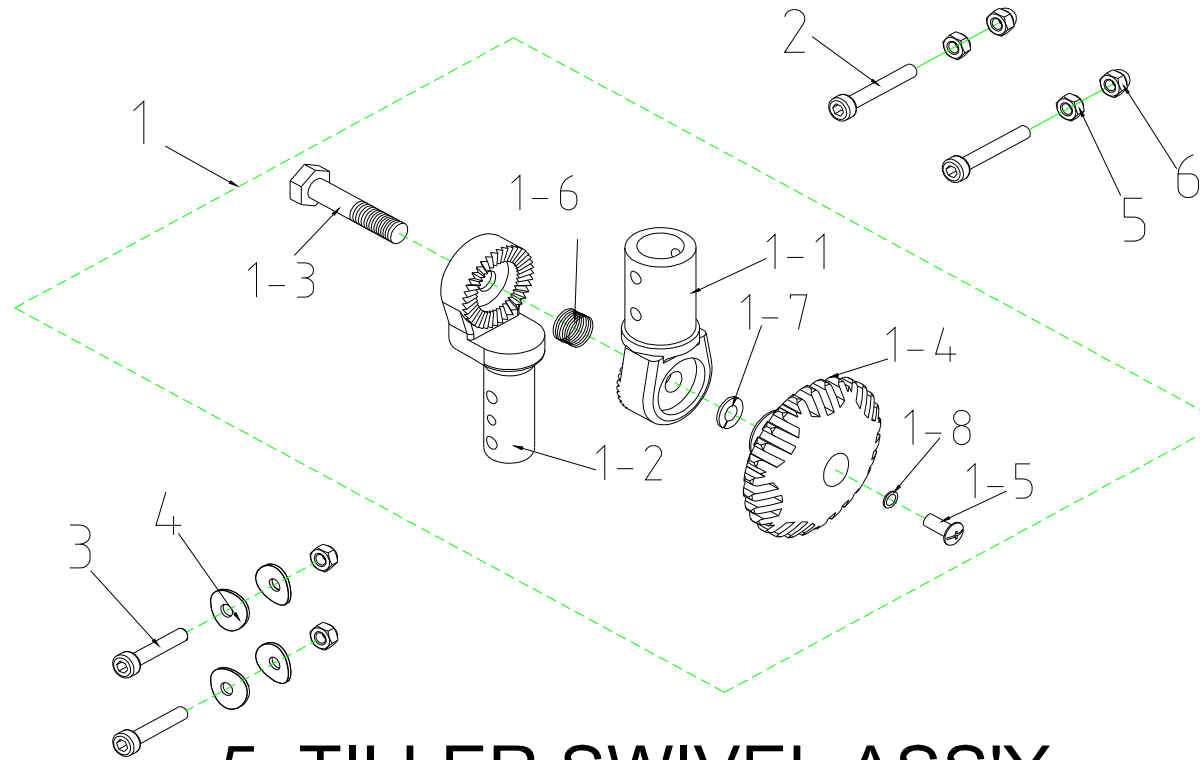


**5. TILLER SWIVEL ASS'Y**

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>5</b>	<b>S02-060-00100</b>	<b>TILLER SWIVEL ASSY</b>	<b>1SET</b>	
5-1	C02-050-00500	TILLER SWIVEL ASS'Y	1	
5-1-1	C02-050-00600	TILLER SWIVEL,UPPER	1	
5-1-2	C02-050-00700	TILLER SWIVEL,LOWER	1	
5-1-3	9036100501503	BOLT,TILLER,M10x28L	1	
5-1-4	C02-026-00100	NUT,TILLER SWIVEL	1	
5-1-5	9005050120802	CR ROUND HD SCREW,M5X12L	1	
5-1-6	C02-09-00700	SPRING,TILLER SWIVEL	1	
5-1-7	9211100200172	SPRING WASHER , M10	1	
5-1-8	9211050100802	SPRING WASHER , M5	1	
5-2	9035060451003	BOLT,M6X45(ED)	2	
5-3	9035060401003	BOLT,M6X40(ED)	2	
5-4	9250060100137	WASHER,M6	4	
5-5	9201060601003	LOCK NUT,M6(ED)	4	
5-6	9205060951003	CAP NUT,M6	2	

T4J Parts List (2010/10/27)

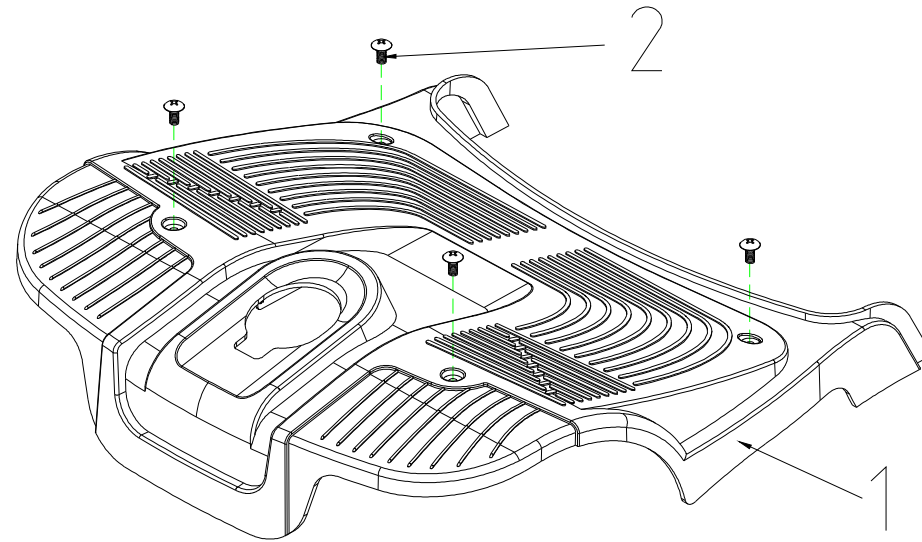


**6. FRONT SHROUD ASS'Y**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>6</b>	<b>S08-060-00500</b>	<b>FRONT SHROUD ASS'Y , BLACK</b>	<b>1</b>	
6-1	C08-060-01000	FRONT SHROUD , BLACK	1	
6-2	9005060151002	BOLT M6x15L	4	

T4J Parts List (2010/10/27)

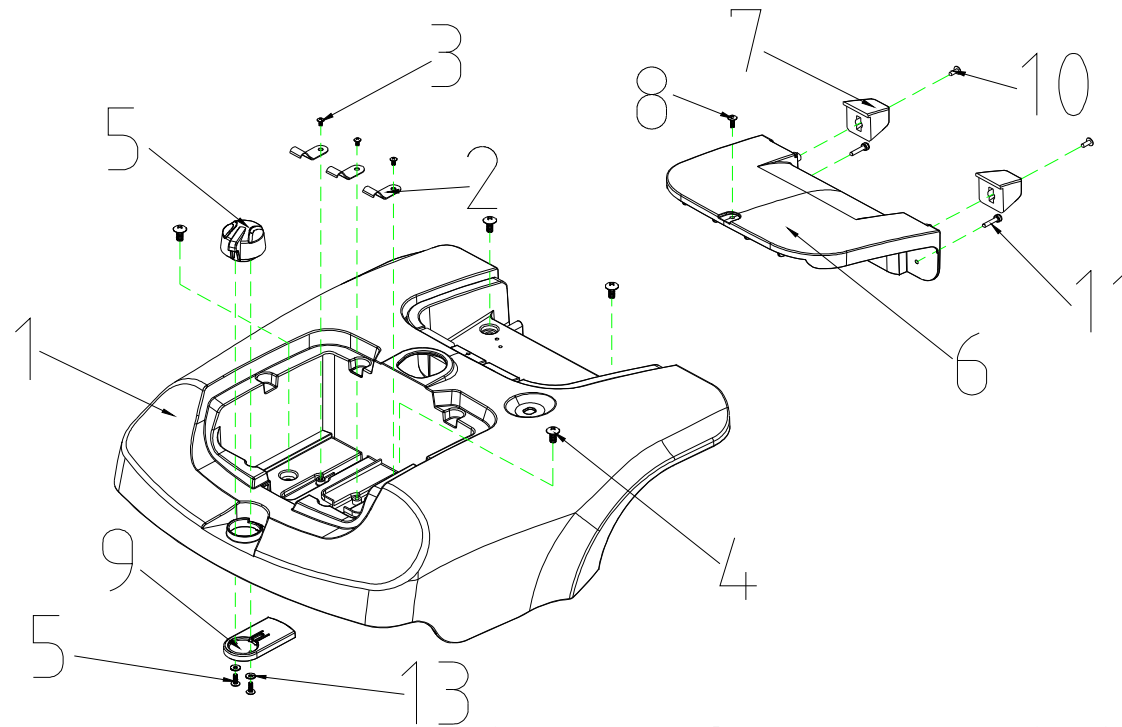

# 7. REAR SHROUD ASS'Y


## WT-T4J 售服零件表

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>7</b>	<b>S08-060-00453</b>	<b>REAR SHROUD ASS'Y , BLUE 53</b>	<b>1</b>	
7-1	C08-060-00253	REAR SHROUD , BLUE 53	1	
7-2	C12-009-00100	U-TYPE SPRING NUT	3	
7-3	9005040080703	BOLT M4x8L(ED)	3	
7-4	9005060151002	BOLT M6x15L	4	
7-5	C08-060-00800	KNOB , BATTERY BOX	1	
7-6	C08-060-00400	COVER , CONTROLLER	1	
7-7	C08-060-00300	PAD , REAR SHROUD	2	
7-8	9011040100003	BOLT , M4x10L	1	
7-9	C08-060-00900	HOLDER , BATTERY BOX	1	
7-10	9029040120002	BOLT , M4x12L	2	
7-11	9011040200003	BOLT , M4x20L(ED)	2	
7-12	9011350150004	BOLT , M3.5x15L	2	
7-13	9214040100102	WASHER , M4	2	



T4J Parts List (2010/10/27)

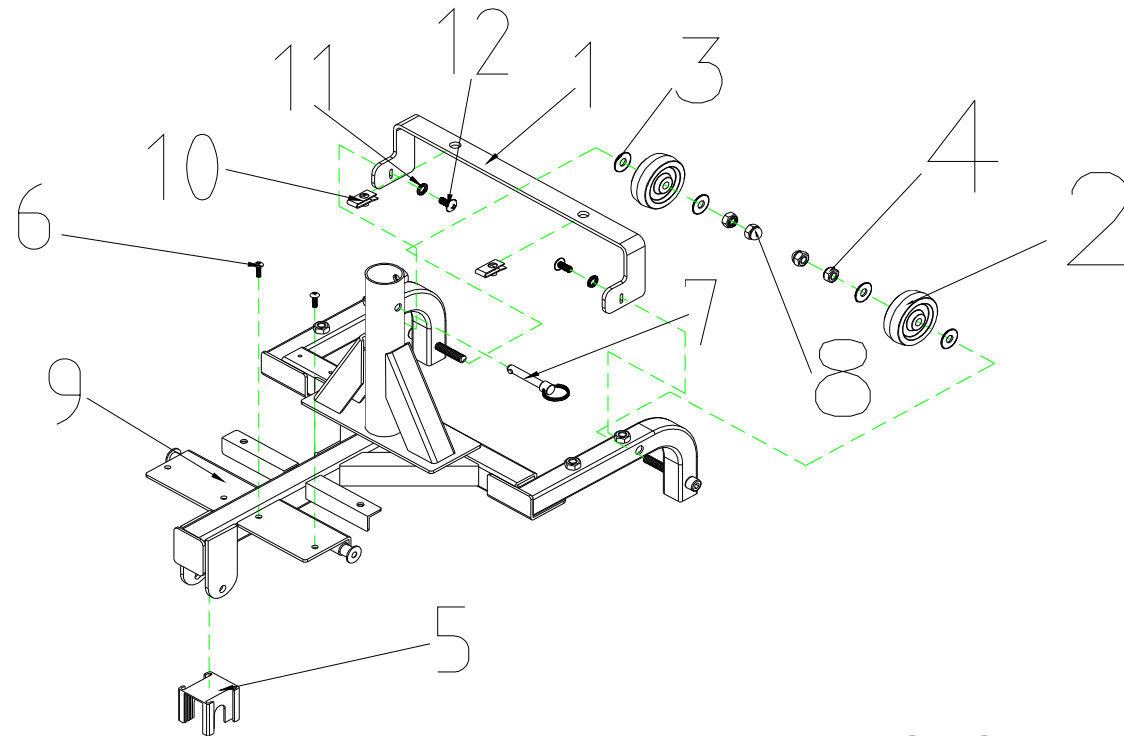


# 8. FRAME REAR ASS'Y

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>8</b>	<b>S01-060-00100</b>	<b>FRAME,REAR SECTION ASS'Y</b>	<b>1SET</b>	
8-1	C01-060-00300	REAR FRAME SECTION	1	
8-2	C28-007-00200	ANT-TIP WHEELS	2	
8-3	9214080100163	WASHER , M8	4	
8-4	9201080801253	LOCK NUT,M8(ED)	2	
8-5	C26-050-00100	FRAME CONNECT BUSH	1	
8-6	9005040150703	BOLT M4x15L(ED)	2	
8-7	C01-001-10501	LOCK PIN , M8x50L	1	
8-8	9205081251253	LID NUT-M8XP1.25	2	
8-9	C01-060-00203	FRAME , REAR SECTION	1	
8-10	9237060000001	U-TYPE SPRING NUT,M6	2	
8-11	9211060150103	DPRINT WASHER , M6	2	
8-12	9005060151002	BOLT M6x15L	2	

T4J Parts List (2010/10/27)

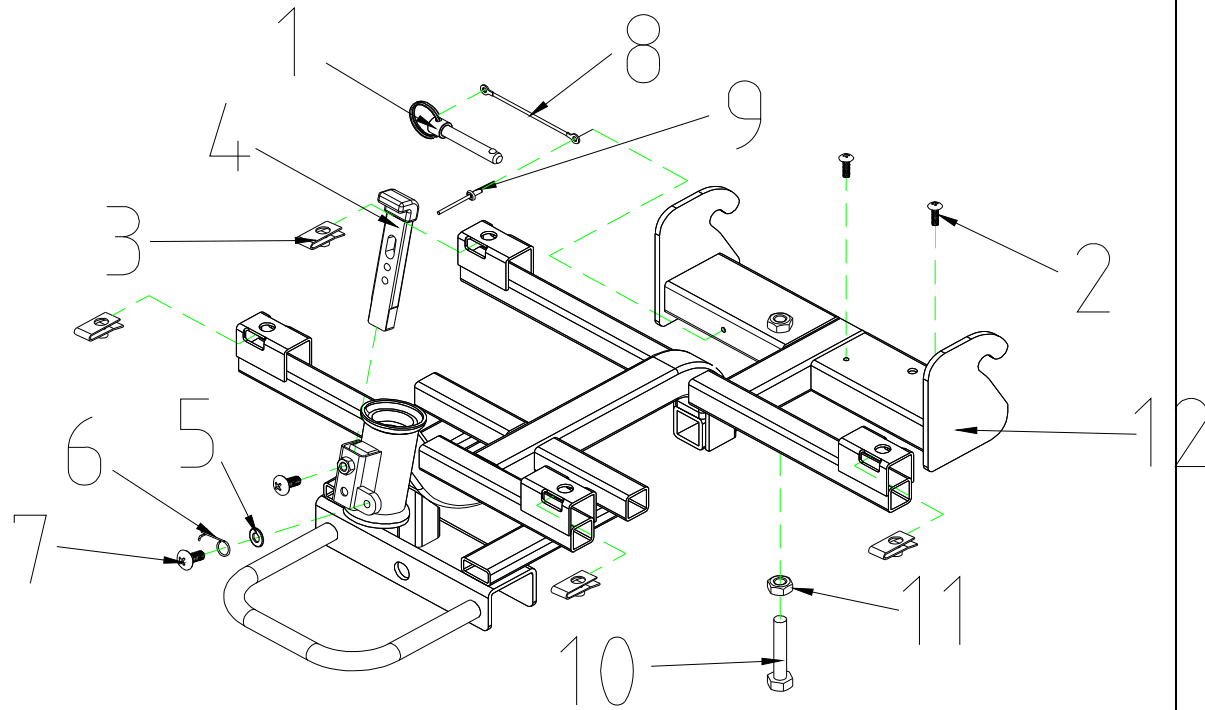


# 9. FRAME FRONT ASS'Y

## WT-T4J 售服零件表

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>9</b>	<b>S01-060-00400</b>	<b>FRAME,FRONT SECTION ASS'Y</b>	<b>1</b>	
9-1	C01-001-10501	LOCK PIN , M8X50	1	
9-2	9005040150703	BOLT M4x15L(ED)	2	
9-3	9237060000001	U-TYPE SPRING NUT , M6	4	
9-4	C02-010-00101	LOCKER	1	
9-5	9214060100133	WASHER,M6	1	
9-6	C02-009-00300	SPRING,LOCKER	1	
9-7	9005060121002	BOLT,M6X12	2	
9-8	C01-050-00700		1	
9-9	C36-001-00100	PIN	1	
9-10	9002080501251	HEX FLANGE BOLT, M8x50L	1	
9-11	9202080601251	NUT,M8	1	
9-12	C01-060-00400	FRAME,FRONT SECTION	1	

T4J Parts List (2010/10/27)

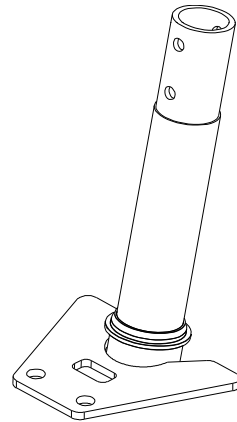


**10.L TYPE STEERING AXLE ASSY**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
10	C03-060-00200	L TYPE STEERING AXLE ASS'Y	1	

T4J Parts List (2010/10/27)

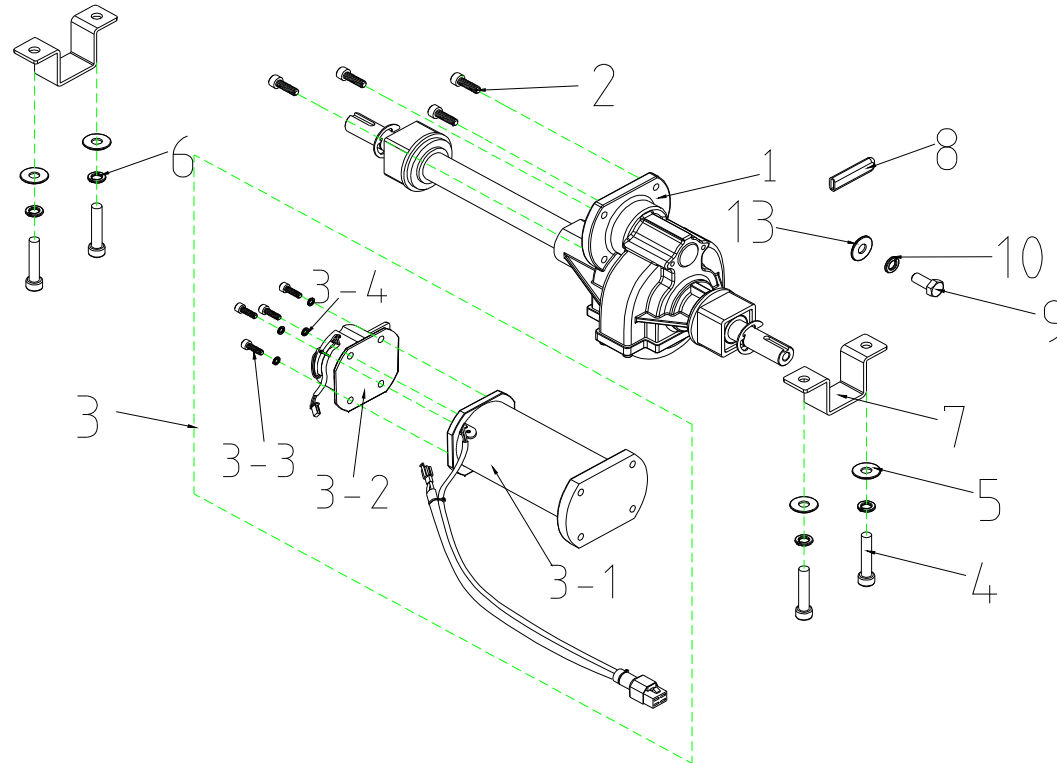


**11. REAR DRIVE ASS'Y  
( W/I BOLT / NUT )**

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>11</b>	<b>S09-050-00100</b>	<b>REAR DRIVE ASS'Y(W/ BOLT,NUT ETC.),7μM 4700RPM</b>	<b>1</b>	
11-1	C09-050-00100	TRANSAXLE ONLY (WITHOUT MOTOR)	1	
11-2	9022060201001	BOLT,BLACK,M6x20L	4	
11-3	C09-009-00604	MOTOR (WITH BRAKE) , 7μM/4700RPM	1	
11-3-1	C09-009-00302	MOTOR ONLY(WITHOUT BRAKE) , 7μM/4700RPM	1	
11-3-2	C09-009-00401	ELECTRO-MAGNETIC BRAKE	1	
11-3-3	9004040120704	BOLT,M4x12L	4	
11-3-4	9211040100602	SPRING WASHER,M4	4	
11-4	9035080401251	BOLT,M8x45L	4	
11-5	9214080150232	WASHER,5/16	6	
11-6	9211080200142	SPRING WASHER , OD=14mm ID=8mm T=2mm	4	
11-7	C09-009-00200	PLATE,MOTOR	2	
11-8	C09-050-00200	PLUG	1	
11-9	9001080201253	BOLT,M8x20L	2	
11-10	9211080200143	SPRING WASHER , OD=14mm ID=8mm T=2mm(ED)	2	



T4J Parts List (2010/10/27)

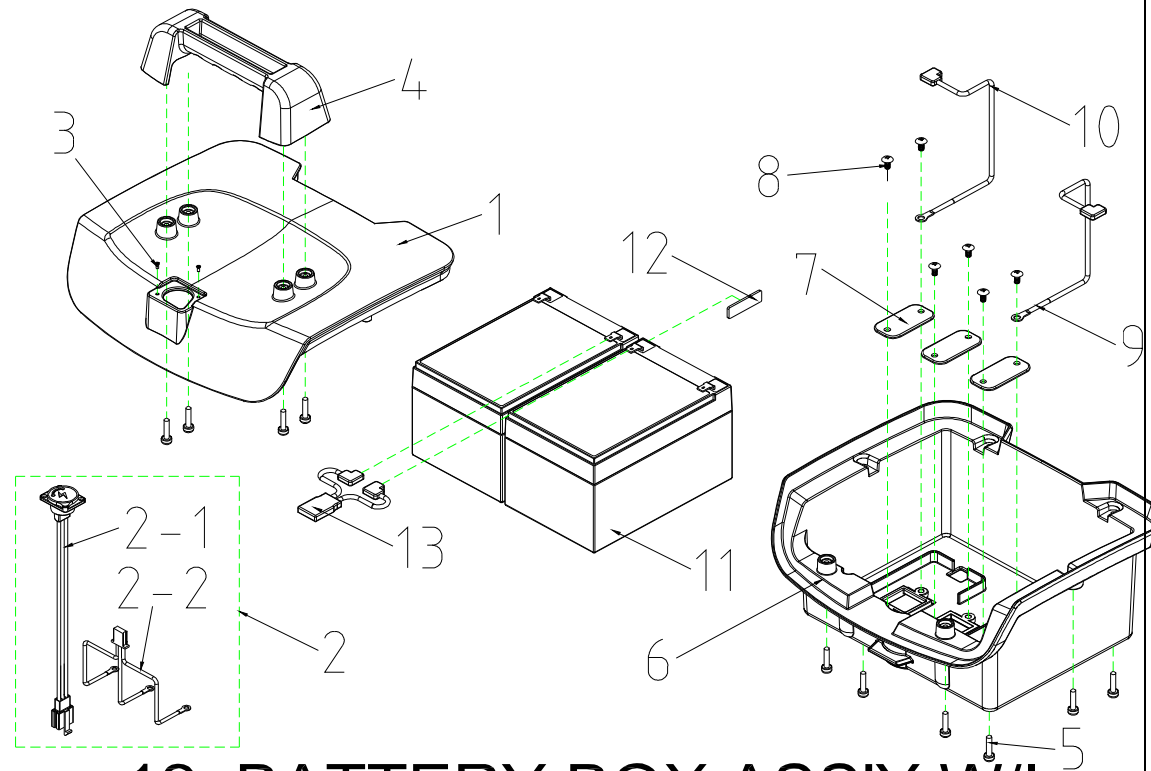


**12. BATTERY BOX ASS'Y W/  
BATTERY**

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>12</b>	<b>S17-060-00100</b>	<b>BATTERY PACK ASS'Y , W/I BATTERY</b>	<b>1</b>	1906912
12-1	C08-060-00500	COVER ,UPER , BATTERY	1	waarom zegt hij dan tegen mij niet
12-2	C17-060-00101	CHARGE CABLE ASS'Y	1	
12-2-1	C17-060-00400	CHARGE CABLE , UPPER	1	
12-2-2	C17-060-00500	CHARGE CABLE , LOWER	1	
12-3	9014250080002	CR PAN HD SCREW , M2.5x8L	2	
12-4	C08-060-00700	BATTERY CASE GRIP	1	
12-5	9011040200003	BOLT M4x20L(ED)	10	
12-6	C08-060-00601	COVER ,LOWER , BATTERY	1	
12-7	C12-009-00400	COPPER PIECE,BATTERY PLANK	3	
12-8	9011040100003	CR ROUND HD SCREW,M4X10L	6	
12-9	C17-009-00700	BATTERY CABLE ( + )	1	
12-10	C17-009-00800	BATTERY CABLE ( - )	1	
12-11	C17-009-01200	BATTERY 12AH	2	
12-12	C33-001-00200	SPONGE ADHESIVE,60X18X3mm	1	
12-13	C17-009-00600	BATTERY CABLE	1	

T4J Parts List (2010/10/27)

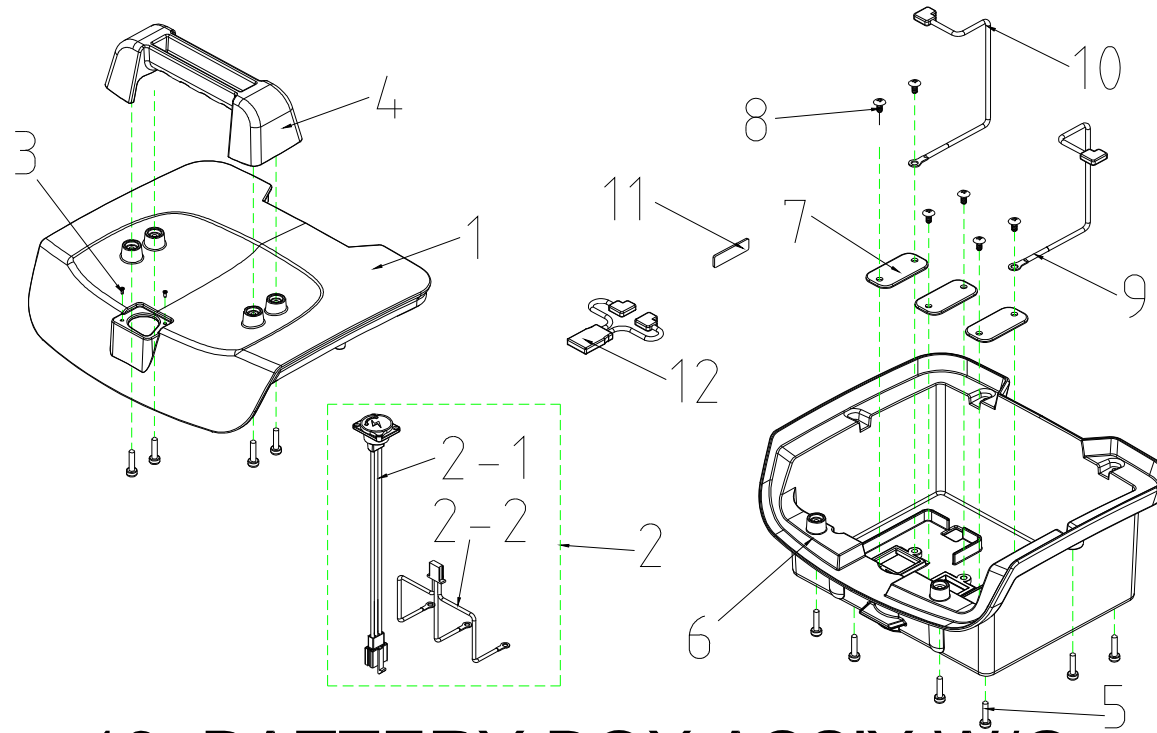


**13. BATTERY BOX ASS'Y W/O BATTERY**

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>13</b>	<b>S17-060-00200</b>	<b>BATTERY PACK ASS'Y , W/O BATTERY</b>	<b>1</b>	
13-1	C08-060-00500	COVER ,UPER , BATTERY	1	1906373
13-2	C17-060-00101	CHARGE CABLE ASS'Y	1	
13-2-1	C17-060-00400	CHARGE CABLE , UPPER	1	
13-2-2	C17-060-00500	CHARGE CABLE , LOWER	1	
13-3	9014250080002	CR PAN HD SCREW , M2.5x8L	2	
13-4	C08-060-00700	BATTERY CASE GRIP	1	
13-5	9011040200003	BOLT M4x20L(ED)	10	
13-6	C08-060-00601	COVER ,LOWER , BATTERY	1	
13-7	C12-009-00400	COPPER PIECE,BATTERY PLANK	3	
13-8	9011040100003	CR ROUND HD SCREW,M4X10L	6	
13-9	C17-009-00700	BATTERY CABLE ( + )	1	1906371
13-10	C17-009-00800	BATTERY CABLE ( - )	1	1906372
13-11	C33-001-00200	SPONGE ADHESIVE,60X18X3mm	1	
13-12	C17-009-00600	BATTERY CABLE	1	1906374

T4J Parts List (2010/10/27)

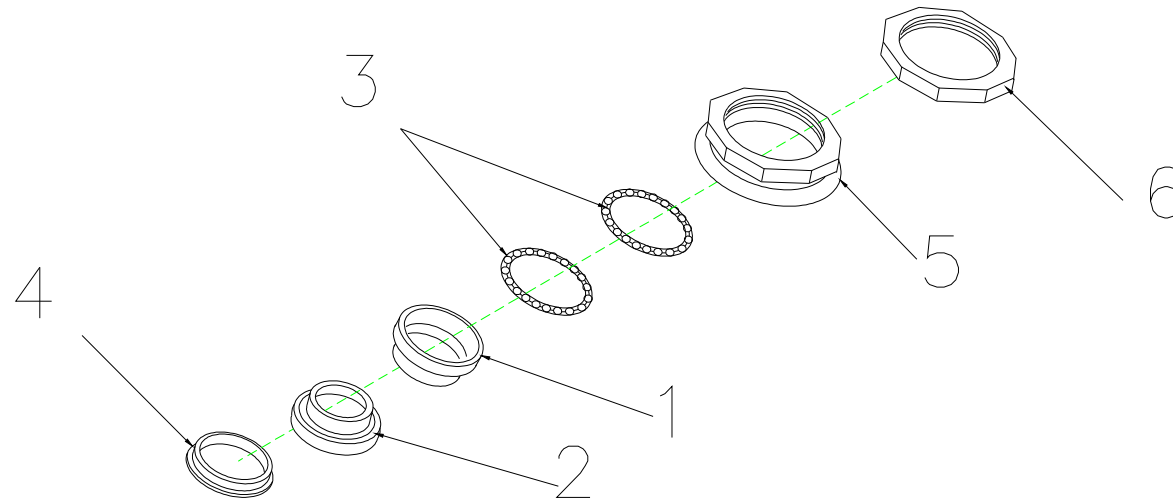


**14.REGULAR BEARING BALL SET**

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>14</b>	<b>S02-060-00200</b>	<b>REGULAR BEARING BALL SET</b>	<b>1SET</b>	
14-1	C29-016-00101	CUP , UPPER	1	
14-2	C29-016-00201	CUP , LOWER	1	
14-3	C29-016-00301	BEARING	2	
14-4	C29-016-00401	BEARING CAGE	1	
14-5	C29-060-00100	BEARING BUSH	1	
14-6	C29-016-00601	LOCKING NUT , BEARING	1	

T4J Parts List (2010/10/27)

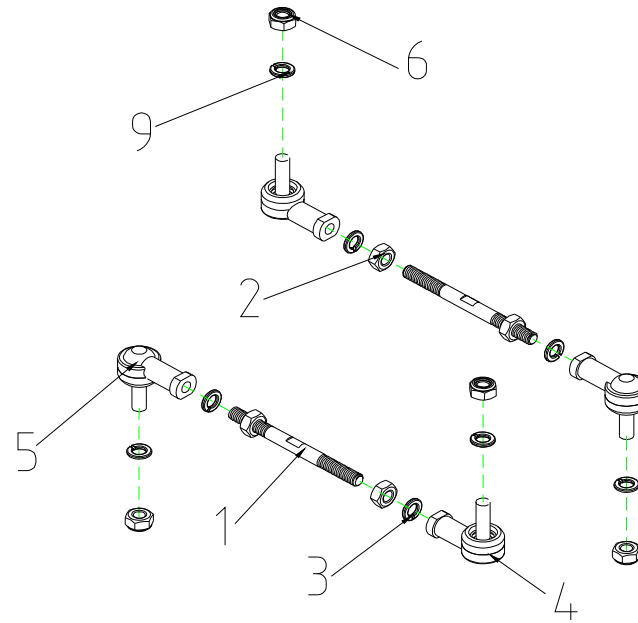


# 16. CONNECTING ROD ASS'Y

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>16</b>	<b>S03-060-00200</b>	<b>CONNECTING ROD ASS'Y</b>	<b>1</b>	
16-1	C03-060-00600	CONNECTING ROD, GUIDING	2	
16-2	9202080601251	NUT, M8XP1.25	2	
16-3	9211080200142	SPRING WASHER, 5/16	8	
16-4	C19-001-00101	JOINT, RBL 8 (L)	2	
16-5	C19-001-10801	JOINT, RBL 8 (R)	2	
16-6	9201080801252	LOCKING NUT, M8	4	



T4J Parts List (2010/10/27)

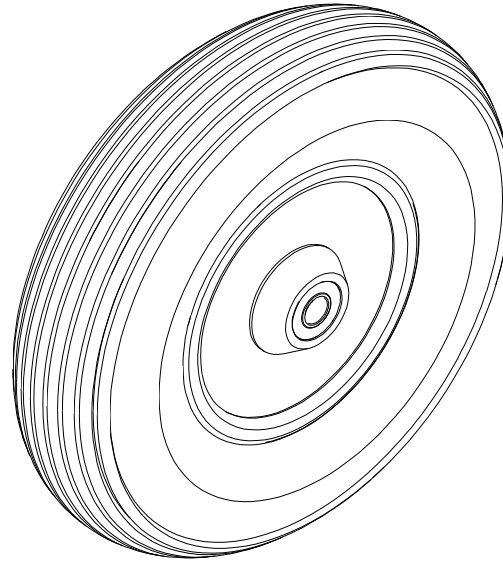


17.FRONT PU FOAMED TIRE ASS'Y

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
17	C05-034-00600	FRONT PU FOAMED TIRE , BLACK	1	

T4J Parts List (2010/10/27)

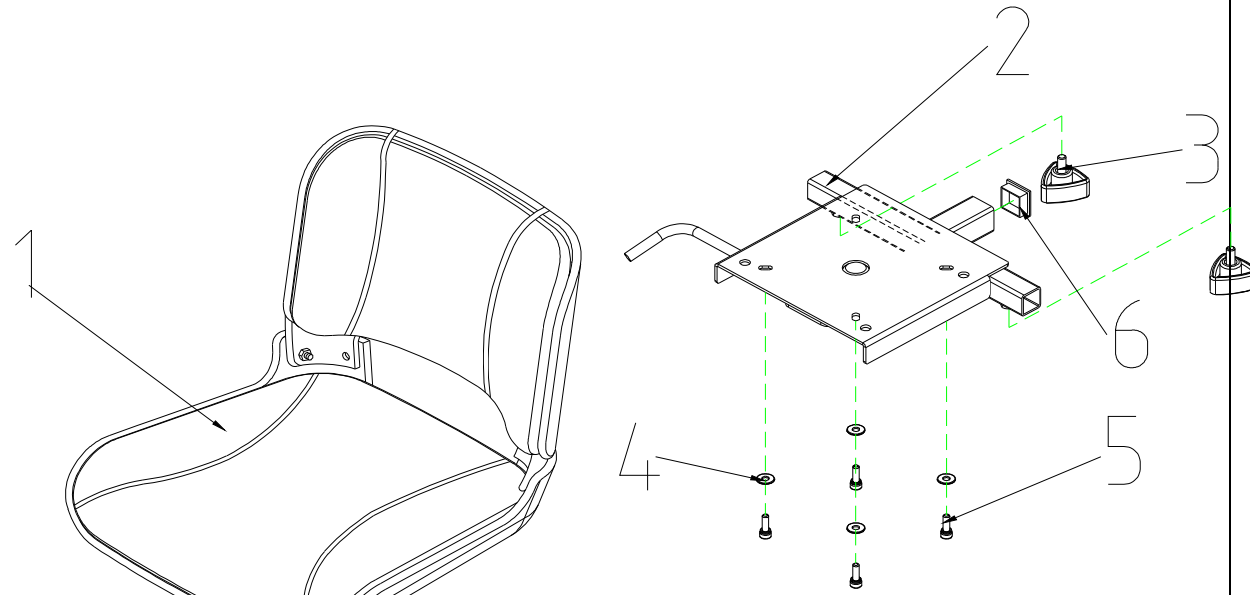


**19. SEAT ASS'Y**

## WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>19</b>	<b>S07-060-00200</b>	<b>SEAT ASS'Y</b>	<b>1</b>	
19-1	C07-060-00300	SEAT, BLACK	1	
19-2	C07-060-00100	SEAT PLATE, T3J	1	
19-3	C30-001-10202	TRIANGEL BOLT, M8x20	3	
19-4	9214070100193	WASHER , Ø19xØ7x1L(ED)	4	
19-5	9022060201001	BOLT M6x20L	4	
19-6	C26-001-00401	PLUG	1	

T4J Parts List (2010/10/27)

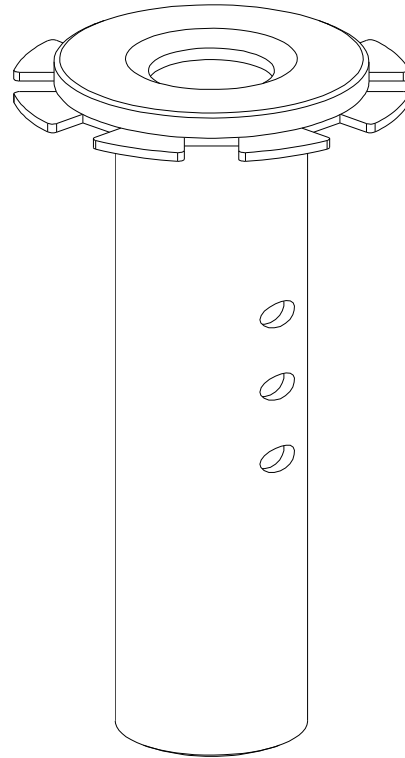


**20. SEAT POST**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
20	C07-060-00500	SEAT POST	1	

T4J Parts List (2010/10/27)

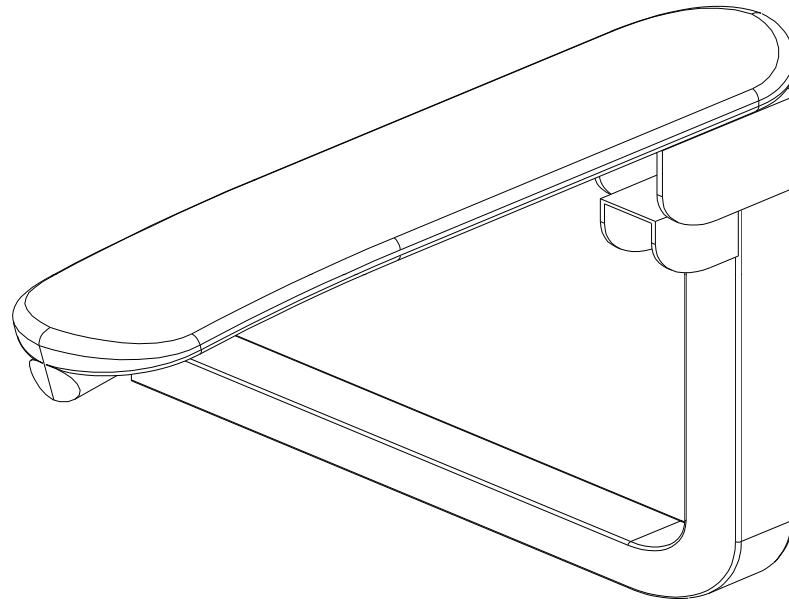


**22. ARMREST , L**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
22	C07-027-00601	ARMREST,L	1	



T4J Parts List (2010/10/27)

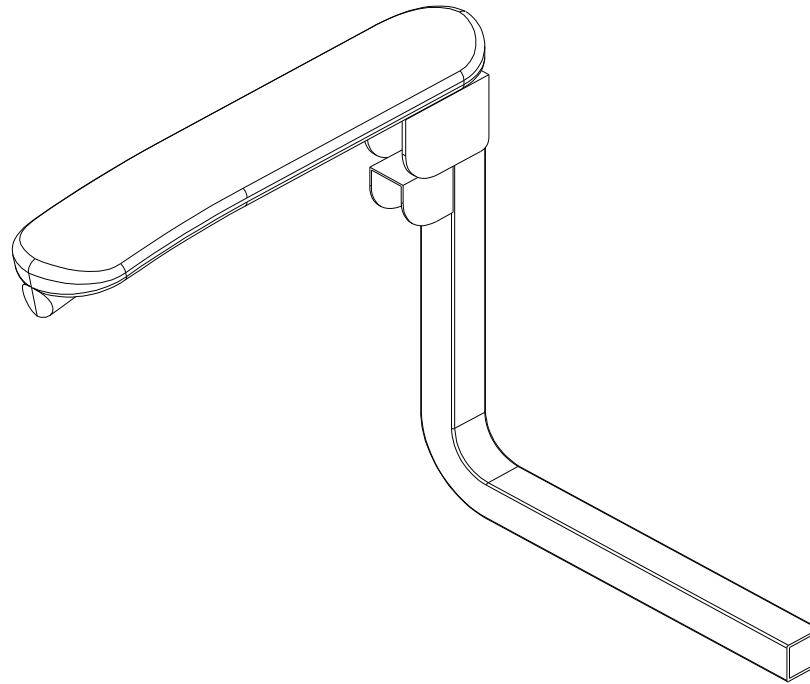


**23.ARMREST,R**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
23	C07-027-00701	ARMREST, R	1	

T4J Parts List (2010/10/27)

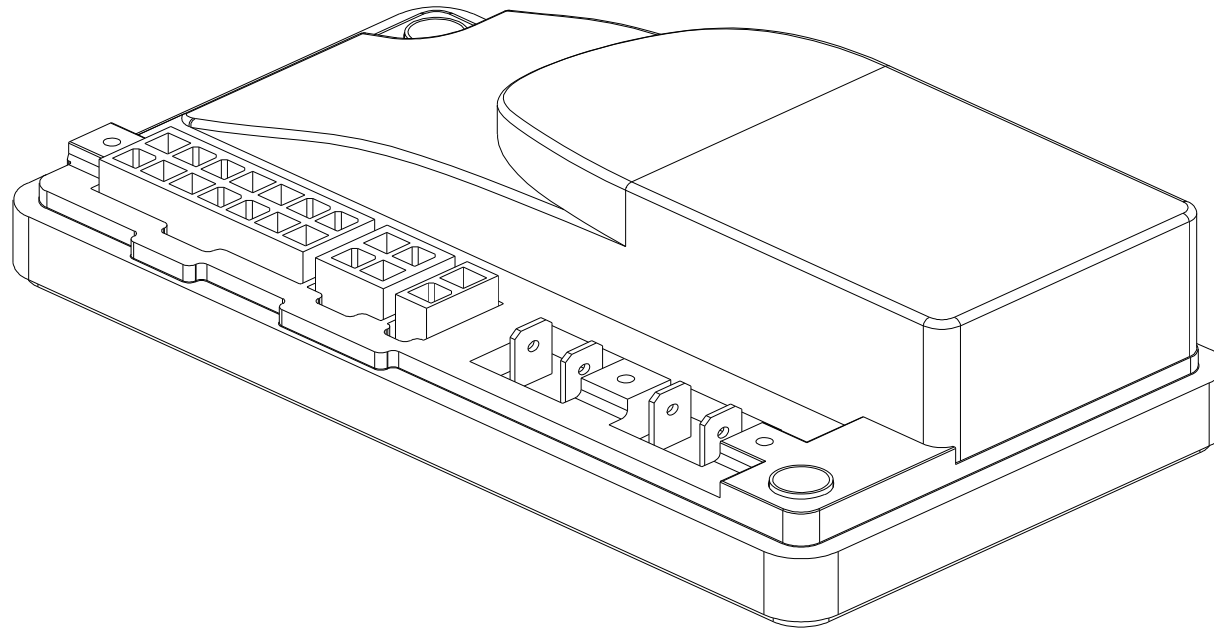


**24. CONTROLLER**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
24	C10-060-00100	CONTROLLER , CURTIS 45A	1	1906370

T4J Parts List (2010/10/27)

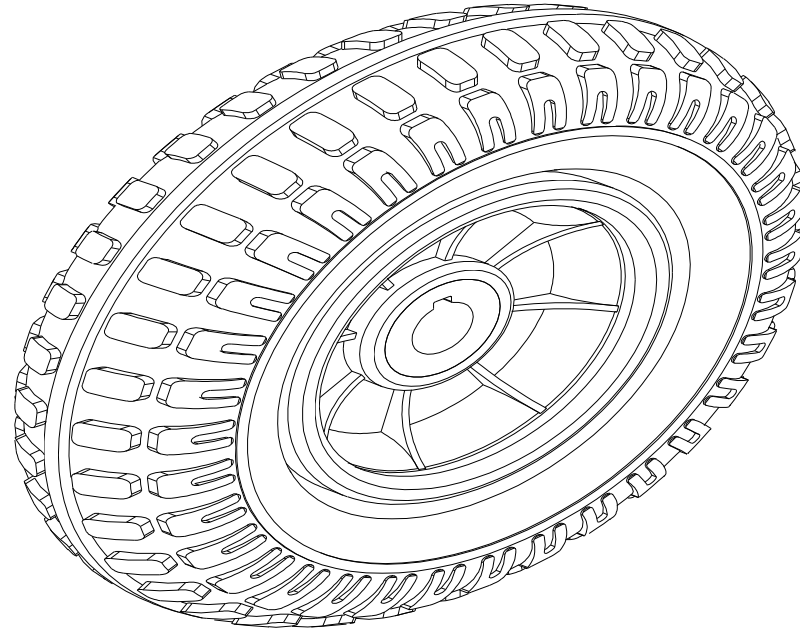


**25. 200x50 PU FOAMED TIRE**

# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
25	C06-034-00401	200x50 PU FOAMED TIRE , REAR	1	

T4J Parts List (2010/10/27)

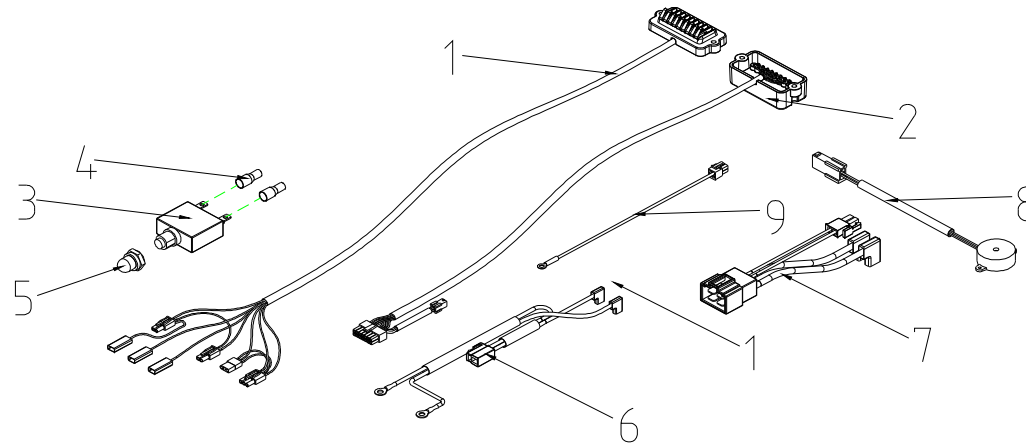

# 26. WIRE HARNESS ASS'Y

## WT-T4J 售服零件表

## T4J Parts List (2010/10/27)

Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
<b>26</b>	<b>S18-060-00100</b>	<b>WIRE HARNESS ASS'Y W/O BUZZER</b>	<b>1</b>	
26-1	C18-060-00100	WIRE HARNESS ASSY,FRONT	1	
26-2	C18-060-00300	WIRE HARNESS ASSY,REAR	1	
26-3	C16-009-00302	BREAKER , 20A	1	
26-4	C26-001-00800	PLUG	2	
26-5	C16-009-00401	TUBE , PROTECT SWITCH	1	
26-6	C17-060-00300	POWER CABLE	1	
26-7	C17-060-00200	MOTOR CABLE	1	
26-8	C12-050-00500	BUZZER	1	
26-9	C17-060-00600	INHIBIT CABLE	1	



# WT-T4J 售服零件表

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
28-1	C23-009-00200	BASKET	1	
28-2	C20-007-00200	GRIP , 120mm	1	
28-3	C12-034-00300	CHARGER , CABLE,CHARGER , EU	1	
28-6	C37-060-00200	OWNER'S MANUAL	1	
28-11	C99-060-00200	CARTON , BE 01	1	

T4J Parts List (2010/10/27)				
Fig&Index NO.	Parts No.	Parts Name	QTY/CAR	Remark
		Description		
29-1	C27-060-00100	LABEL , SPEED , NO LOGO	1	
29-6	C27-009-05600	LABEL , KEY ON/OFF	1	
29-7	C27-060-00200	LABEL , LOGO , REAR SHROUD , BE 01	1	
<b>29-10</b>	<b>S27-060-00300</b>	<b>LABEL ASS'Y,REAR SHROUD , BE 01</b>	<b>1</b>	
29-10-1	C27-009-01700	LABEL , EMI	1	
29-10-2	C27-001-00200	LABEL , CE	1	
29-10-3	C27-001-01900	LABEL , NEVER LIFT UP	1	
29-10-4	C27-060-00300	LABEL , LOCK / UNLOCK	1	
29-10-5	C27-027-00600	FREEWHEEL-ENGAGED LABEL		