Option SE39, SE40, SE41:

“Electrical seat, Backrest”

Your wheelchair is equipped with an electrical seat, backrest. This option allows you to adjust the inclination of the seat and backrest in an easy/fast way.

1. Technical Specifications

Description	Dimensions
Angle adjustment seat	Navix: 0° - 15° (stepless) Timix: 3° - 22° (stepless) Forest 3: 3° - 22° (stepless)
Angle adjustment backrest	0° - 30° (stepless)
Storage temperature electronics	-20°C - + 40°C
Usage temperature electronics	+5°C - + 40°C

Table 1

2. Explanations of symbols

CE conformity

3. Safety rules

- ⚠ If the actuator is defect contact your specialist dealer.
- ⚠ All cables should be laid without making contact to the floor and without kinks.
- ⚠ Ensure that no cables become squeezed.
- ⚠ Never take any obstacles, slopes or perform long drive distances when the seat inclined more than 10°, since this could cause serious damage, injury and/or risk of stability.
- ⚠ The minimal inclination (0°: Navix RWD / 3°: Forest 3) is not meant to be for driving with the wheelchair. In this position the front wheels touch the seat frame. This position is foreseen for getting out the wheelchair. Better to drive with a seat angle off 5° or more (Not for Timix and Navix FWD).

5. Use

You can adjust the backrest or seat inclination with your operator control (electrical adjustment). Refer to the manual of the operator control for the backrest or seat inclination instructions. Refer to table 1 for the different ranges.

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