VERMEIREN

PEPI

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ

EN

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CONTENTS

Section	Page
Description	2
Usage	2
Upholstery	3
Settings of the desired shape	3
Adjustment of the seat	3
Components	3
Technical data	3
Safety	4
Cleaning and maintenance	4
Storage	4
Disposal	5
Guarantee	5
Conformity	5

Dealer information:

This instruction manual is part and parcel of the product and it should accompany each product delivered.

Last updated: 04/2010

All rights reserved, including translation.

No part of this manual may be reproduced in any form whatsoever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated, or distributed by using electronic systems.

© N.V. Vermeiren N.V., 2009

DESCRIPTION

The bath deckchair for children, PEPI model, was made in order to satisfy your expectations in the sector of products supporting children's hygiene.

The construction of the PEPI deckchair is made of aluminium pipes that ensure stable and failure-free use for a long time. Thanks to its versatility of positioning, the product can be fully adapted to the user's needs.

The PEPI deckchair is to be used in a bathtub or under a shower in order to support the stabilisation of children at the age of 2 or more years. It is particularly recommended for children with:

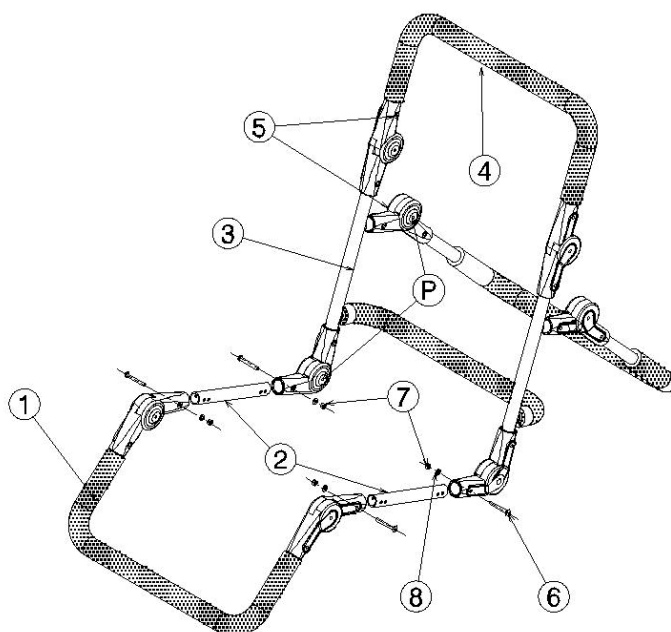
- Infantile cerebral palsy,
- Paralysis,
- Injury or amputation of limbs,
- Degeneration or deformation of limbs,
- Degeneration or injury of joints,

The individual shape should be selected in consideration of:

- Height and body weight (maximum load: 30 kg),
- Physical and mental fitness,
- Terms of use.

USAGE

As a supplementary device during bath in a bathtub or under a shower, the PEPI bathing deckchair can be adapted optimally to the child's needs in any position (either sitting or lying) due to the adjustment of the inclination angle of the four segments: foot-rest (1), seat (2), back-rest (3) and head-rest (4). The areas where the deckchair contacts the ground are protected with foam covers so as not to damage the bathtub surface.

UPHOLSTERY

The frame is padded with upholstery made of water-permeable sieve cloth. The pressure of the cloth can be adjusted with belts with velcro fastenings.

The stabilisation of the child is ensured by belts with velcro fastenings, which make it possible to adjust the pressure easily to the child's needs, thus protecting the child against consequences of uncontrolled movements. These belts can be relocated in any manner thanks to two sewn-in belts running along the entire deckchair. The deckchair is provided with a hip belt and two belts for stabilisation of legs.

SETTINGS OF THE DESIRED SHAPE

Hinges (5) are used for setting the desired angle between parts of the deckchair frame. In order to change the angle of the selected frame segment, press the button of the relevant hinge (at the same time on both sides of the frame) and set the desired angle, and then release the buttons.

The headrest can be inclined backwards at any angle for easier movement of the head. This solution makes it also easier to wash the child's head. If there is a need to immobilise the head, this is done by means of two braces attached to two belts running along the entire deckchair. If the use of braces is not required, the headrest part is covered by soft foam so that the child could rest its head safely and comfortably.

ADJUSTMENT OF THE SEAT

The deckchair construction makes it possible to adjust the depth of the seat. In order to set the desired depth of the seat, unscrew 4 screw caps (7) and remove four screws (6) from hinges (5), and then change the position of hinges on the seat pipe (2) as appropriate so that one of the two holes on both ends of the pipe (2) suits the relevant hole of the hinge (5), then put the screws (6) again in hinge holes, place the pads and screw caps (8 and 7) and fasten the whole construction tightly.

COMPONENTS

1. Aluminum frame
2. Sieve cloth upholstery
3. Two belts for stabilisation of legs
4. Hip belt
5. Two head-stabilising braces
6. Operating manual

TECHNICAL DETAILS

(Given for the fully-assembled, standard version). Please note that adding accessories will alter the technical details accordingly.

	PEPI
Total width	38 cm
Seat depth	20 cm
Length of the total lying area	110 cm
Length of the support	63 cm
Maximum load	30 kg
Weight	3,7 kg

We reserve the right to introduce technical changes. Differences of +/- 1.5 cm/kg, 1.5° are possible.

SAFETY

-  The PEPI bath deckchair is a therapeutical device and should be used under supervision!
-  Use this deckchair only on flat and even surfaces!

CLEANING AND MAINTENANCE

In order to retain the proper aesthetic value of your product, it is necessary to ensure its proper maintenance. The following guidelines will be useful for the fulfilment of this aim.

Upholstery

Upholstery is removed and can be washed in washing machines (at a maximum temperature of 60°C). Washing in sheaths or cloths is recommended.

-  In case of strong contamination, clean manually with the use of soft detergents.
-  Do not clean upholstery with very strong detergents, such as solvent or caustic agents; use of hard brushes is not recommended, either.
-  We are not responsible for any damages arising after the use of improper detergents.
-  Jet-steam cleaning is not recommended.

Plastic elements

Such details should be cleaned manually with generally available products upon reading of the guidelines of the manufacturer of such product.

External coating

The applied anodic coat is important due to its high aesthetic value and anti-corrosive properties. These coats do not require special cleaning (it is sufficient to wipe them with a damp or dry cloth).

Disinfection

Disinfection can be performed with the use of generally available products upon reading of the instructions of the manufacturer of such product. Please remember that they may penetrate into the upholstery.

-  During the disinfection, use protective gloves and follow the instructions of the manufacturer of such product.
-  The manufacturer of the product being disinfected is not responsible for any consequences of improper use of disinfectants.

STORAGE

- Store in a dry place (between + 5 °C and + 45 °C).
- The relative humidity of the air should be between 30% and 70%.
- Ensure there is protection against corrosion (e.g. salt water, sea air, sand, dust) by providing sufficient covering or packaging.
- Store all removed parts together in one place (or mark them if necessary) to avoid mixing up with other products when re-assembling.
- No component must be stored with a heavy weight on it.

DISPOSAL

To dispose of the PEPI, contact a local waste disposal company or return the product to your dealer who, after some hygiene measures, will be able to return it to the manufacturer, who will take dispose of it and recycle it properly, separated into material groups.

Packaging materials can be taken to suitable tips or recycling centres, or to your dealer.

Your dealer will also be able to advise you should you have any queries.

GUARANTEE

Excerpt from the "General business conditions":

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

Claims are invalid when:

- the agreed condition is only minimally deviated from
- usability is only minimally impaired
- wear and tear is normal
- faults are due to improper assembly or neglected maintenance
- faults are due to improper use
- errors are due to improper handling by customers or their contractual partners or service centres
- damage that arises after the transfer of risk as a result of faulty or negligent handling, excessive demands, unsuitable equipment, faulty assembly or that are due to particular external factors that are not assumed under the contract.

(...)

CONFORMITY

The PEPI complies with the requirements of the European directive:

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- (DIN) EN 12182: 1999 (dle platnosti)
- (DIN) EN 12183: 1999 (dle platnosti)

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:

Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com