Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKA PRO INSTALACI

Verso

Contents

EN

Preface	4
1. This product	5
2. Scope of delivery	6
3. Adjustments	6
3.1. Tools	7
3.2. Seat depth	8
3.3. Footrest length	10
3.4. Arm pad depth	11
3.5. Operating console depth	11
3.6. Armrest height	12

Preface

To support you on the installation and repairs of this powered wheelchair, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

- Verso

Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for wheelchairs For specialist dealer
	Drawings of (spare) parts For specialist dealer
	EC declaration of conformity

1. This product

1. Handgrips
2. Backrest
3. Armrests
4. Arm pads
5. Operating console
6. Seat
7. Safety belt
8. Leg rests or footrests
9. Footplates
10. Steering wheels (front wheels)
11. Driving wheels with motors / Rear wheels
12. Front light (optional)
13. Rear light (optional)
14. Battery
15. Cross
16. Identification plate

2. Scope of delivery

The following components are part of the delivery:

- Seat frame
- Drive train including controller and battery
- Armrests
- Footrests
- Battery charger
- Tools
- Manual
- Options

Before use, check if everything is included and that no products are damaged.

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

3. Adjustments

CAUTION

Risk of injuries or damage

- Only use the settings described in this manual.
- Only use parts described in this manual.
- Only use genuine spare or replacement parts approved by Vermeiren.
- Prevent bystanders from accessing the working area.
- Keep the (dis)assembling area clean and tidy, to prevent falling.
- Collect all small parts in a box or bag.
- While (dis)assembling and operating the wheelchair, make sure no objects or body parts get caught between moving parts. In particular, pay attention to the position of the cables.
- Turn the wheelchair off before you perform any mechanical adjustments and make sure that the free-wheel lever is in braking position.
- Make sure all screws are firmly secured before driving.


- Software changes are only allowed if executed by skilled personnel, taking into account the user and environment (stability, acceleration, indoor/outdoor, etc.).
- i** Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

3.1. Tools

To (dis)assemble and adjust the wheelchair, the following tools are needed:


- Wrench no. 10
- Allen key no. 3, 4, 5

3.2. Seat depth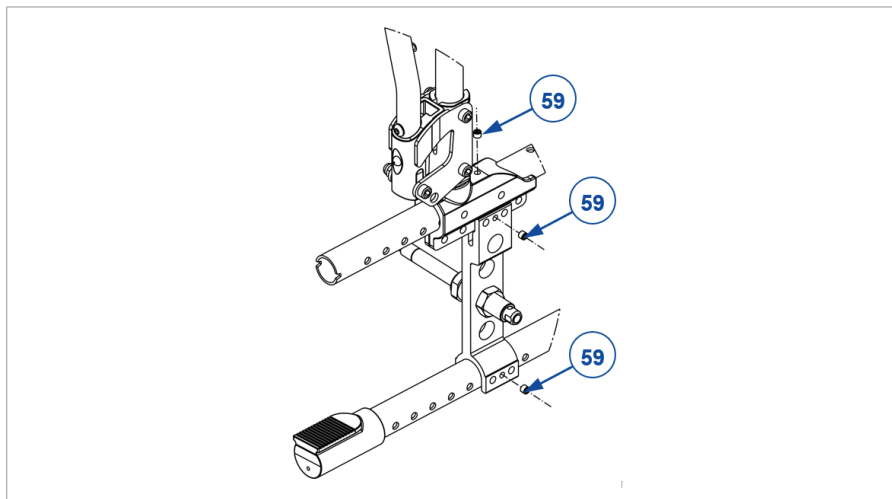

i Important remark

When adjusting the backrest connection, make sure that all 3 connection screws (item 59, in example) are tightened firmly. To prevent that theft-protection systems in shops react on the wheelchair frame, the connection screws (59) must be turned through the frame coating!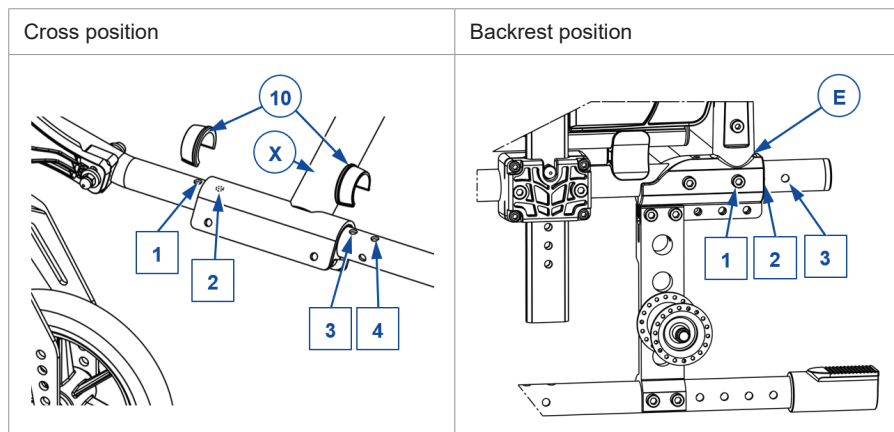

Seat depth	Cross position holes	Backrest position holes
440 mm	Hole 1 and 3	1
420 mm (Default)	Hole 2 and 4	1
480 mm	Hole 1 and 3	3
460 mm	Hole 1 and 3	2

3.2.1. Cross position (if adjustable)

1. Remove clip (10), at each side of the wheelchair.
2. Move the cross (X) forwards/backwards to the desired position according to the settings in the tables of previous sections.
3. Replace both clips (10) at each side of the wheelchair to lock the cross position.


3.2.2. Backrest connection position

1. Demount both socket screws of the backrest connection, at both sides of the wheelchair.
2. Move the backrest connection (A/B/C/D/E) forwards/backwards to the desired position according to the settings in the tables of previous sections.
3. Mount the 2 socket screws firmly; at both sides of the wheelchair.
4. Verify if seat height and seat angle are correctly set.

3.2.3. Brake position

Adjust the brake position according to the backrest connection position.


3.3. Footrest length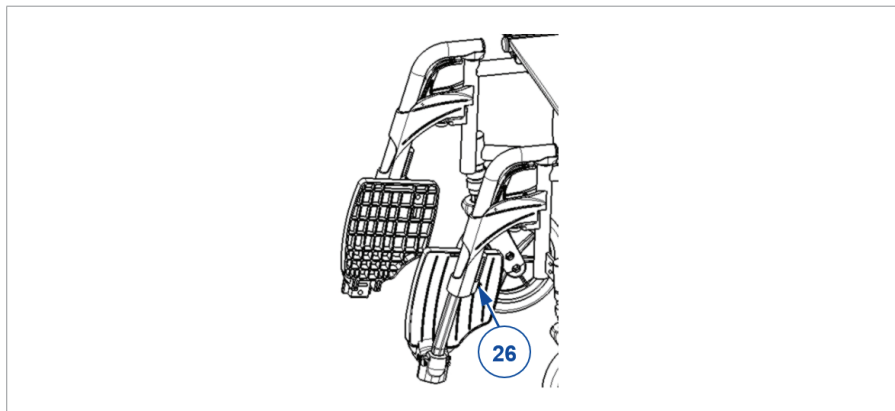

Adjust the length as follows:

1. Remove the socket screw (26) at the back of the footrest.
2. Adjust the footrest to a comfortable length.
3. Retighten the socket screw firmly.
4. Repeat this for the second footrest. Make sure the footrests are positioned identically.

3.4. Arm pad depth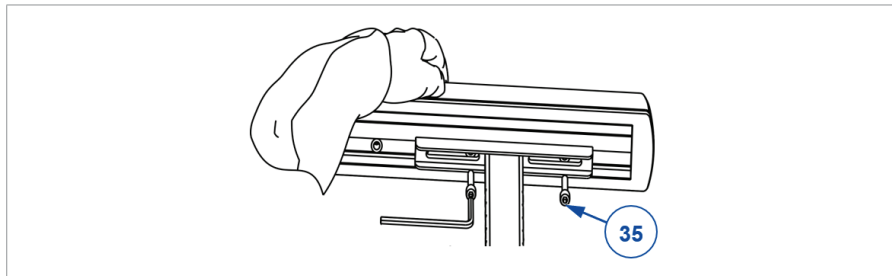


Adjust the depth of the arm pad as follow (range 120 mm: step less):

1. Loosen both socket screws (35) at the bottom of the arm pad.
2. Move the arm pad to the front or rear within the range of the slotted holes (42,5 mm).
If this not enough, remove the socket screws and attach the arm pad using the other holes (3 positions available).
3. Tighten both socket screws firmly.
4. Repeat this for the second arm pad. Make sure both arm pads are positioned identically.

3.5. Operating console depth

1. Loosen both socket screws (35) at the bottom of the arm pad.
2. Move the console attachment plate to the front or rear within the range of the slotted holes (50 mm).
3. Tighten both socket screws firmly.

3.6. Armrest height

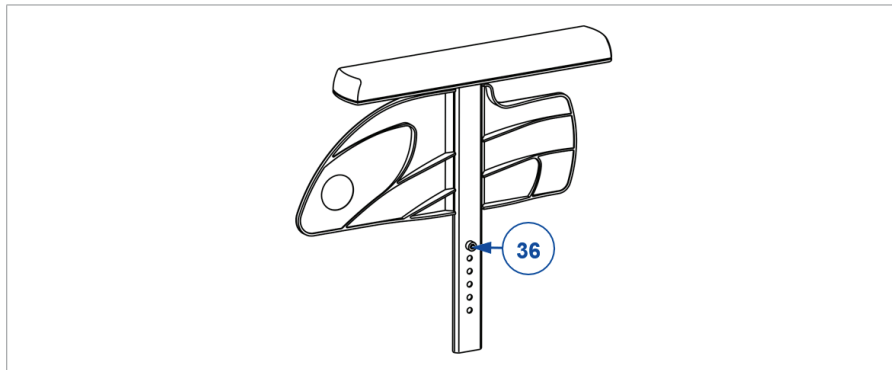The height of the armrest is adjustable in 6 positions as follows:

1. Pull the armrest out of the armrest holder.
2. Remove the socket screw (36) that sets the height.
3. Mount the screw in the desired hole.
4. Insert the armrest in the armrest holder.
5. Check the height; readjust if necessary.
6. Repeat this for the second armrest. Make sure both armrests are positioned identically.

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Positie van het kruis	Positie van de rugbevestiging
A technical drawing of a chair's crossbar assembly. It shows a horizontal bar with several adjustment points. Labels include '10' in a circle pointing to a bracket, 'X' in a circle pointing to a specific point, and numbers '1', '2', '3', and '4' in boxes pointing to different adjustment screws or points along the bar.	A technical drawing of a chair's backrest attachment mechanism. It shows a vertical support with a horizontal bar for the backrest. Labels include 'E' in a circle pointing to a top bracket, and numbers '1', '2', and '3' in boxes pointing to adjustment screws or points on the vertical support.

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Posizione della crociera	Posizione dello schienale
A technical drawing of a seat crossbar. It shows a horizontal bar with several adjustment points labeled 1, 2, 3, and 4. A locking mechanism is labeled 10. A circled 'X' indicates a specific adjustment point.	A technical drawing of a seat backrest assembly. Label 'E' points to the backrest. Labels 1, 2, and 3 point to adjustment points on the backrest frame.

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All rights reserved, including translation.