User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

2217E, 2219E

Contents

Preface	4
1. Your product	6
1.1. Options	7
2. Before use	8
2.1. Intended use	8
2.2. General safety instructions	9
2.3. Symbols on the tricycle	10
2.4. Storage	10
3. Using your tricycle	11
3.1. Operating the brakes	11
3.2. Adjusting the saddle	12
3.3. Battery status and charging	13
3.4. Riding your tricycle	17
3.5. Keys	18
4. Maintenance	18
4.1. Points of maintenance	18
4.2. Maintenance instructions	19
4.3. Troubleshooting	20
4.4. Expected lifespan	21
4.5. Reuse	21
4.6. End of use	21
4.7. Warranty	21
5. Technical specifications	22

Preface

Congratulations! You are now owner of a Vermeiren tricycle!

This tricycle is made by qualified and committed personnel. It is designed and produced according to high quality standards, guarded by Vermeiren.

Thank you for your trust in the products of Vermeiren. To support you on the use of this tricycle and its operating functions, this manual is offered. Please read this manual carefully; it will help you to get familiar with the operation, capabilities and limitations of your tricycle.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for tricycles For specialist dealer
	EC declaration of conformity

1. Your product

2217E

2219E

1. Handgrips
2. Bell
3. Handlebar
4. Brake lever
5. Front reflector
6. Mudguard
7. Front wheel
8. Frame
9. Pedals
10. Chain cover
11. Rear wheels
12. Rear reflector
13. Basket
14. Saddle
15. Battery
16. Identification plate

1.1. Options

Contact your specialist dealer about options. He will gladly advise you.

2. Before use

2.1. Intended use

In this paragraph a brief description of the intended use of your product is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- This product is a medical device.
- Indications and contraindications: The user can ride the tricycle by himself. The tricycle is intended for people with stiff or damaged joints, heart insufficiencies or poor blood circulation, balance disturbances, cachexia, ... The pedal assistance is intended for users with reduced muscular power in their legs. You should NOT use this tricycle if you suffer from physical or mental impairments that may put you, or other people, in danger while driving. For this reason, consult your doctor first and make sure that your specialist dealer is informed about his advice.
- The tricycle is suitable for outdoor use.
- The intended use environments are paved roads and gravel roads that are smooth. The device is not intended to be used on off-road or mountain bike use, or for any kind of jumping or racing.
- The tricycle is designed and produced solely to transport 1 person with a maximum weight as specified in §5.. It is not designed for transportation of heavy goods or objects, nor for any use other than previously described.
- Only use accessories and spare parts approved by Vermeiren.
- Please read all technical details and limits of your tricycle in chapter 5..
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.2. General safety instructions

CAUTION

Risk of injuries and/or damage

- Carefully read and follow the instructions in this manual. Otherwise you may get injured or your tricycle may get damaged.

Keep the following general warnings in mind during use:

- Do not use your tricycle if you are under influence of alcohol, medicines or other substances that may influence your riding abilities.
- Be aware that some parts of your tricycle may get very hot or cold due to ambient temperature, solar radiation or heating devices. Be careful when touching. Wear protective clothing if the weather is cold.
- Do not modify your tricycle in any way.
- To prevent injury and/or damage to your tricycle, make sure that no objects and/or body parts are caught in the spokes of the wheels.
- Investigate the effects of shifting the centre of gravity on the behaviour of the tricycle, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles.
- The tricycle must be used according the applicable traffic regulations.
- Adapt your riding to weather and traffic conditions.
- To be better visible when riding in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the front and rear of the tricycle are clearly visible.
- Never exceed the maximum load of your tricycle.
- Be careful when riding in bends, the tricycle can lose sideways stability.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

2.3. Symbols on the tricycle

	Maximum weight of the user in kg
	Type designation
	Catalogue number
	Serial number
	Medical device
	Manufacturer
	Date of manufacture
	Declaration of conformity
	Attention: important information
	It is advised to read the manual
	Risk of entrapment

2.4. Storage

Make sure that your tricycle is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 5..

When your product is not used on a daily basis, make sure to switch off the battery as well as the drive electronics, see **§3.3.2.** and **§3.4.2.**

3. Using your tricycle

⚠ WARNING

Risk of injuries


- First read previous chapters and inform yourself about the intended use. Do NOT use your tricycle unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

3.1. Operating the brakes

⚠ WARNING

Risk of injuries

- Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...). Check the condition of the tires before each use.
- The brakes are adjustable and can wear. Check the operation of the brakes before each use.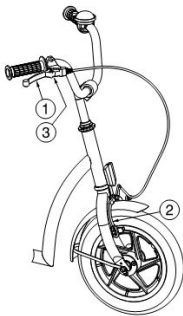

To apply the manual brake:


1. Pull the brake lever **(1)** to the handgrip.
2. The brake **(2)** shall be pressed against the front wheel.

To release the manual brake:

1. Release the brake lever **(1)**.

To activate the parking brakes: Block the brakes by pushing the plate **(3)**.

The front wheel brake and rear wheel brake are activated by following the same instructions. Left or right side lever can be chosen at the moment of purchase.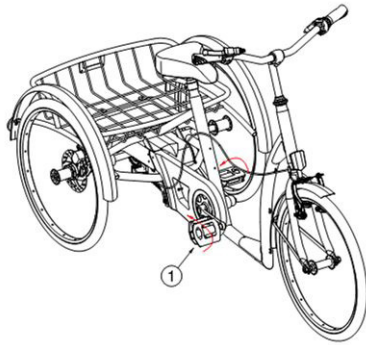

To apply the torpedo brake on the rear wheels:


1. Turn the pedals **(1)** backwards.

⚠ CAUTION

Risk of injuries

- Shift gears by turning the lever one gear at a time, and reduce the force applied to the pedals while shifting. If you try to force operation of the shifting lever while the pedals are being turned with force, your feet may slip off the pedals and the tricycle may tip over.

Turn the gear lever to shift to each of the three gears.

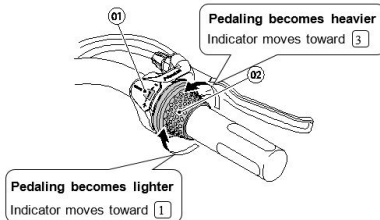1. Gear indicator
2. Gear shift lever

3.2. Adjusting the saddle

⚠ CAUTION

Risk of injuries

- Never adjust the saddle when the user is sitting on the saddle.
- Check that the saddle is firmly secured before using the tricycle.

To adjust the height of the saddle (stepless):

1. Loosen the screw **(1)**.
2. Adjust the height of the saddle **(2)** to a comfortable position.
3. Tighten the screw **(1)** properly.

The maximum height is indicated by a line.

3.3. Battery status and charging

⚠ WARNING

Risk of injuries and damage due to fire

- Only use the battery charger delivered with the tricycle. The use of any other charger may be dangerous (fire hazard).
- The battery charger is only intended to charge the batteries delivered with the tricycle, not to charge any other batteries.
- All components of the pedal assist system are matched to each other. Never modify the system and do not adapt any of the delivered parts, such as cables, plugs or battery charger. Never open or change the battery or the connection points.
- Protect the battery and battery charger against flames, high and low temperatures (see chapter 5.), moisture, sunshine, severe shocks (for example dropping). Do NOT use the battery if this has been the case.
- Charge the battery with the battery charger, indoors, in a well-ventilated area, out of children's reach.
- Remove the battery from the tricycle before transporting the tricycle in a car.
- Keep the ventilation openings of the battery charger clean and free of accumulated dust. Blow away the dust and clean the battery charger enclosure with a slightly moistened cloth, if required.

CAUTION

Risk of damage

- Battery self-discharge and the quiescent current of the connected users will drain the battery charge slowly. The battery may be irreversibly damaged if the battery is fully discharged. Therefore, make sure that the battery is charged on time.
 - When your product is not used on a daily basis, make sure to switch off the battery as well as the drive electronics, see **§3.3.2.** and **§3.4.2.**
 - Read the storage and maintenance instructions in **§4.** and technical details in **§5.**
 - Do not charge batteries below 0°C. Move the battery to a warmer place and start charging.
 - Keep the connection point for the battery charger free of dust and other contamination.
 - In case of any problems due to which the battery cannot be charged according to the following instructions, please contact your specialist dealer.
- i** It is recommended to unplug the battery charger from the mains when not in use. This is to prevent unnecessary energy consumption.

Always check the battery for damage before charging. Press the charge indicator button for 2 seconds to switch on the battery. When no LED of the charge indicator lights up, the battery may be damaged. When at least one, but not all LEDs of the charge indicator, is lit, then fully charge the battery pack before using for the first time.

Do not charge or use a damaged battery, contact your specialist dealer.


To ensure full battery capacity, completely charge the battery with the charger before using it for the first time. The battery can be recharged at any time on the tricycle or while removed from the tricycle.

Instructions to charge the battery:

1. Switch the battery off.
2. If desired, remove the battery from the battery housing and prevent dirt or fluids from entering the charging socket.
3. Connect the charger plug to the battery pack **(1)**.
4. Connect the charger mains plug into the wall socket.
5. The LED indicator on the battery charger will show the status of the charging process:

- Red - Charging
- Green - Standby / Fully Charged

6. Once the battery is fully charged, remove the battery charger from the wall socket and the battery.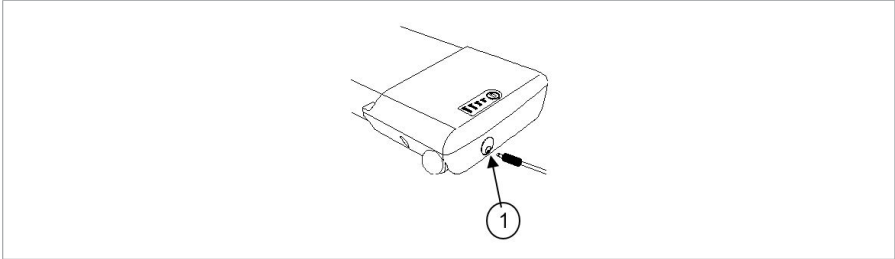

3.3.1. Battery charge indicator


There are two battery charge indicators:

	<p>3.3.1.1. On the battery pack (located underneath the basket, rear side of the tricycle):</p> <ol style="list-style-type: none"> 1. Press button (2) on the battery pack. 2. The LED's of the battery charge indicator (1) will indicate the battery charge level.
	<p>3.3.1.2. Operator control (located on the handlebar, front side of the tricycle):</p> <p>When the electric system is switched on, the battery indicator (5, §3.4.1.) will indicate the battery charge level: 1 LED = low battery, 4 LEDs = full battery.</p>

Because of the wiring, both indications can deviate a little bit. In that case, take the worst charge level indication.

3.3.2. Switching the electric device On/Off

Press the on/off switch on the side of the battery.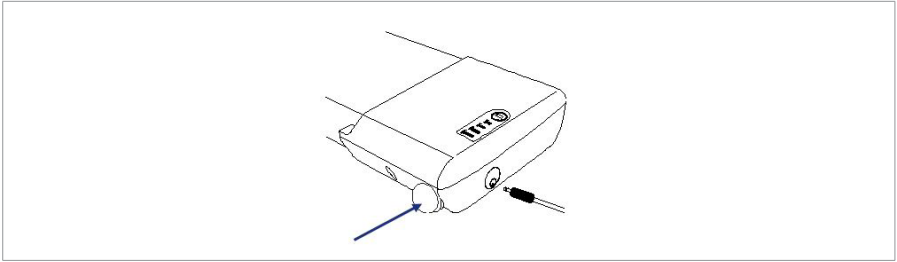


3.3.3. Mounting / removing the battery

To mount the battery:

1. Place the key **(1)** and turn the key towards the unlock symbol.
2. Place the battery in the battery housing as far as possible. There is only 1 possible way to insert the battery.
3. Turn the key **(1)** to the lock symbol.
4. Make sure that the battery is fixed well.
5. Remove the key **(1)**. (Never remove the key while riding).

To remove the battery:

1. Place the key **(1)**.
2. Turn the key **(1)** to the unlock symbol.
3. Pull the battery gently out of the battery housing.

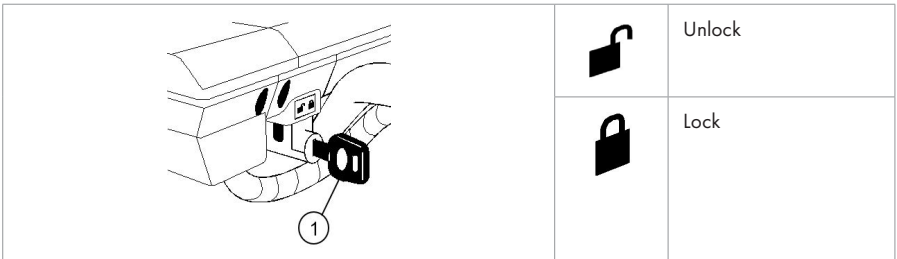3.4. Riding your tricycle

⚠ WARNING

Risk of injuries

- Before using the tricycle in busy or potentially dangerous areas or roads with heavy traffic, familiarize yourself with the operation of your tricycle. Please practice in a wide and open area like a park. Try out the different assistance levels.

When the assist system is turned ON, the electric system engages and will provide assistance only while pedaling. The amount of power depends on your pedaling force and the assistance level. When you stop pedaling or reach a speed more than 17 km/h the assistance is switched off automatically. When the assistance is switched off (assistance level: 0), you can ride normally with your tricycle.

3.4.1. Operator control interface

3.4.2. ON/OFF

Press the on/off button **(1)** to switch the system on/off. When the tricycle stops and the system is not used for 5 minutes, the assist system switches off by itself.

3.4.3. Choosing the power assist level

Press the MODE button **(2)** to select the desired power assist level (1 = lowest, 3 = highest).

When switching on the control system, the meter will automatically restore the power assist level to “low”.

3.4.4. Power-assisted start function

When the pedals are in an uncomfortable position to start cycling, an assisting start function is available in which the tricycle drives at a speed of max. 6 km/h.

Press the power-assisted start button **(4)** to activate the assisting start function. The tricycle will drive until you press the button or the brake again.

3.5. Keys

Your E-tricycle is delivered with 2 keys: a main key and spare key. The keys are identified by a serial number. Keep the spare key in a safe place and write the key serial number down for key replacement.

4. Maintenance

i Regular care ensures that your tricycle is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: www.vermeiren.com.

4.1. Points of maintenance

⚠ CAUTION

Risk of injuries and damage

- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

i The last page of this manual contains a registration form for the specialist dealer to record each service.

The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

4.1.1. Before each use

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see § 4.2.2..
- Condition of wheels/tyres, see § 4.2.1..
- Condition of frame parts: No deformation, instability, weakness or loose connections.
- Brakes: Undamaged and functioning.

Contact your specialist dealer for possible repairs or part replacements.

4.1.2. Yearly or more often

Have your tricycle inspected and serviced by your specialist dealer, at least once a year, or more often. The minimum maintenance frequency depends on use and should therefore be commonly agreed upon with your specialist dealer.

4.1.3. If stored

Make sure that your tricycle is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 5..

When your product is not used on a daily basis, make sure to switch off the battery as well as the drive electronics, see §3.3.2. and §3.4.2..

4.2. Maintenance instructions

4.2.1. Wheels and tyres

- ❗ Proper working of the brakes depends on the state of the tyres, and can change due to wear and contamination (water, oil, mud, ...).

Keep your wheels free of wires, hair, sand and fibres.

Check the profile of the tyres. If the tread depth is less than 1 mm, the tyres need to be replaced. Contact your specialist dealer for this matter.

Inflate each pneumatic tyre to the correct pressure (see pressure indication on the tyres).

4.2.2. Cleaning

CAUTION

Risk of damage by moisture

- Never use a hose or high-pressure cleaner to clean the tricycle.

Wipe all rigid parts with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents to clean.

4.2.3. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

4.3. Troubleshooting

Even if you use your tricycle properly, a technical problem may occur. In this case, contact your local specialist dealer.

WARNING

Risk of injuries and damage

- NEVER attempt to repair your tricycle yourself.

The following symptoms may indicate a serious problem. Therefore always contact your specialist dealer if you spot any of the following deviations:

- Strange sound;
- Uneven tread wear on one of the tyres;
- Jerky movements;
- Damaged or broken wheel assemblies.

4.4. Expected lifespan

Your tricycle is constructed of aluminium components. The tricycle is designed to have an average lifespan of 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan will increase or decrease.

4.5. Reuse

Before each reuse, have the tricycle disinfected, inspected and serviced according to the instructions in § 4.1. and § 4.2..

4.6. End of use

At end of life, you need to dispose your tricycle according to the local environmental legislation. The best way to do so, is to disassemble the tricycle to facilitate the transport of recyclable parts.

4.7. Warranty

The warranty on this product is subject to the general terms and conditions of each country.

5. Technical specifications

The technical details below are only valid for this tricycle, at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your tricycle has been modified, damaged, or is severely worn.

Brand	Vermeiren	
Product group	Tricycles	
Type	2217E 2219E	
Description	Dimensions	
	2217E	2219E
Maximum user weight	110 kg	125 kg
Maximum basket load	25 kg	25 kg
Overall length	1600 mm	1808 mm
Overall width	670 mm	768 mm
Overall height	986 mm - 1174 mm	1121 mm - 1309 mm
Total mass	31,5 kg	33,8 kg
Rated slope	7°	7°
Ground clearance	97 mm	125 mm
Frame tube height (entry height)	174 mm	194 mm
Diameter of front wheels	20" x 1,75	24" x 1 3/8"
Tyre pressure front wheels	2,5 - 5 bar	3,5 bar
Diameter of rear wheels	20" x 1,75	24" x 1 3/8"
Tyre pressure rear wheels	2,5 - 5 bar	3,5 bar
Saddle height	709 mm - 950 mm	789 mm - 1030 mm
Distance between saddle and pedal	586 mm - 827 mm	640 mm - 880 mm
Handgrip height (measured from the ground)	Depends on handlebar model and stem	Depends on handlebar model and stem
Minimum height pedal	123 mm	150 mm
Maximum height pedal	373 mm	455 mm
Distance between saddle and handlebar	372 mm - 401 mm Adjustable: -20 mm	417 mm - 446 mm Adjustable: -20 mm
Minimum turning radius	2750 mm	3000 mm

Brand	Vermeiren	
Product group	Tricycles	
Type	2217E 2219E	
Description	Dimensions	
	2217E	2219E
Rear gear	3	3 or 7
Front gear	Not applicable	Not applicable
Gear ratio	184%	184%
Brakes	V-brake - Front Disc brake - Rear (torpedo brake available - disc brake as option)	Disc brake - Rear (torpedo brake available - disc brake as option)
Battery	10 Ah	
System	36V	
Motor	250W	
Assistance up to	17 km/h	
Assistance level	1-3	
Energy consumption *	>25 km *	
Storage and use temperature	-20°C - +35°C	
Storage and use humidity	30% - 70%	
<p>We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°</p> <p>* The theoretical driving distance is influenced by battery charge level, selected assistance level, temperature and wind condition, user weight and load in the basket, tire pressure and condition of the tires, selected gear, accelerations, drive chain condition, condition of the road, slopes, ...</p>		

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

VERMEIREN GROUP

Vermeirenplein 1/15

2920 Kalmthout

Belgium

www.vermeiren.com

Version: H, 2024-11

Basic UDI-DI: 5415174 1218062217E 52

5415174 1218062219E 58

All rights reserved, including translation.

VERMEIREN
we care for you