Toilet Stacy

Install the backrest tubing behind the commode using the snap buttons. Ensure that the snap buttons are secured in the holes.

Slide the toilet bowl into the toilet bowl holder. Slide the toilet bowl inside through the frontside. Ensure that the toilet bowl is correctly positioned before sitting down.

Adjust the seat height using the snap buttons on the legs. Ensure that the snap buttons are secured in the holes. Ensure that all four legs are set to the same height before use.

The armrests can be folded away. To lower the armrests press the side buttons. Be careful for your fingers.

Pull the armrest upwards again. An audible "click" sound will be heard when the armrest is securely locked into position.


Ask assistance if you are uncertain on the usage of your product.

! Users weight: 130 kg

! The toiletseat Stacy can be cleaned with normal household cleaning products. Avoid using aggressive cleaning products which can cause discoloration.

Dimensions:
 - width: 57 cm
 - length: 53 cm
 - total height (with lid upwards): 80-95 cm
 - seat height: 44-59 cm
 - leg height adjustment: 15 cm
 - seat width: 45 cm
 - seat depth: 45 cm
 - armrest height: 23 cm
 - height backrest: 24 cm
 - total weight: 8,6 kg

Button to fold away the armrest

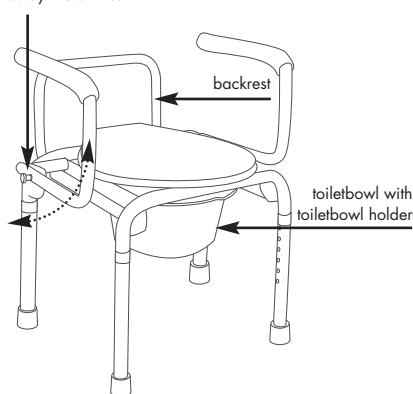Subject to technical alterations. © 2016

VERMEIREN
Opening up new horizons in mobility

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Vermeiren Nederland B.V.