
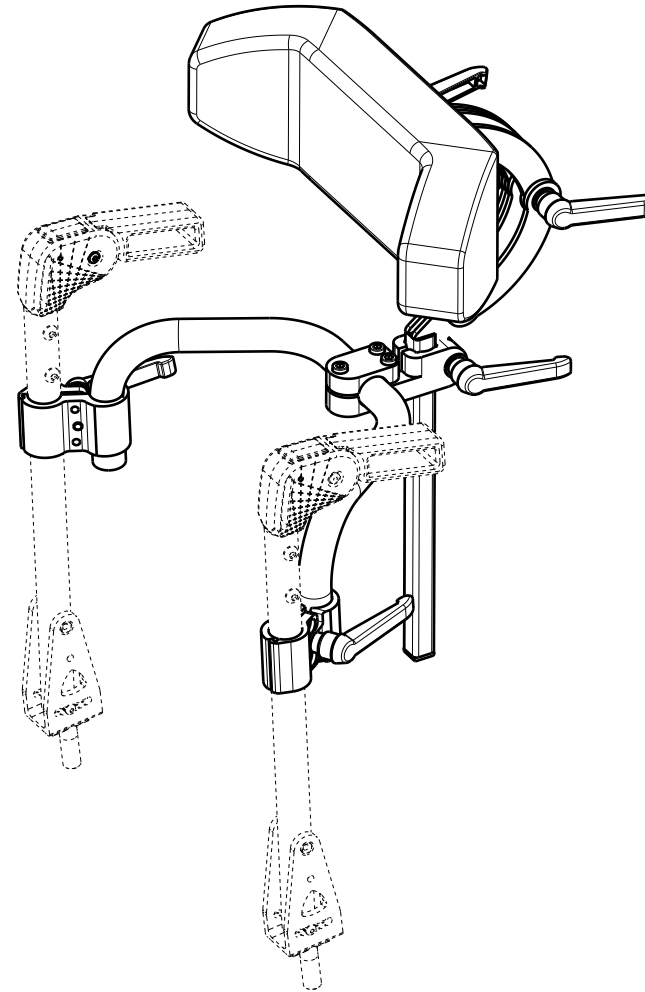


EP00587 / A

HEADREST ADJUSTABLE TRIGO				Plan No.
				EP00587 / A
Sheet 1/2				
Scale: 1:3	Drafted	Verified	Confirmed	
Date	10/06/2021	10/06/2021	10/06/2021	
Initials	AS	MK	TK	

Material			
No.	Plan	Article	Quantity
1	YP09051 (30-34)	2633365	1
	YP09054 (36-40)	2633465	1
	YP09058 (42-46)	2633565	1
	YP09060 (48-50)	2633665	1
2	Y011987	1669854	1
3	Y011148	1669754	6
4	XP02044	GP14899	2
5	XP02045	GP14999	2
6	XP01417	2706799	1
7	XP01418	2706899	1
8	M6x20	1905873	2
9	M8x30	1005860	1
10	M8x12	1900712	3
11	M6x25	1001494	3
12	M6x16	2994065	6
13	M8x35	1001526	3
14	X001119	1001554	4
15	CAPD22	1990648	2
16	PLUG15X15	1005899	1
17	Cordura	2996862	1
	Nylon	2996863	1



EP00587 / A

HEADREST ADJUSTABLE TRIGO				Plan No.
				Sheet 2/2 EP00587 / A
Scale: 1:3	Drafted	Verified	Confirmed	
Date	10/06/2021	10/06/2021	10/06/2021	
Initials	AS	MK	TK	