


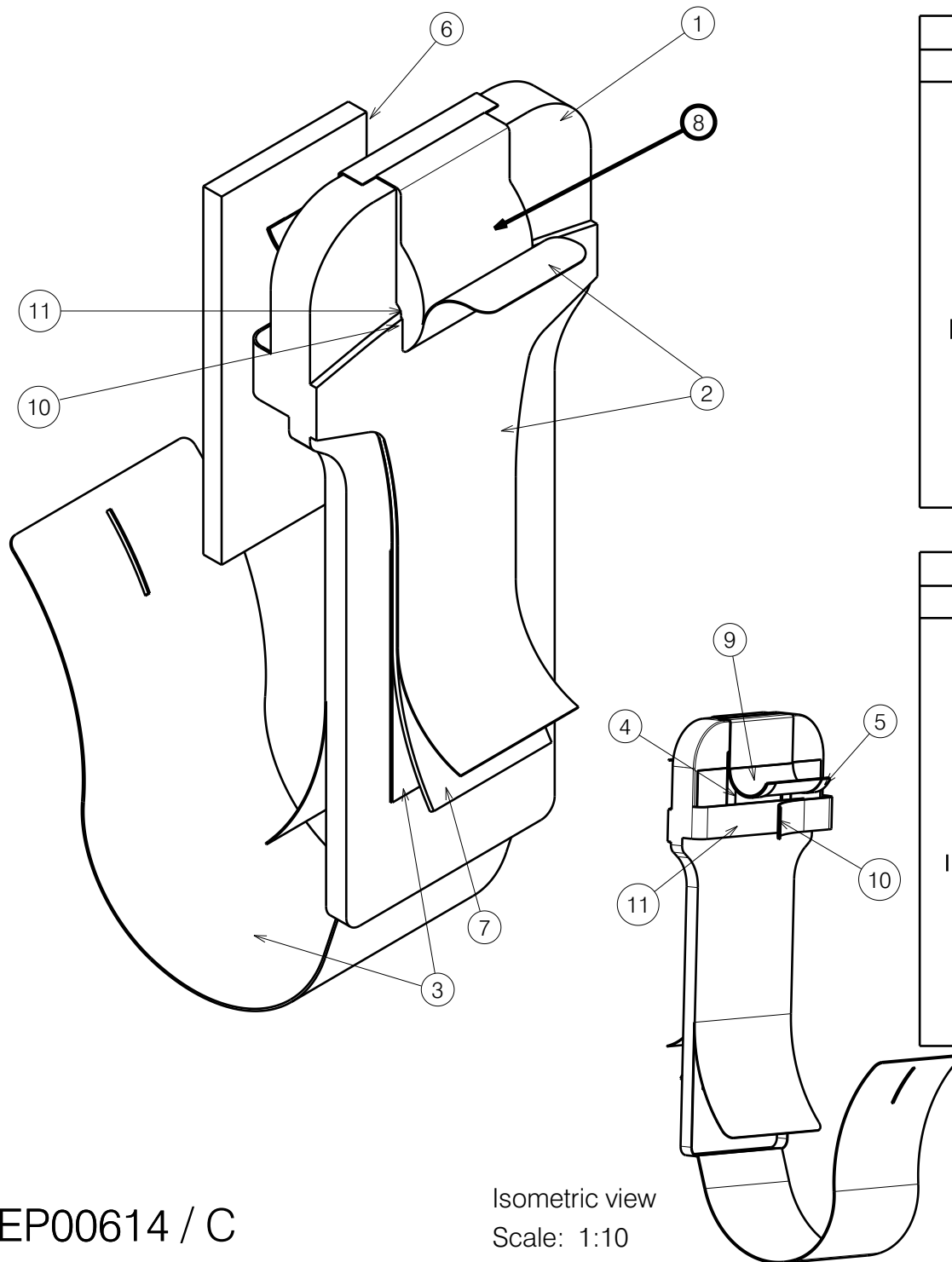

Isometric view  
Scale: 1:10

| BACKREST GREY/GREEN GEMINI II 32 2020: 2996830 |                               |         |          |
|------------------------------------------------|-------------------------------|---------|----------|
| No.                                            | Plan                          | Article | Quantity |
| 1                                              | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,75m    |
| 2                                              | FABRIC ALCALA (63) GREEN      | 2996798 | 0,2m     |
| 3                                              | NYLON BLACK                   | 1008943 | 0,3m     |
| 4                                              | VELCRO BLACK 25 MM            | 1009709 | 0,3m     |
| 5                                              | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,3m     |
| 6                                              | XP03151                       | 2996723 | 1        |
| 7                                              | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,01rl   |
| 8                                              | VULLING CARDAFILL             | 2995111 | 0,01kg   |
| 9                                              | VELCRO BLACK 100MM            | 2991789 | 0,3m     |
| 10                                             | VELCRO HOOK BLACK 50 MM       | 1009791 | 0,8m     |
| 11                                             | VELCRO BLACK 50 MM            | 1009792 | 0,6m     |

| BACKREST GREY/BLUE GEMINI II 32 2020 : 2996831 |                               |         |          |
|------------------------------------------------|-------------------------------|---------|----------|
| No.                                            | Plan                          | Article | Quantity |
| 1                                              | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,75m    |
| 2                                              | FABRIC ALCALA (65) BLUE       | 2996797 | 0,2m     |
| 3                                              | NYLON BLACK                   | 1008943 | 0,3m     |
| 4                                              | VELCRO BLACK 25 MM            | 1009709 | 0,3m     |
| 5                                              | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,3m     |
| 6                                              | XP03151                       | 2996723 | 1        |
| 7                                              | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,01rl   |
| 8                                              | VULLING CARDAFILL             | 2995111 | 0,01kg   |
| 9                                              | VELCRO BLACK 100MM            | 2991789 | 0,3m     |
| 10                                             | VELCRO HOOK BLACK 50 MM       | 1009791 | 0,8m     |
| 11                                             | VELCRO BLACK 50 MM            | 1009792 | 0,6m     |

EP00614 / C

BACKREST GEMINI 2020

Plan No.

Page 1/3

EP00614/ C

Scale: 1:10

Drafted

Verified

Confirmed

Date

21/09/2021

12/02/2025

12/02/2025

Initials

OY

PM

AS



| BACKREST GREY/BLACK GEMINI II 32 2020: 2996832 |                               |         |          |
|------------------------------------------------|-------------------------------|---------|----------|
| No.                                            | Plan                          | Article | Quantity |
| 1                                              | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,75m    |
| 2                                              | FABRIC ALCALA (14) BLACK      | 2994281 | 0,2m     |
| 3                                              | NYLON BLACK                   | 1008943 | 0,3m     |
| 4                                              | VELCRO BLACK 25 MM            | 1009709 | 0,3m     |
| 5                                              | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,3m     |
| 6                                              | XP03151                       | 2996723 | 1        |
| 7                                              | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,01rl   |
| 8                                              | VULLING CARDAFILL             | 2995111 | 0,01kg   |
| 9                                              | VELCRO BLACK 100MM            | 2991789 | 0,3m     |
| 10                                             | VELCRO HOOK BLACK 50 MM       | 1009791 | 0,8m     |
| 11                                             | VELCRO BLACK 50 MM            | 1009792 | 0,6m     |

| BACKREST BLACK/BLUE AIRFL GEMINI II 32 2020 : 2996834 |                               |         |          |
|-------------------------------------------------------|-------------------------------|---------|----------|
| No.                                                   | Plan                          | Article | Quantity |
| 1                                                     | FABRIC AIR FLOW BLACK 900     | 2992087 | 0,75m    |
| 2                                                     | FABRIC AIR FLOW BLUE 9570     | 2992088 | 0,2m     |
| 3                                                     | NYLON BLACK                   | 1008943 | 0,3m     |
| 4                                                     | VELCRO BLACK 25 MM            | 1009709 | 0,3m     |
| 5                                                     | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,3m     |
| 6                                                     | XP03151                       | 2996723 | 1        |
| 7                                                     | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,01rl   |
| 8                                                     | VULLING CARDAFILL             | 2995111 | 0,01kg   |
| 9                                                     | VELCRO BLACK 100MM            | 2991789 | 0,3m     |
| 10                                                    | VELCRO HOOK BLACK 50 MM       | 1009791 | 0,8m     |
| 11                                                    | VELCRO BLACK 50 MM            | 1009792 | 0,6m     |

| BACKREST GREY/PURPLE GEMINI II 32 2020 : 2996833 |                               |         |          |
|--------------------------------------------------|-------------------------------|---------|----------|
| No.                                              | Plan                          | Article | Quantity |
| 1                                                | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,75m    |
| 2                                                | FABRIC ALCALA (62) PURPLE     | 2996800 | 0,2m     |
| 3                                                | NYLON BLACK                   | 1008943 | 0,3m     |
| 4                                                | VELCRO BLACK 25 MM            | 1009709 | 0,3m     |
| 5                                                | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,3m     |
| 6                                                | XP03151                       | 2996723 | 1        |
| 7                                                | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,01rl   |
| 8                                                | VULLING CARDAFILL             | 2995111 | 0,01kg   |
| 9                                                | VELCRO BLACK 100MM            | 2991789 | 0,3m     |
| 10                                               | VELCRO HOOK BLACK 50 MM       | 1009791 | 0,8m     |
| 11                                               | VELCRO BLACK 50 MM            | 1009792 | 0,6m     |

| BACKREST GREY/BLUE GEMINI II 40 2020 : 2996973 |                               |         |          |
|------------------------------------------------|-------------------------------|---------|----------|
| No.                                            | Plan                          | Article | Quantity |
| 1                                              | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,85m    |
| 2                                              | FABRIC ALCALA (65) BLUE       | 2996797 | 0,25m    |
| 3                                              | NYLON BLACK                   | 1008943 | 0,4m     |
| 4                                              | VELCRO BLACK 25 MM            | 1009709 | 0,4m     |
| 5                                              | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,4m     |
| 6                                              | XP03179                       | 2996796 | 1        |
| 7                                              | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,02rl   |
| 8                                              | VULLING CARDAFILL             | 2995111 | 0,02kg   |
| 9                                              | VELCRO BLACK 100MM            | 2991789 | 0,4m     |
| 10                                             | VELCRO HOOK BLACK 50 MM       | 1009791 | 1m       |
| 11                                             | VELCRO BLACK 50 MM            | 1009792 | 0,8m     |

EP00614 / C

| BACKREST GEMINI 2020 |            |            |            | Plan No.                                                                              |
|----------------------|------------|------------|------------|---------------------------------------------------------------------------------------|
| Page 2/3             |            |            |            | EP00614/ C                                                                            |
| Scale: 1:10          | Drafted    | Verified   | Confirmed  |  |
| Date                 | 21/09/2021 | 12/02/2025 | 12/02/2025 |                                                                                       |
| Initials             | OY         | PM         | AS         |                                                                                       |

| BACKREST GREY/BLACK GEMINI II 40 2020: 2996974 |                               |         |          |
|------------------------------------------------|-------------------------------|---------|----------|
| No.                                            | Plan                          | Article | Quantity |
| 1                                              | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,85m    |
| 2                                              | FABRIC ALCALA (14) BLACK      | 2994281 | 0,25m    |
| 3                                              | NYLON BLACK                   | 1008943 | 0,4m     |
| 4                                              | VELCRO BLACK 25 MM            | 1009709 | 0,4m     |
| 5                                              | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,4m     |
| 6                                              | XP03179                       | 2996796 | 1        |
| 7                                              | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,02rl   |
| 8                                              | VULLING CARDAFILL             | 2995111 | 0,02kg   |
| 9                                              | VELCRO BLACK 100MM            | 2991789 | 0,4m     |
| 10                                             | VELCRO HOOK BLACK 50 MM       | 1009791 | 1m       |
| 11                                             | VELCRO BLACK 50 MM            | 1009792 | 0,8m     |

| BACKREST BLACK/BLUE AIRFL GEMINI II 40 2020 : 2996976 |                               |         |          |
|-------------------------------------------------------|-------------------------------|---------|----------|
| No.                                                   | Plan                          | Article | Quantity |
| 1                                                     | FABRIC AIR FLOW BLACK 900     | 2992087 | 0,85m    |
| 2                                                     | FABRIC AIR FLOW BLUE 9570     | 2992088 | 0,25m    |
| 3                                                     | NYLON BLACK                   | 1008943 | 0,4m     |
| 4                                                     | VELCRO BLACK 25 MM            | 1009709 | 0,4m     |
| 5                                                     | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,4m     |
| 6                                                     | XP03179                       | 2996796 | 1        |
| 7                                                     | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,02rl   |
| 8                                                     | VULLING CARDAFILL             | 2995111 | 0,02kg   |
| 9                                                     | VELCRO BLACK 100MM            | 2991789 | 0,4m     |
| 10                                                    | VELCRO HOOK BLACK 50 MM       | 1009791 | 1m       |
| 11                                                    | VELCRO BLACK 50 MM            | 1009792 | 0,8m     |

| BACKREST GREY/PURPLE GEMINI II 40 2020 : 2996975 |                               |         |          |
|--------------------------------------------------|-------------------------------|---------|----------|
| No.                                              | Plan                          | Article | Quantity |
| 1                                                | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,85m    |
| 2                                                | FABRIC ALCALA (62) PURPLE     | 2996800 | 0,25m    |
| 3                                                | NYLON BLACK                   | 1008943 | 0,4m     |
| 4                                                | VELCRO BLACK 25 MM            | 1009709 | 0,4m     |
| 5                                                | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,4m     |
| 6                                                | XP03179                       | 2996796 | 1        |
| 7                                                | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,02rl   |
| 8                                                | VULLING CARDAFILL             | 2995111 | 0,02kg   |
| 9                                                | VELCRO BLACK 100MM            | 2991789 | 0,4m     |
| 10                                               | VELCRO HOOK BLACK 50 MM       | 1009791 | 1m       |
| 11                                               | VELCRO BLACK 50 MM            | 1009792 | 0,8m     |

| BACKREST GREY/GREEN GEMINI II 40 2020: 2996972 |                               |         |          |
|------------------------------------------------|-------------------------------|---------|----------|
| No.                                            | Plan                          | Article | Quantity |
| 1                                              | FABRIC ALCALA (22) LIGHT GREY | 2996799 | 0,85m    |
| 2                                              | FABRIC ALCALA (63) GREEN      | 2996798 | 0,25m    |
| 3                                              | NYLON BLACK                   | 1008943 | 0,4m     |
| 4                                              | VELCRO BLACK 25 MM            | 1009709 | 0,4m     |
| 5                                              | VELCRO HOOK BLACK 25 MM       | 1009710 | 0,4m     |
| 6                                              | XP03179                       | 2996796 | 1        |
| 7                                              | MOUSSE ON ROLL 5MM 180MX2,10M | 1990296 | 0,02rl   |
| 8                                              | VULLING CARDAFILL             | 2995111 | 0,02kg   |
| 9                                              | VELCRO BLACK 100MM            | 2991789 | 0,4m     |
| 10                                             | VELCRO HOOK BLACK 50 MM       | 1009791 | 1m       |
| 11                                             | VELCRO BLACK 50 MM            | 1009792 | 0,8m     |

EP00614 / C

| BACKREST GEMINI 2020 |            |            |            | Plan No.                                                                              |
|----------------------|------------|------------|------------|---------------------------------------------------------------------------------------|
| Page 3/3             |            |            |            | EP00614/ C                                                                            |
| Scale: 1:10          | Drafted    | Verified   | Confirmed  |  |
| Date                 | 21/09/2021 | 12/02/2025 | 12/02/2025 |                                                                                       |
| Initials             | OY         | PM         | AS         |                                                                                       |