

Material			
No.	Plan	Article	Quantity
1	YP05755(R)	2723554	1
	YP05757(L)	2723654	1
2a	YP07443(R)	27635..	1
	YP07444(L)	27636..	1
2b	YP07999(R)low	26176..	1
	YP08000(L)low	26177..	1
2c	YP07446(R)	27637..	1
	YP07447(L)	27638..	1
2d	YP08487(R)low	26178..	1
	YP08488(L)low	26179..	1
2e	XP01938(R)carbon	2993612	1
	XP01941(L)carbon	2993615	1
2f	XP01939(R)carbon	2993613	1
	XP01973(L)carbon	2993631	1

Degrease the screws before using the glue.  
Odtluscic sruby przed uzyciem kleju.


Isometric view  
Scale: 1:3

Use WEICONLOCK  
AN 302-43


EP00248 / E

REMOVABLE SIDE GUARD TRIGO 36-50				Plan No. EP00248/ E
Scale: 1:3	Drafted	Verified	Confirmed	
Date	20/10/2017	13/08/2020	13/08/2020	
Initials	AS	MK	TK	


2a

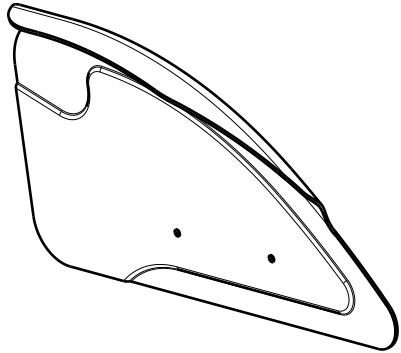


2b- low

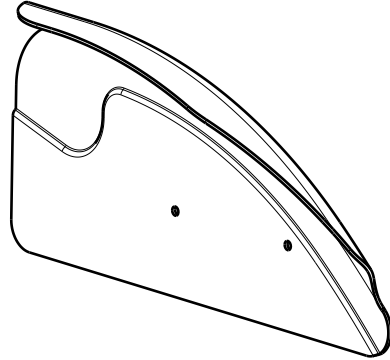


2c

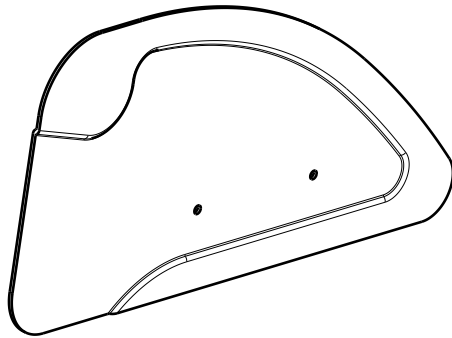


2d -low

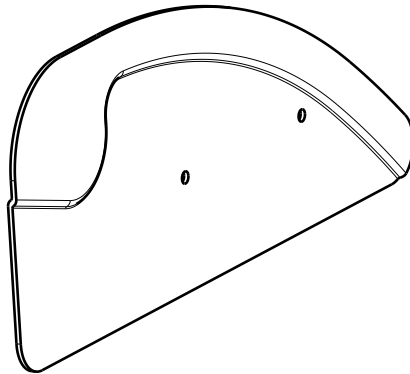

2e-carbon

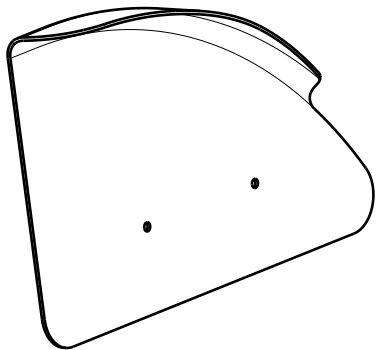

2f-carbon

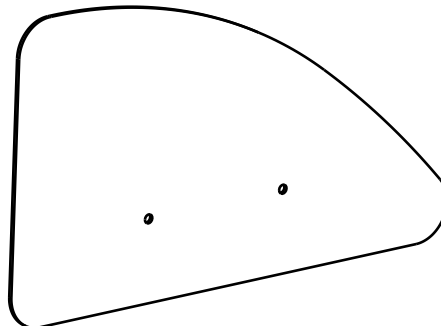

EP00248 / E

Material			
No	Plan	Article	Quantity
3	XP02068	GP15099	1
4	XP03056	GP23299	1
5	XP01730	GP11200	1
6	M5X12_DIN912	2994072	1
7	M6x25	2994897	2
8	M4x16	2994070	2
9	Nut_M5_black	2994006	2
10	Nut_M4	1001678	2
11	M5_DIN_125_A	2994069	4
12	M5x12	2994074	4
13	---	2994073	0,8m
* 14	XP02274(R)	2996467	1
	XP02275(L)	2996468	1
15	XP03080	2996656	1
16	XP03091	2996662	1

\*Dont use with carbon.

REMOVABLE SIDE GUARD TRIGO 36-50

Plan No.

EP00248/ E

Scale: 1:3

Drafted

Verified

Confirmed

Date

20/10/2017

13/08/2020

13/08/2020

Initials

AS

MK

TK

