


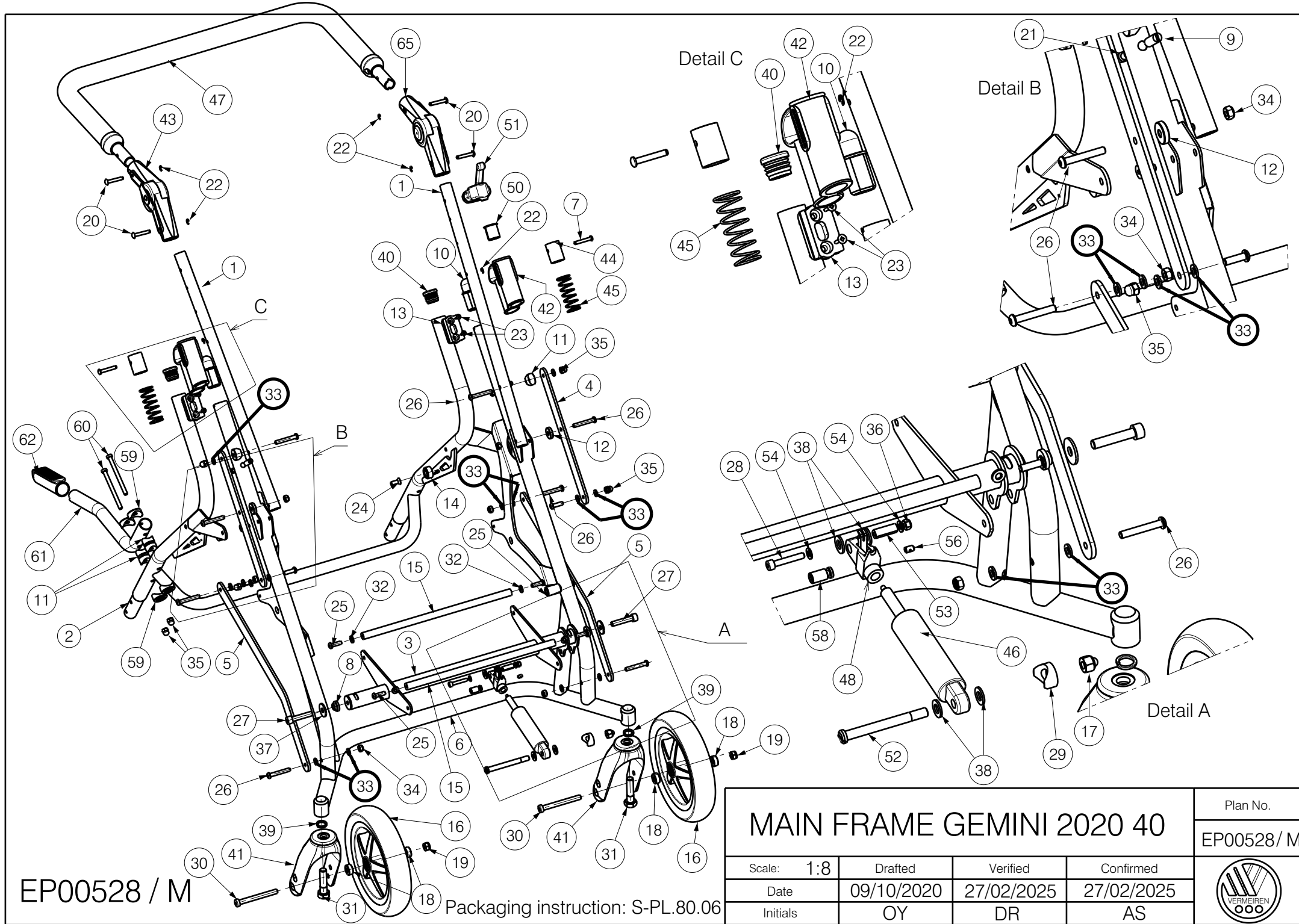

EP00528 / M

Packaging instruction: S-PL.80.06

MAIN FRAME GEMINI 2020 40				Plan No.
				EP00528/ M
Scale: 1:8	Drafted	Verified	Confirmed	
Date	09/10/2020	27/02/2025	27/02/2025	
Initials	OY	DR	AS	

Material			
No.	Plan	Article	Quantity
1	YP00899	1836354	2
2	YP09494 DP	2657154	1
3	YP09334	2644554	1
4	YP08588	2622054	2
5	YP08589	2622154	2
6	YP09343	2644854	1
7	XP00138	1817200	2
8	XP00146	1818700	2
9	XP00224	1835100	2
10	XP00225	2990145	2
11	XP00226	2990146	6
12	XP00227	2990147	2
13	XP00228	2990148	2
14	XP01100	2992081	2
15	XP03115	GP24600	2
16	XP03131	1917241	2
	XP03199	1917465	2
17	M8 DIN 986	2995398	1
18	XP03139	GP00288	4
19	M8 DIN 985	1996213	2
20	X002506	1525200	4
21	X002698	1552300	2
22	D4 DIN 6799	1993722	6
23	D4.8x22 DIN 7981	2994918	4
24	M6x16 DIN 7991	2994064	2
25	M6x20 ISO 7380	2994010	6
26	M6x40 ISO 7380	2994066	8
27	M8x40 DIN 912	2995996	2
28	M5x30 DIN912	2995561	1

Material			
No.	Plan	Article	Quantity
29	X000862	1006463	1
30	M8x75 DIN7984	2994912	2
31	M12x45 DIN 931	2994917	2
32	M6 DIN 125	2994414	4
33	M6 DIN 125A PVC	2994068	14
34	M6 DIN 985	2994063	8
35	M6 DIN 1587	2994059	6
36	M5 DIN 985	2994006	1
37	M8 DIN9021	2994911	2
38	M8 DIN125 PVC	2994415	4
39	M12 DIN 127A	2996210	2
40	Tube Cap D25	1001608	2
41	---	1991447	2
42	---	2990084	2
43	---	2990085	1
44	---	2990093	2
45	---	2990094	2
46	XC00813	3992509	1
47	XP03175	2996692	1
48	---	1994748	1
* 49	---	1992149	1
50	XP03164	2996749	1
51	XP03165	1996806	1
52	XP03423	2651200	1
53	X004906	1757400	1
54	M5 DIN 985	1000972	2
* 55	---	1908105	2
56	---	1994749	1
* 57	---	2993049	0,9m
58	---	1994750	1

Material			
No.	Plan	Article	Quantity
59	XP01439	2993524	4
60	M6X70 DIN7984	2995570	2
61	YP09490	2657054	1
62	X003596	1998528	1
63	-	-	-
64	-	-	-
65	---	2990149	1

* - not depicted

Stickers			
No.	Description	Article	Quantity
*66	Red band	T001249	2
*67	White band	T001250	2

Add stickers to the string bag,
do not put on the stroller

Packaging instruction: S-PL.80.06

EP00528 / M

Set 2990084			
No.	Plan	Article	Quantity
33	---	2990084	1
36	---	2990093	1
37	---	2990094	1

MAIN FRAME GEMINI 2020 40				Plan No.
				EP00528/ M
Scale: 1:8	Drafted	Verified	Confirmed	
Date	09/10/2020	27/02/2025	27/02/2025	
Initials	OY	DR	AS	