


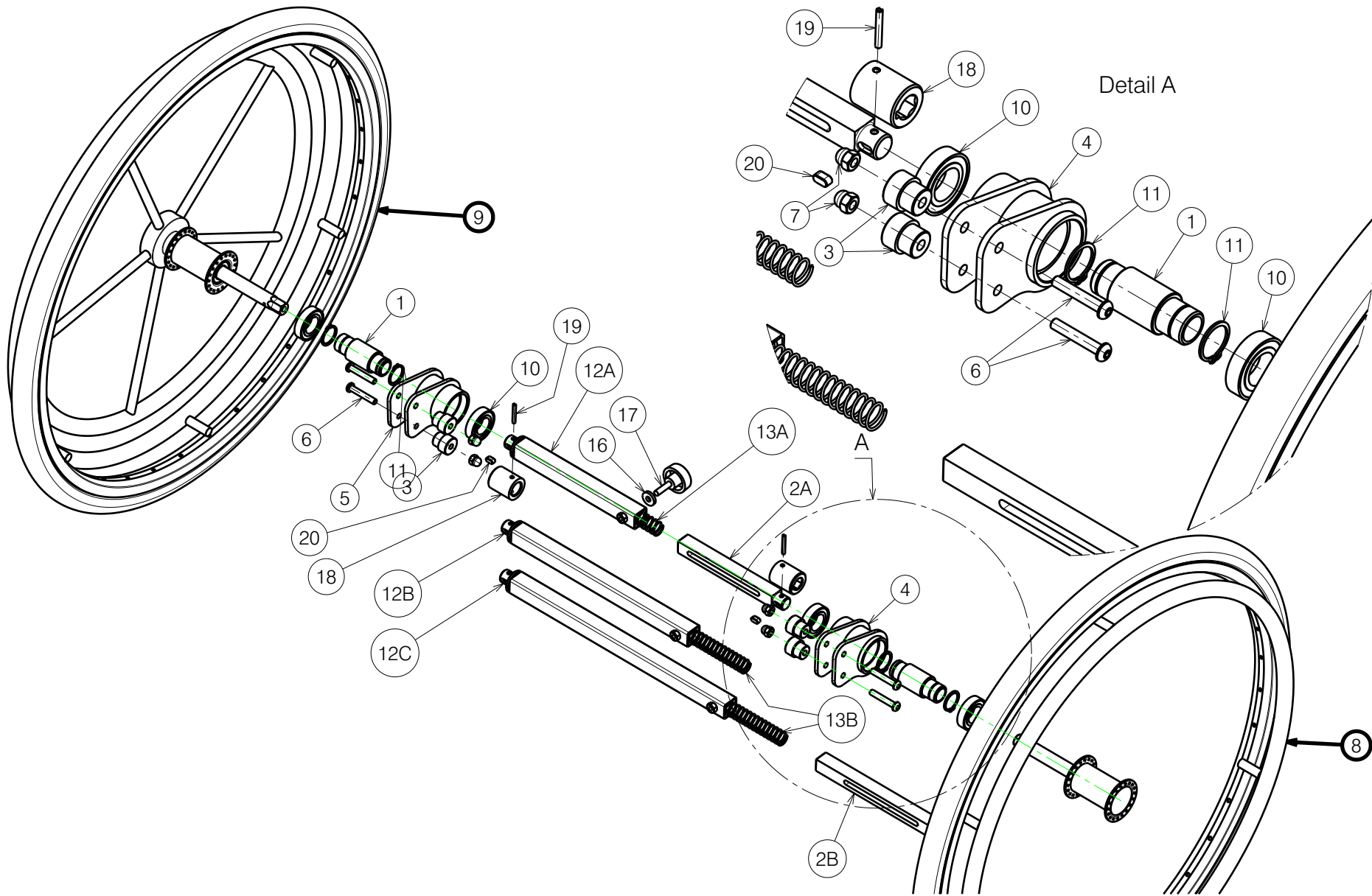

Detail A

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

V300 DRIVE SYSTEM HEM2

Scale: 1:6	Drafted	Verified	Confirmed
Date	29/09/2010	29/04/2025	29/04/2025
Initials	DD	PM	AS

Sheet 1/2

2991042

Plan No.

E000652/I



E000652 / I

2A Seat width 38 to 52
 2B Seat widths 53 to 60
 12A Seat widths 38 to 44
 12B Seat widths 46 to 52
 12C Seat widths 53 to 60
 13A Seat widths 38 to 44
 13B Seat widths 46 to 60

Material			
No.	Plan	Article	Quantity
1	X002976	1995343	2
2A	P00065423	2684100	1
2B	P00069096	2684300	1
3	X004810	1742000	4
4	Y017195	1742654 (L)	1
5	Y017206	1742754 (R)	1
6	M6x35 ISO7380	1006057	4
7	M6 DIN986	1008492	4
8	Wheel grey	1901208	1
	Wheel black*	1918198	1
9	Wheel grey	1901745	1
	Wheel black*	1918199	1
10	61904-2RS1	1994568	4
11	Ø20 DIN471	1995259	4
12A	P00064457	2683500	1
12B	P00071833	2683600	1
12C	P00071834	2683700	1
13A	X005892	G103100	1
13B	X005891	G103000	1
14	---	---	---
16	M6 DIN7349	1003321	1
17	M6x25	1903394	1
18	P00064455	GP29900	2
19	D3x26 DIN 1481	1908821	2
20	5x5x10 DIN6885A	2998153	2

*option

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

V300 DRIVE SYSTEM HEM2			
Scale: 1:6	Drafted	Verified	Confirmed
Date	29/09/2010	29/04/2025	29/04/2025
Initials	DD	PM	AS

