


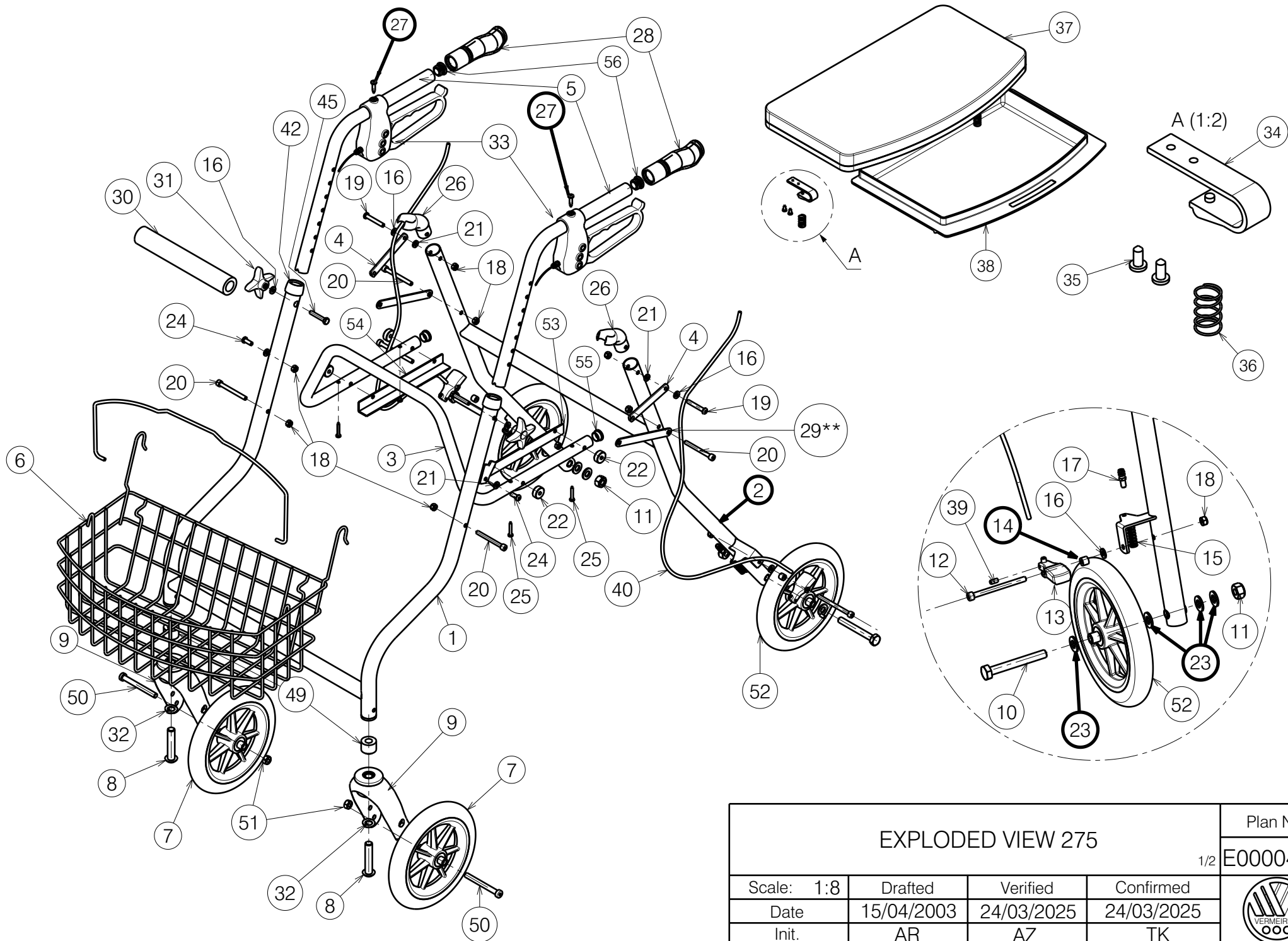

EXPLODED VIEW 275				Plan No.
				E000042/T
Scale: 1:8	Drafted	Verified	Confirmed	
Date	15/04/2003	24/03/2025	24/03/2025	
Init.	AR	AZ	TK	

Material				Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	YP09307DP	26435...	1	34	XP00071	2990040	2
2	YP00372	18117...	1	35	Screw D4x12	1990816	2
3	Y004067	15072...	1	36	X004865	1003901	4
4	Y004072	15073...	2	37	P00087222	1994587	1
5	YP09302	2643404	2	38	X002335	1993411	1
6	P00087176	1611704	1	39	M5x10 DIN 916/45H	1001485	1
7	Front Wheel	1916560	2	40	L=980mm	1003279	2
8	M12x60	1993419	2	41	---	---	---
9	Forks	1991447	2	42	XP00028	2990012	2
10	M10x80 DIN931	1001457	2	43*	---	---	---
11	M10 DIN 985	1001683	2	44*	LOGO VERMEIREN	T001002	1
12	M6x75 DIN 912/8.8	1993413	2	45	M6x35 DIN931	2995333	2
13	X002306	1502304	2	46*	---	1003558	1
14	X003698	1004550	2	47*	---	W620150	2
15	X002448	1517200	2	48*	---	W620151	2
16	D6,4 DIN 125 A	1001564	6	49	XP00108	GP00604	2
17	---	1003558	2	50	M8x76 DIN912/8.8	1998993	2
18	M6 DIN985	1001680	10	51	M8	1998994	2
19	M6x40	1006111	2	52	Rear Wheel	1914606	2
20	M6x60 DIN 912/8.8	1005957	4	53	Y004199	1511304	1
21	M6 DIN125A	1000973	4	54	Y004200	1511404	1
22	XP00099	2990067	4	55	Cap D19	1001602	2
23	D10,5 DIN 125 A	1001558	8	56	Cap D22	1990648	2
24	M6x16	1008459	2	* - Not Depicted ** - For Switz Only *** - Option FX + Carton U000254/			
25	X002305	1001545	4				
26	X002305	1993120	2				
27	D4.8X22 DIN 7981	2990070	2				
28	X005513	1992600	2				
29**	YP05288	2993604	2				
30	---	1993417	1				
31	Sterknop M6	2995334	2				
32	M12 DIN 127 A	1001572	2				
33	A0081572	1909113	1				

EXPLODED VIEW 275

2/2

Scale:

Date

Init.

Drafted

15/04/2003

AR

Verified

24/03/2025

AZ

Confirmed

24/03/2025

TK

Plan No.

E000042/T

