



no	part nu	part	QT		
1	1920656	Main frame ONE	1		
2	1918862	front fork	1		
3	1918856	STEERING AXLE ONE	1		
4	1920339	FRONTAXLE L	1		
5	1918854	FRONTAXLE R	1		
6					
7	1920443	COLOR PANEL SET FRONT green			
7	1920444	COLOR PANEL SET FRONTorange			
7	1920445	COLOR PANEL SET FRONTbleu			
7	1920446	COLOR PANEL SET FRONT pink			
8					
8					
8					
8					
9	1918852	FRONTCOVER	1		
10					
11					
12	1918851	SHROUD ONE	1		
13					
14					
15	1918858	BALL JOINT RIGHT	2		
16					
17					
18	1918859	CONNECTION ROD	2		
19					
20	1918857	BALL JOINT LE			
21					
22					
23					
24					
25					
26					

27					
28					
29					
30					
31					
32	1918855	FRONTWHEEL ONE		2	
33	1920632	FRONT BUMPER		1	
34	1920634	WASHER M4 NF E25-513		6	
35	1920633	BOLT M4X20 DIN 7985			
36					
37					
38					
39	1919090	POWER MODULE			
40					
41					
42	1920657	BATTERY CONNECTOR HOLDER		1	
43	1920658	BATTERY CONNECTOR		1	
44					
45					
46					
47					
48					
49					
50					
51					
52					
53					
54					
55					
56					
57	1918861	FRONT CONTACT POINT HOLDER		1	
58					
59	1918860	CONTACT POINT FRONT		4	


no	part nu	part	QT		part nu	part	QT
1					17		
2	1918886	HANDLE REAR	1		18		
3	1920439	COLOR PANEL SET REAR green			19		
3	1920440	COLOR PANEL SET REAR orange			20		
3	1920441	COLOR PANEL SET REAR bleu			21		
3	1920442	COLOR PANEL SET REAR pink			22		
4					23		
4					24		
4					25		
4					26	1918887	DIFF.+MOTOR+REM
					26	1920395	DIFF
					26	1920396	MOTOR
				NOT DEPICTED	1922014	BUSH TRANSMISION	1
					26	1920397	MOTOR BRAKE
5	1918853	COVER MIDDLE	1		27	1918888	REAR WHEEL
6					28	1918889	REAR CONTACT POINT HOLDER
7					29		
8					30		
9					31	1918890	CONTACT POINT REAR
11	1921454	REAR RED REFLECTOR			32		
12					33		
13	1918882	REAR COVER	1				
14							
15							
16							
not depicted	1922190	RED MOTORBRAKE LEVER COVER	1				

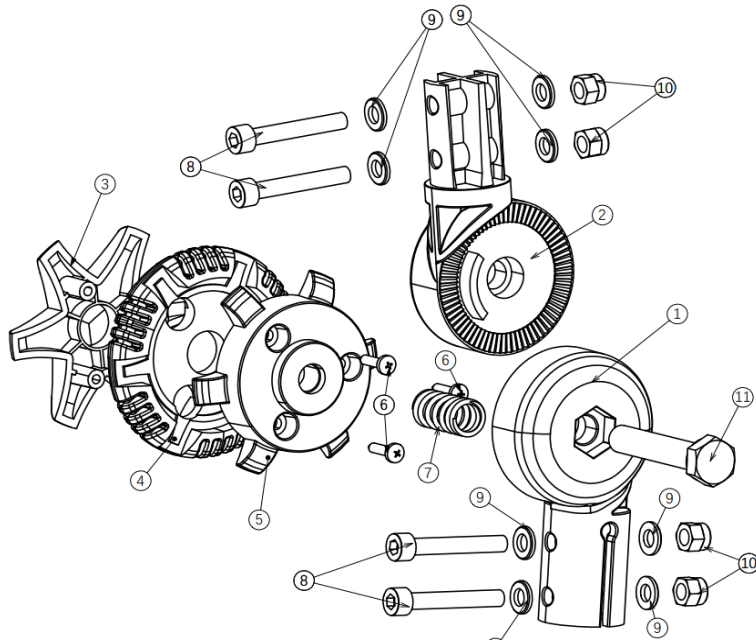


no	part nu	part	QT			
1						
2						
3	1921320	knob angle adjust outside	1			
4	1921319	knob angel adjust middle	1			
5	1921318	knob angle adjust inside	1			
6	1921322	bolt M5 X 16	3			
7	1921321	spring angle adjust knob	1			
8						
9						
10						
11	1921317	bolt steering angle adjust	1			

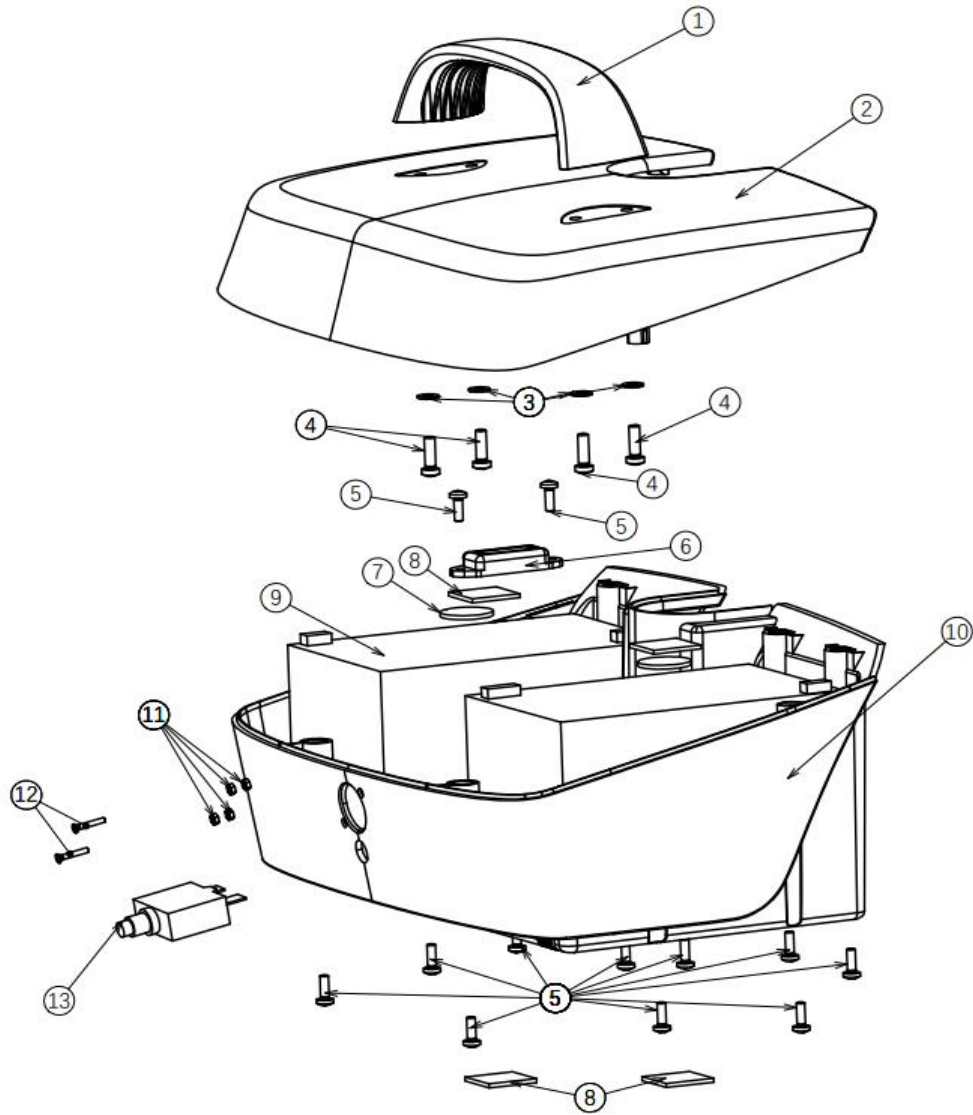

no	part nu	part	QT				
0	1920296	BATTERYBOX COMPLETE	1				
0	1918891	BATTERY BOX WITHOUT BATTERIES	1				
1							
2							
3							
4							
5							
6							
7							
8							
9	1919621	CPX15 -12	2				
10							
11							
12							
13	1920531	MAIN FUSE					
0	1919798	CHARGER	1	not depicted			

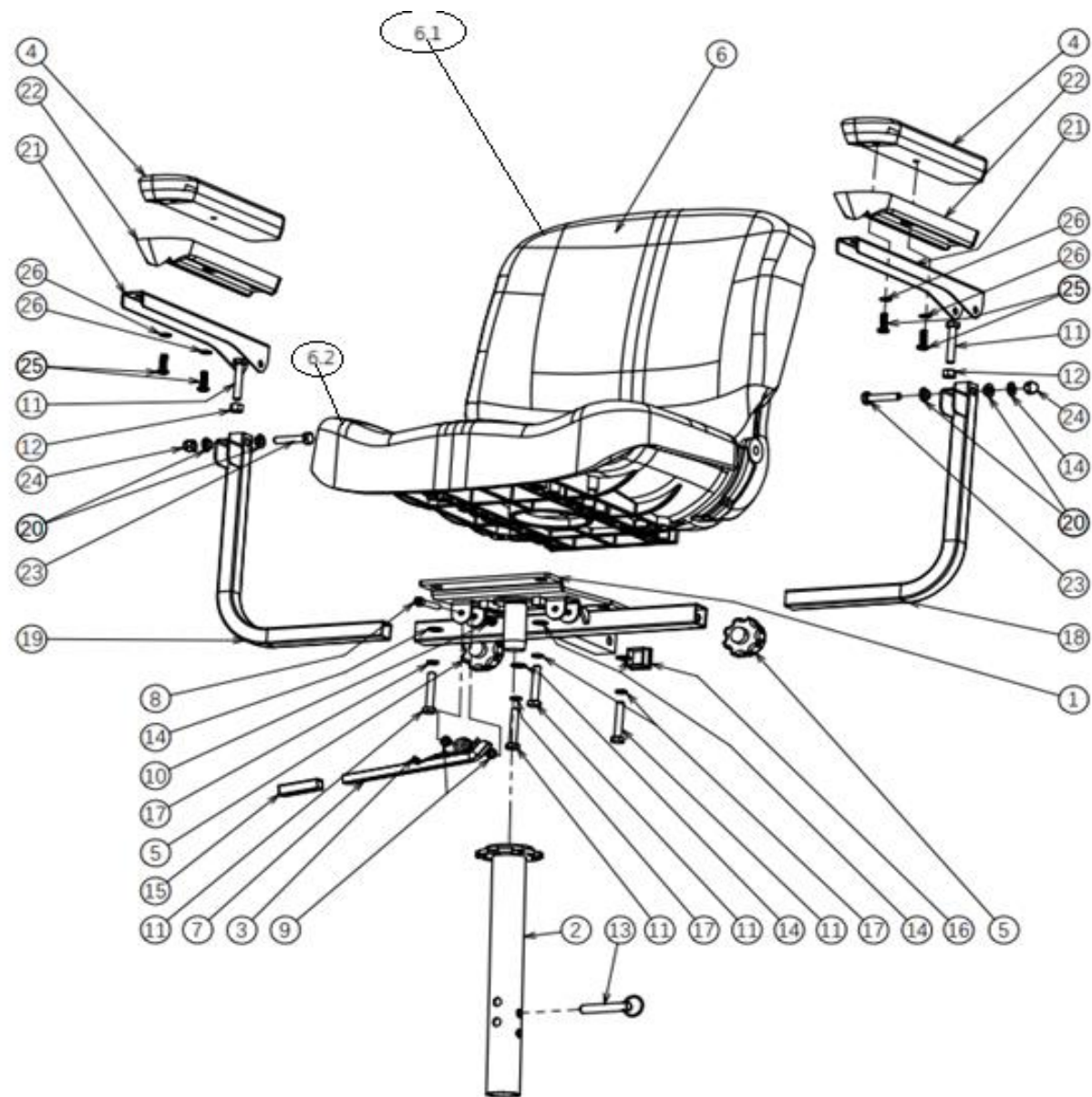


no	part nu	part	QT			
0						
1						
2	1918898	SEAT TUBE	1			
3						
4	1918895	ARM PAD	2			
5	1920266	BOLT ARMREST FIXATION	2			
6	1918899	CHAIR	1			
6.1	1920640	BACK	1			
6.2	1920641	SEAT	1			
7	1920645	SEAT HANDLE	1			
8						
8						
9						
10						
11						
12						
13						
14						
15						
16						
17						
18	1918903	ARMREST LOWER LEFT	1			
19	1918907	ARMREST LOWER RIGHT	1			
20						
21	1918931	ARMRESR UPPER PART	2			
22						
23						
24						
	1920830	ARMREST COMPLETE L	1			
	1920831	ARMREST COMPLETE R	1			
	1921438	SEAT PLATE COMP W/A TUBE	1			

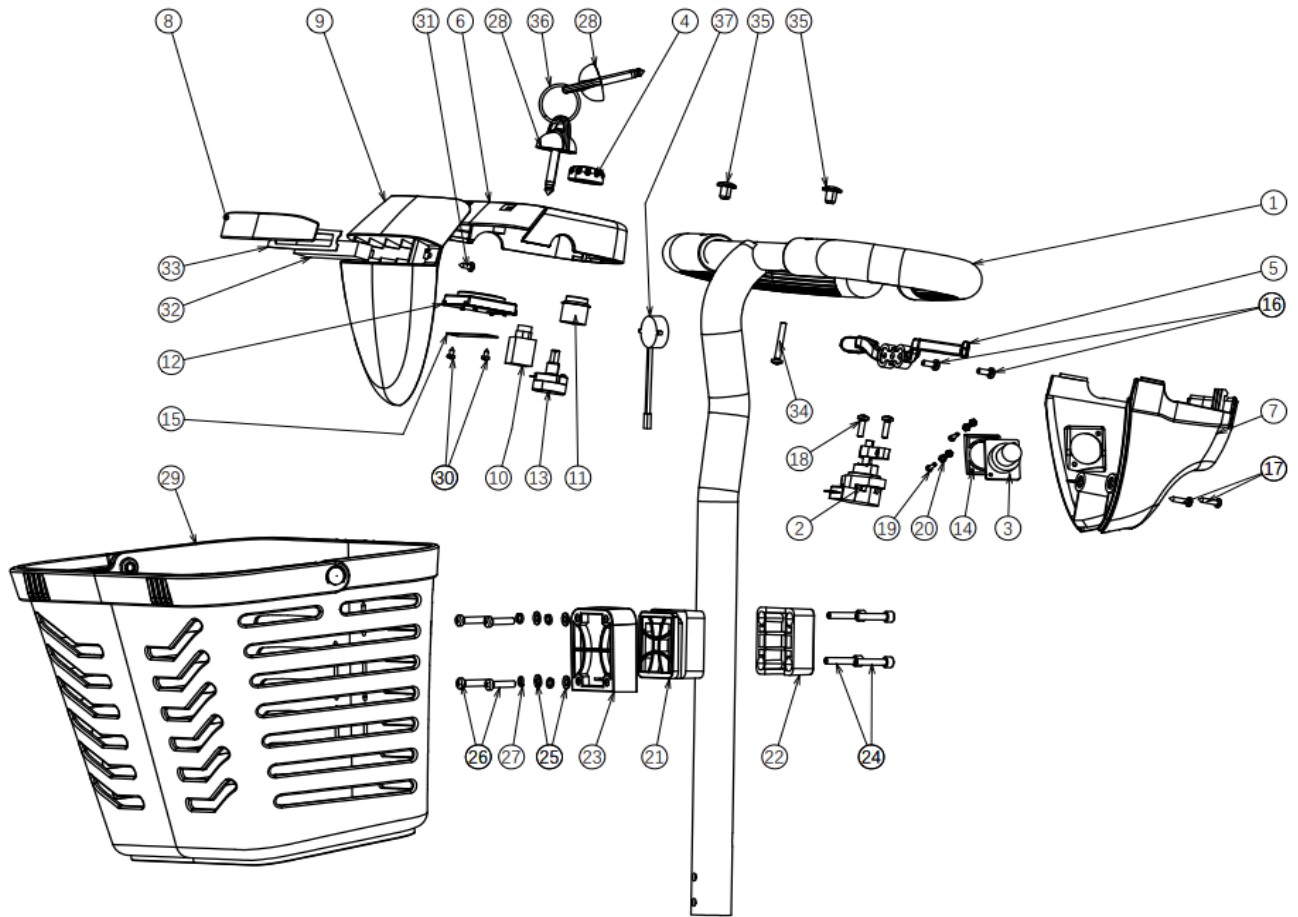

no	part nu	part	QT
1			
2	1918911	SPEED DIAL ASSEMBLY	1
3	1918918	CHARGER SOCKET	1
4			
5	1918910	SPEED LEVER	1
6			
7	1918913	STEERING COVER BACK	1

8	1921463	FRONT LIGHT COVER	1
9	1918914	STEERING COVER FRONT	1
10	1919437	CONTACT IGNITION	1
11	1920423	HORN SWITCH	1
12	1918912	BATTERY INDICATOR	1
13	1918909	POT SPEED	1
14			
15			
16			
17			
18			
19			
20			
21	1920502	ASSEMBLY PLATE BASKET MIDDLE	1
22	1920510	ASSEMBLY PLATE BASKET REAR	1
23	1920501	ASSEMBLY PLATE BASKET FRONT	1
	1920832	BASKET HOLDER COMPLETE	1
24	1920511	BOUT BZKT M5X25 DIN912	2
25	1920504	WASHER	4
26	1920505	BOLT M5X20	4
27			
28	1918908	IGNITION KEY	1
29	1920509	BASKET	1
30			
31			
32	1920629	FRONT LIGHT	1
33			
34			
35			
	1918977	MIRROR LEFT	1
	1918978	MIRROR RICHT	1
37	1920628	HORN	1

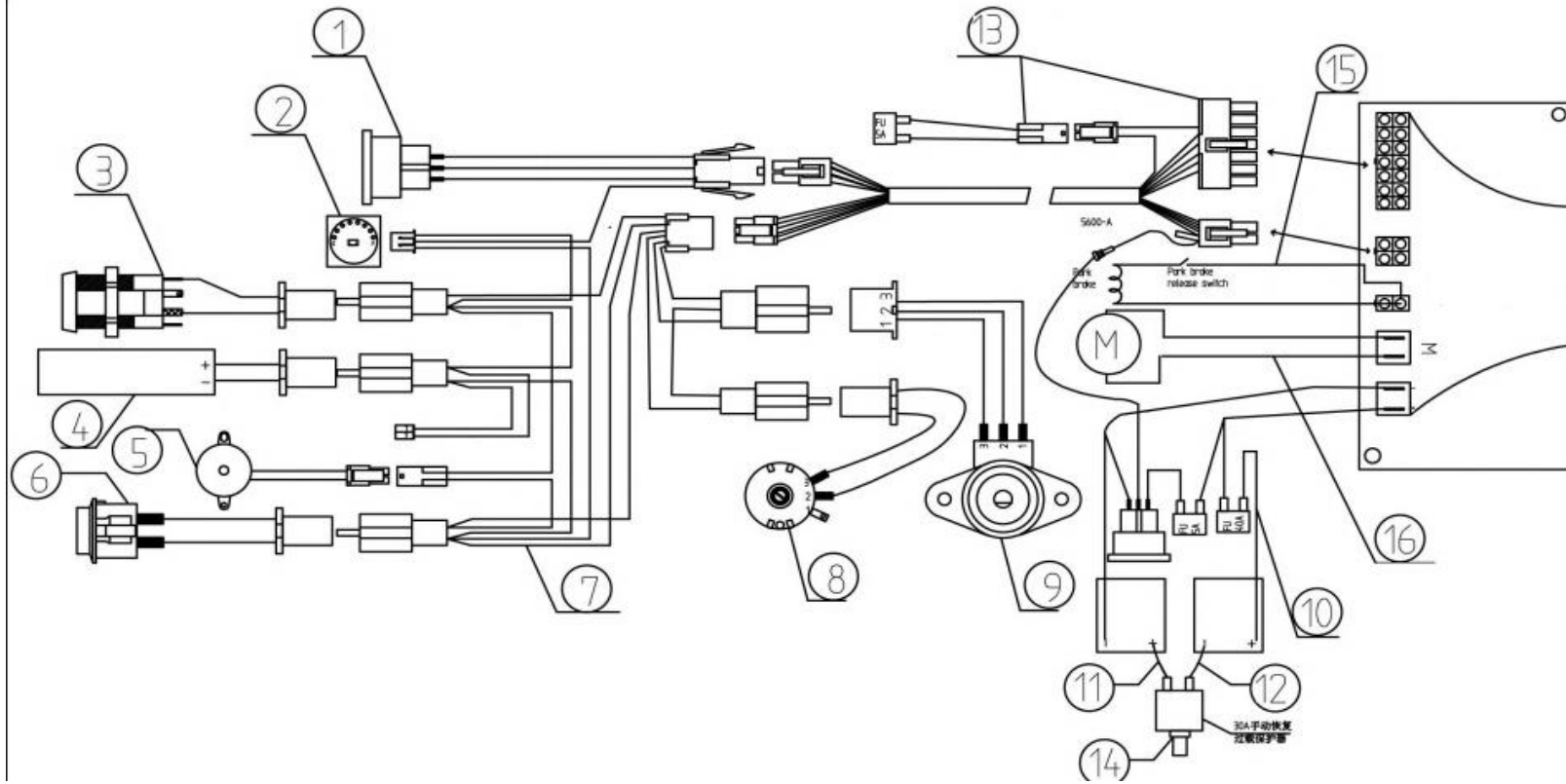

no	part nu	part	QT	
----	---------	------	----	--

1	1918918	CHARGER SOCKET	1	
2				
3				
4				
5				
6				
7	1918976	MAIN CABLE	1	
8				
9	1918909	POT. SPEED	1	
10	1921636	CABLE + FUSE	1	
11				
12				
13				
14				
15				
16				