


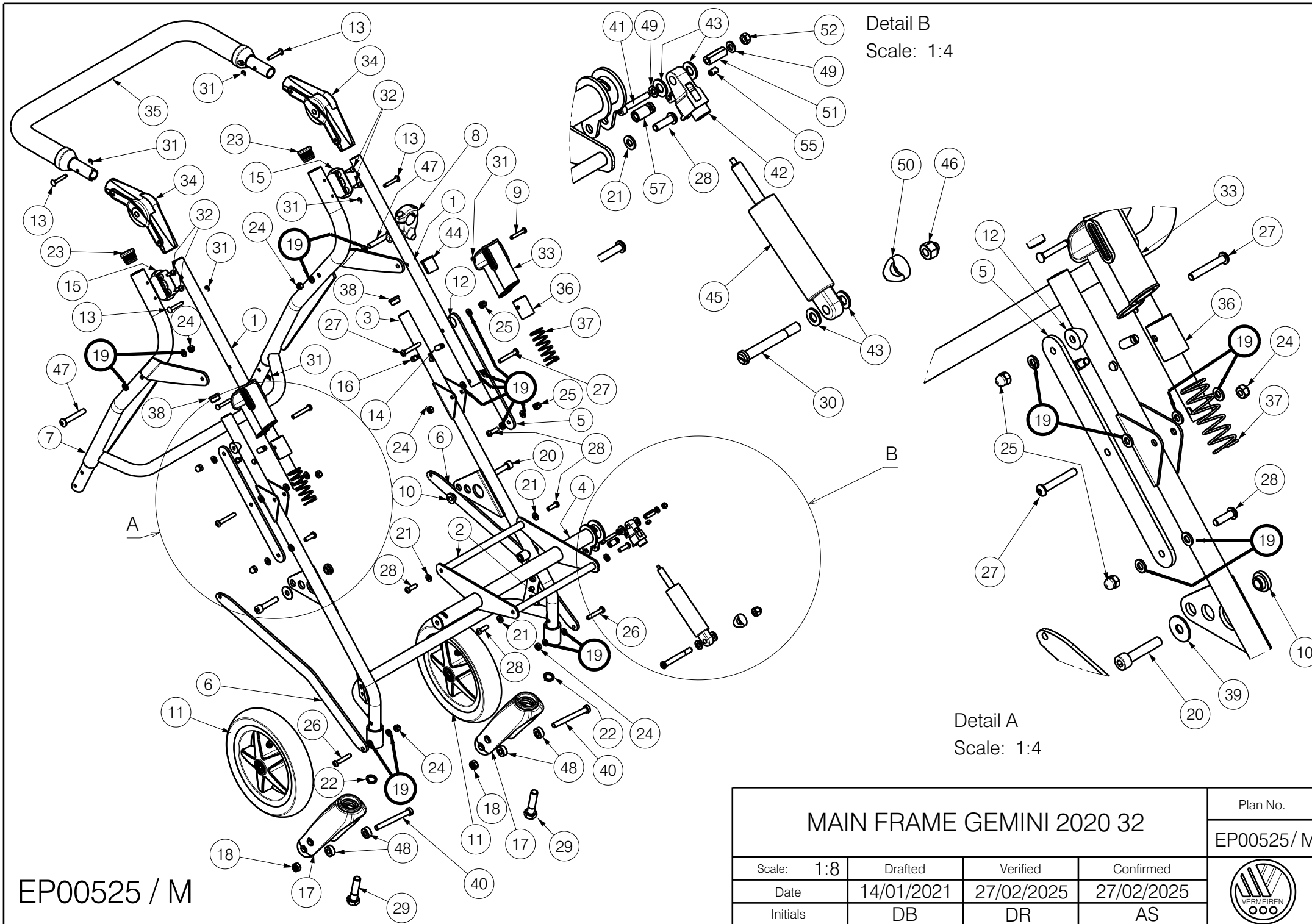

MAIN FRAME GEMINI 2020 32				Plan No.
				EP00525 / M
Scale: 1:8	Drafted	Verified	Confirmed	
Date	14/01/2021	27/02/2025	27/02/2025	
Initials	DB	DR	AS	

Material			
No.	Plan	Article	Quantity
1	YP08602	2622554	2
2	XP03114	GP24500	2
3	YP09337	2644754	1
4	YP09332	2644254	1
5	YP08320	2613554	2
6	YP08321	2613654	2
7	YP08399	2615354	1
8	XP03165	1996806	1
9	XP00138	1817200	2
10	XP00146	1818700	2
11	XP03131	1917241	2
	XP03199	1917465	2
12	X000586	1005742	2
13	X002506	1525200	4
14	X002671	1552000	2
15	X002679	1994452	2
16	X002698	1552300	2
17	P00006412	1991447	2
18	M8 DIN 985	2994009	2
19	M6 DIN 125 PVC	2994068	20
20	M8x40 DIN 912	2994915	2
21	M6 DIN 125	2994414	4
22	DIN7980 M12	2996210	2
23	Tube Cap D25	1001608	2
24	M6 DIN 985	2994063	6
25	M6 DIN 1587	2994059	4
26	M6x35 ISO 7380	2994131	2
27	M6x40 ISO 7380	2994066	6
28	M6x20 ISO 7380	2994010	6
29	M12x45 DIN 931	2994917	2
30	XP03422	2651100	1
31	D4 DIN 6799	1993722	6
32	D4.8x22 DIN 7981	2994918	4
33	---	2990084	2

Material			
No.	Plan	Article	Quantity
34	---	2990085	2
35	XP03168	2996677	1
36	---	2990093	2
37	---	2990094	2
38	Tube Plug D19	1001602	2
39	M8 DIN 9021	2994911	2
40	M8x75 DIN 7984	2994912	2
41	M5x30 DIN 912	2995561	1
42	---	1994748	1
43	M8 DIN 125 PVC	2994415	4
44	XP03164	2996749	1
45	XC00813	3992509	1
46	M8 DIN 986	1902024	1
47	M6x50 ISO 7380	2994965	2
48	XP03139	GP00288	4
49	M5 DIN 125 PVC	1000972	2
50	X000862	1006463	1
51	X004906	1757400	1
52	M5 DIN 985	2994006	1
* 53	---	1992149	1
* 54	---	1908105	2
55	---	1994749	1
* 56	---	2993049	0,7m
57	---	1994750	1

Stickers			
No.	Description	Article	Quantity
*58	Red band	T001249	2
*59	White band	T001250	2

Add stickers to the string bag,
do not put on the stroller

* - not depicted

Set 2990084			
No.	Plan	Article	Quantity
33	---	2990084	1
36	---	2990093	1
37	---	2990094	1

MAIN FRAME GEMINI 2020 32

Scale: 1:8

Drafted

Verified

Confirmed

Date

14/01/2021

27/02/2025

27/02/2025

Initials

DB

DR

AS

Plan No.

EP00525/ M

EP00525 / M