


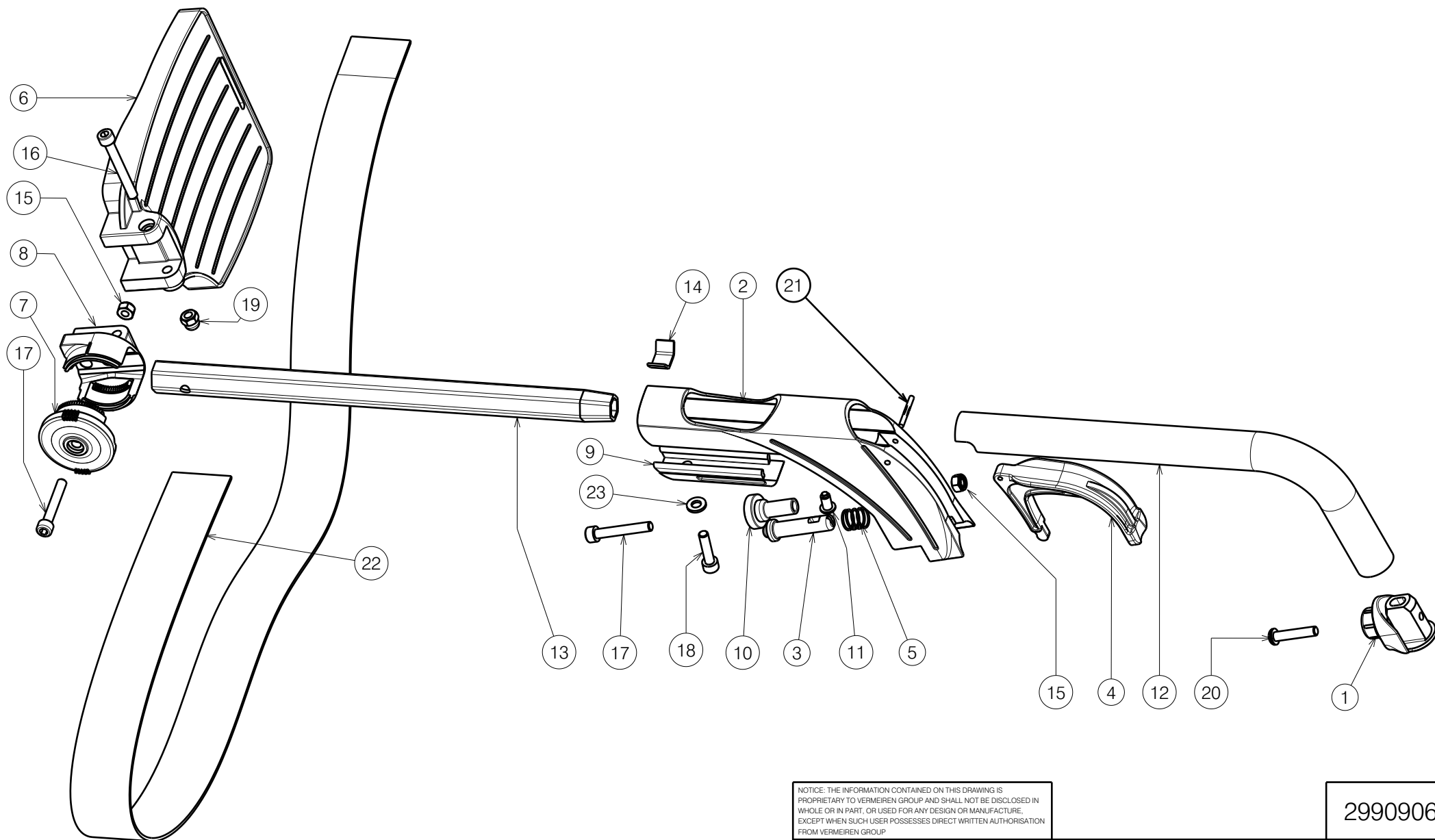

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

2990906

ADJUSTABLE B06 V200 V300 V300DL

Plan No.

E000629/N

Scale: 3:10	Drafted	Verified	Confirmed
Date	28-04-2010	08-07-2025	08-07-2025
Initials	TM	MP	SD



Material			
No.	Plan N°	Article N°	Quantity
1	X004414	1904130	1
2	X004416	1904202	1
3	X004417	1719200	1
4	X004418	1904207	1
5	X004423	1717300	1
6	X004570	1904646 (L)	(39-42) 1
	X004571	1904647 (R)	(39-42) 1
	X004457	1904475 (L)	(44-46) 1
	X004453	1904210 (R)	(44-46) 1
	X004572	1904648 (L)	(48-60) 1
	X004573	1904649 (R)	(48-60) 1
6*	XC00158	3991003 (L)	(39) 1
	XC00157	3991002 (R)	(39) 1
7	X004464	1904212	1

Material			
No.	Plan N°	Article N°	Quantity
8	X004465	1904213	1
9	X004658	1904968	1
10	X004660	1905195	1
11	X004687	1905639	1
12	Y015007	17300..	1
13	Y016499	F006269	1
14	Y016501	1733500	1
15	M6 DIN985	1001680	2
16	M6x50 DIN912	1007970	1
17	M6x40 DIN912	1003909	2
18	M6x25 DIN912	1001494	1
19	Nut M6	1008492	1
20	M6x30 ISO7380	1009588	1
21	D3x26.5 DIN 1475	1912021	1
22	---	1905642	1
23	M6 DIN125	1001564	1

* Only for Express

Article numbers for sets			
No.	Plan	Article	Quantity
I	Butt set	17416	2
II	Footplate 39-42	1906153	1
	Footplate 44-46	1906154	1
	Footplate 48-50	1906155	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

2990906

ADJUSTABLE B06 V200 V300 V300DL

Plan No.

E000629/N

Scale: --	Drafted	Verified	Confirmed
Date	28-04-2010	08-07-2025	08-07-2025
Initials	TM	MP	SD

