


Put in front or behind seat
depending on seat depth

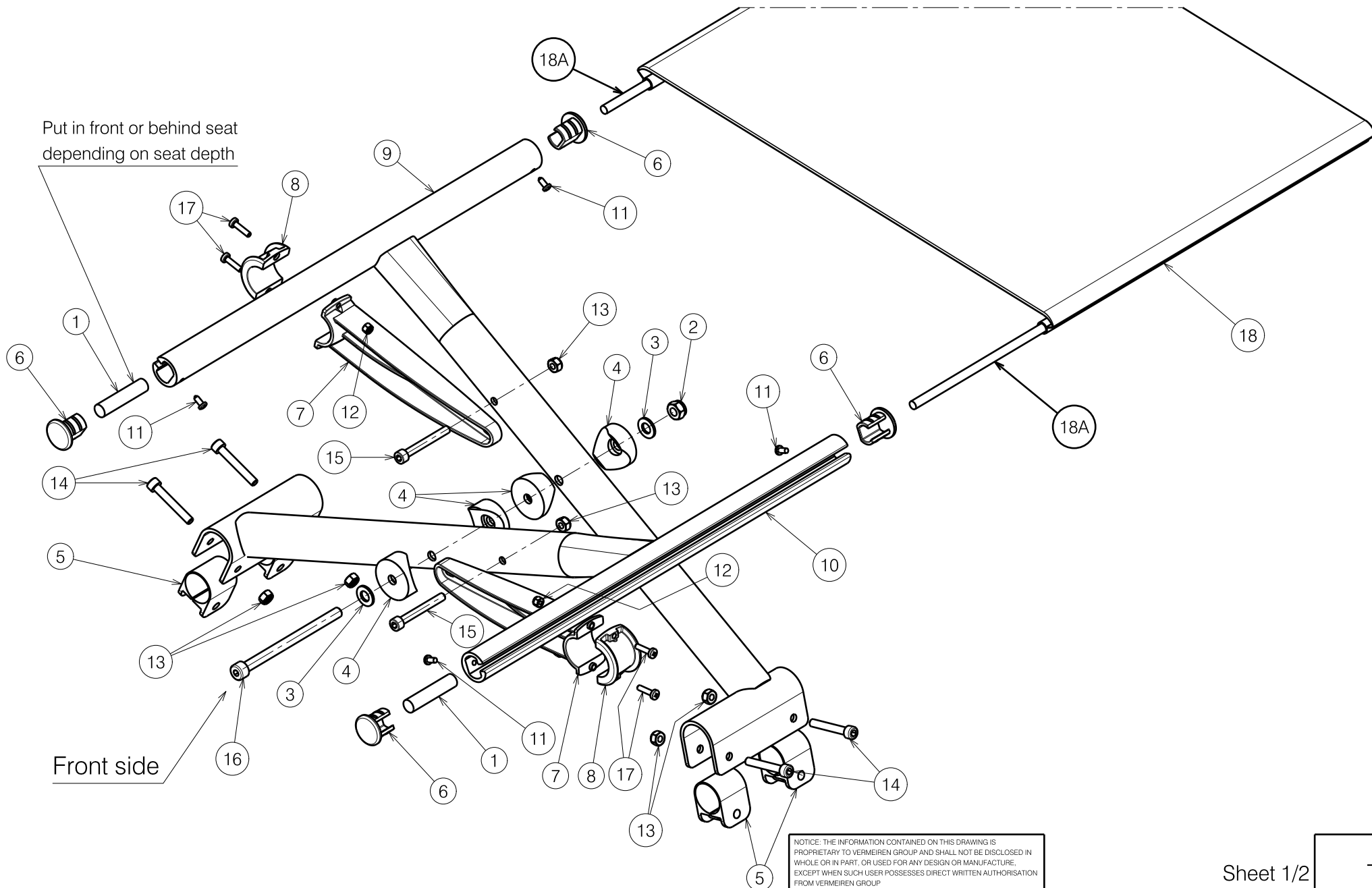

Front side

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 1/2

D200N CROSS

Plan No.

E00077545/B.1

Scale: 1:4

Drafted

Verified

Confirmed

Date

6/03/2024

11/03/2024

11/03/2024

Initials

SD

SD

PJ



Material			
No.	Plan	Article	Qty
1	P00077544	192xxxx	2
2	X001016	1001682	1
3	X001119	1001554	2
4	X002133	1992424	4
5	X003692	1998763	4
6	X004452	1904058	4
7	Y018378	F006644 (38)	2
	Y018379	F006645 (40)	2
	X004606	1905091 (44/48)	2
	X004607	1905092 (42/46/50)	2
	X005002	1907928 (52)	2
8	X004620	1905090	2
9	YC01901	1C67954 (38)	1
	YC01902	1C68054 (40)	1
	YC01903	1C68154 (42)	1
	YC01904	1C68254 (44)	1
	YC01905	1C68354 (46)	1
	YC01906	1C68454 (48)	1
	YC01907	1C68554 (50)	1
	YC01908	1C68654 (52)	1

Material			
No.	Plan	Article	Qty
10	YC01909	1C68754 (38)	1
	YC01910	1C68854 (40)	1
	YC01911	1C68954 (42)	1
	YC01912	1C69054 (44)	1
	YC01913	1C69154 (46)	1
	YC01914	1C69254 (48)	1
	YC01915	1C69354 (50)	1
	YC01916	1C69454 (52)	1
11	1001530	1001530	4
12	M4 DIN985	1001678	4
13	M6 DIN985	1001680	6
14	M6x40 DIN912	1003909	4
15	1007970	1007970	2
16	M8x110 DIN912	1991711	1
17	M4x16	2990185	4
18	---	3994747 (38)	1
	---	3994748 (40)	1
	---	3994749 (42)	1
	---	3994750 (44)	1
	---	3994751 (46)	1
	---	3994752 (48)	1
	---	3994753 (50)	1
	---	3994754 (52)	1
18A	---	3994764	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/2

D200N CROSS				Plan No.
				E00077545/B.1
Scale: 1:4	Drafted	Verified	Confirmed	
Date	6/03/2024	11/03/2024	11/03/2024	
Initials	SD	SD	PJ	