


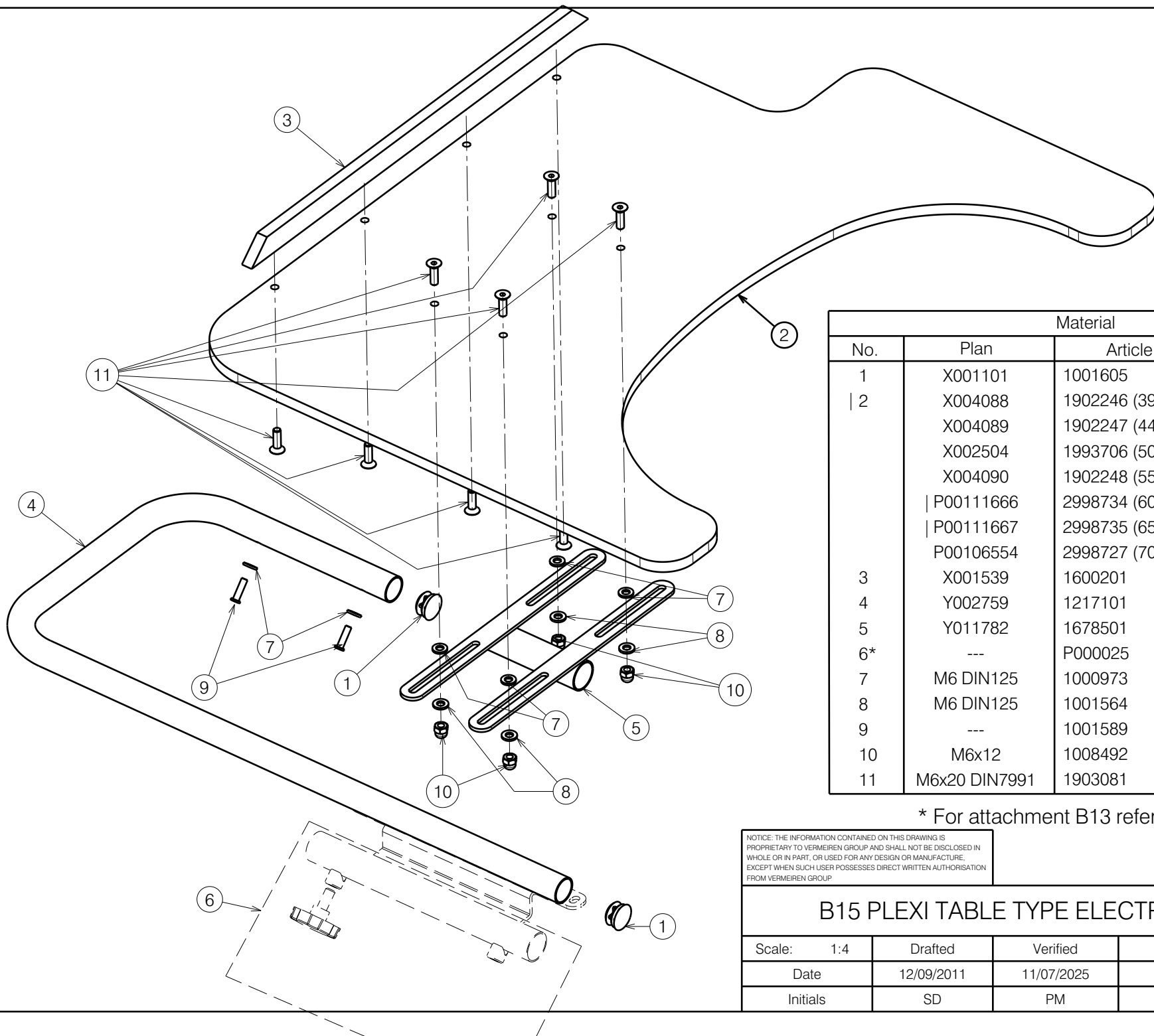

Material			
No.	Plan	Article	Quantity
1	X001101	1001605	2
2	X004088	1902246 (39/40/42)	1
	X004089	1902247 (44/45/46)	1
	X002504	1993706 (50)	1
	X004090	1902248 (55)	1
	P00111666	2998734 (60)	1
	P00111667	2998735 (65)	1
	P00106554	2998727 (70)	1
3	X001539	1600201	1
4	Y002759	1217101	1
5	Y011782	1678501	1
6*	---	P000025	1
7	M6 DIN125	1000973	6
8	M6 DIN125	1001564	4
9	---	1001589	2
10	M6x12	1008492	4
11	M6x20 DIN7991	1903081	8

* For attachment B13 refer to P000025

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

B15 PLEXI TABLE TYPE ELECTRO

Scale: 1:4	Drafted	Verified	Confirmed
Date	12/09/2011	11/07/2025	11/07/2025
Initials	SD	PM	MK

Plan No.

E000744/F

