



Standard version			
No.	Plan	Article	Quantity
1	P00109010	2228954	2
2	P00117455	2229154	2
3	M6x60 ISO7380	1916196	4
4	M6 DIN986	2994059	4
5	M6 DIN125	2994414	4

Quantities are for one wheelchair

30° version  
Left side depicted

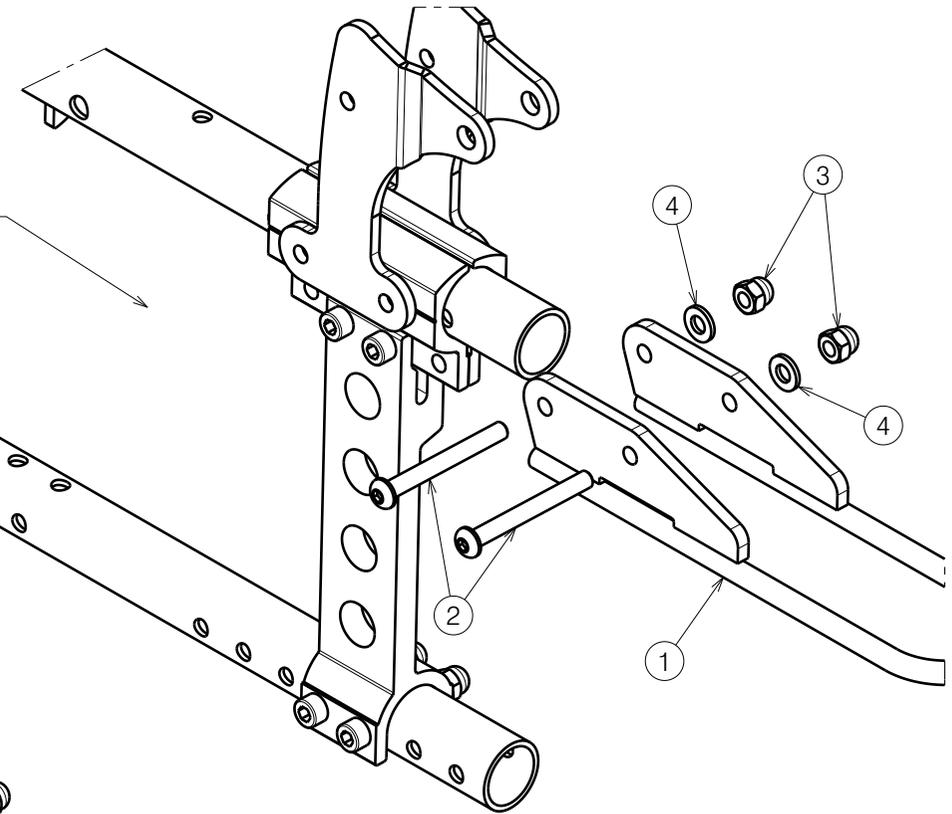

Standard version  
Left side depicted

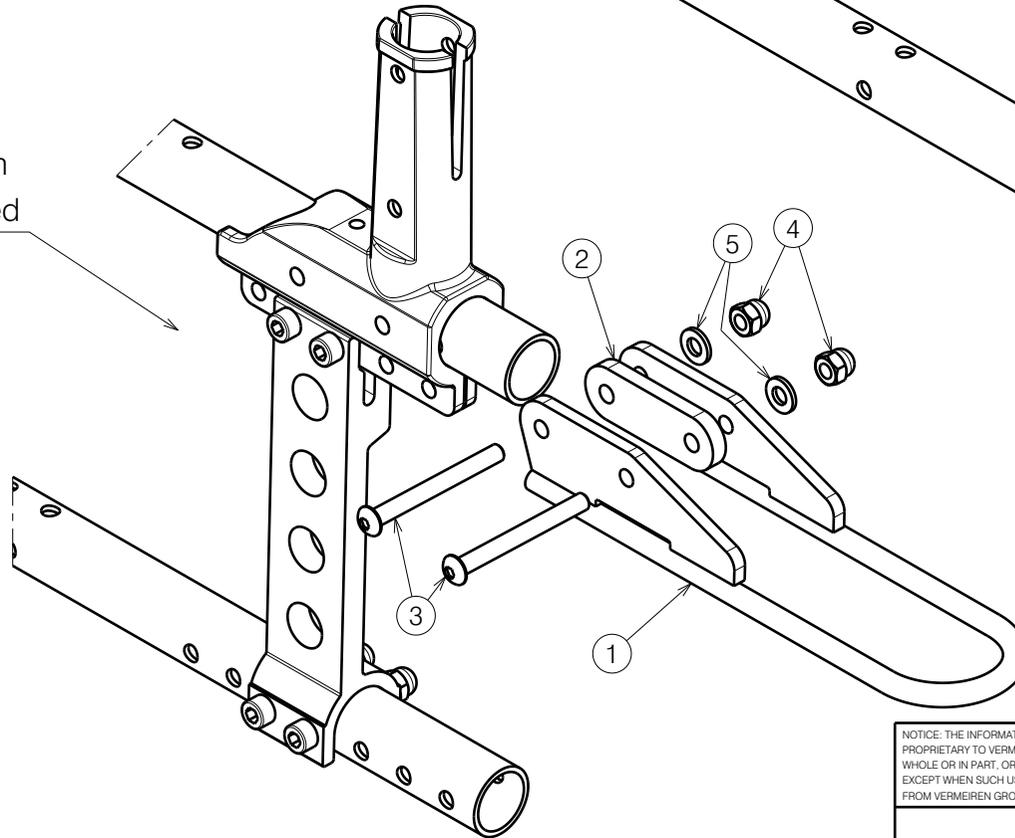

30° version			
No.	Plan	Article	Quantity
1	P00109010	2228954	2
2	M6x55 ISO7380	2994376	4
3	M6 DIN986	2994059	4
4	M6 DIN125	2994414	4

Quantities are for one wheelchair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

VERS0 REAR TAXI HOOK				---
				Plan No. E00117456/A.1
Scale: 2:5	Drafted	Verified	Confirmed	
Date	27/10/2025	27/10/2025	27/10/2025	
Initials	SD	SD	TS	