


Material			
No.	Plan	Article	Quantity
1	YP06972	2743765	1
2	P00050248	2670465	1
3	XP02231	2993879	1
4	XP02229(R)	2993877	1
	XP02230(L)	2993878	1
5	XP02079	GP15300	2
*6	P00107339	GP32226	2
7	XP01914(L)	GP13300	1
	XP01915(R)	GP13400	1
8	XP02922	GP22126	2
9	---	1007521	1
10	M3	2994899	2
11	M6x20	2994010	2
	M6x35	1915941	2
12	M6 DIN 125	2994414	2
13	P00049519	2997554	1

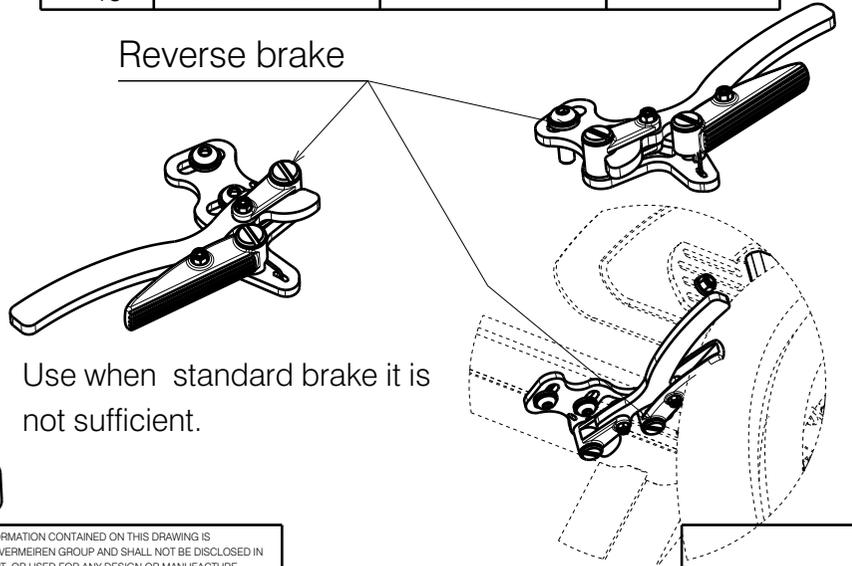

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

SPORTS BRAKE (R,L) TRIGO

Scale:	1:1	Drafted	Verified	Confirmed
Date	10/01/2018	09/09/2025	09/09/2025	
Initials	AS	PM	AS	

Plan No.
EP00370/K

EP00370 / K * Use an radial riveting machine after adjusting and tightening.
** Use if standard version collide with frame.