

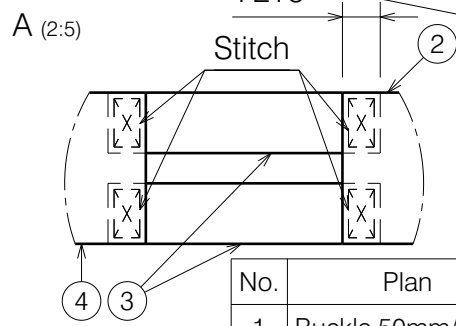


4-BELTS BACKREST - assembly:

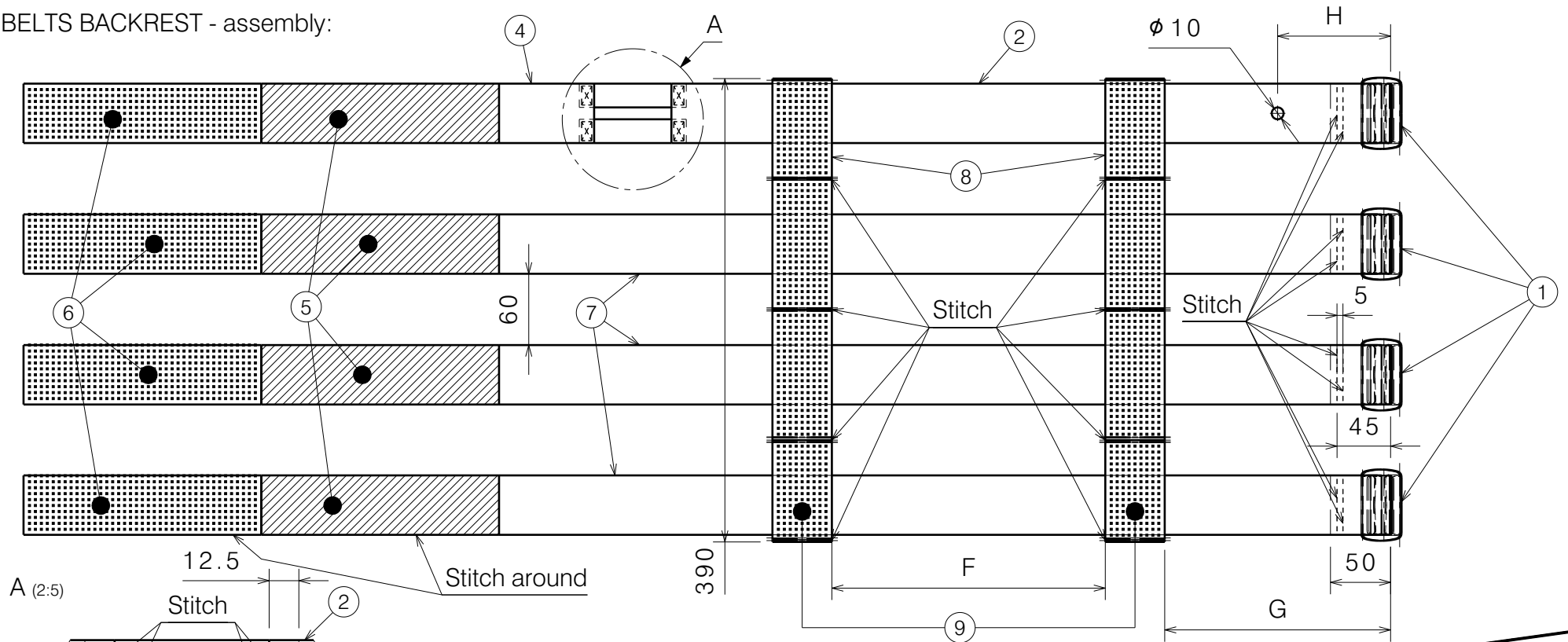

No.	Plan	Article	Qty
1	Buckle 50mm/Black	1009511	4
2	Belt 50mm	1009520	1xA [m]
3	Belt 20mm	1009515	0,18 [m]
4	Belt 50mm	1009520	1xB [m]
5	Vecro 50mm	1009792	4xD [m]
6	Vecro 50mm Hook	1009791	4xE [m]
7	Belt 50mm	1009520	3xC [m]
8	Belt 50mm	1009520	1,56 [m]
9	Vecro 50mm Hook	1009791	1,56 [m]
10*	Black thread	W200016	N/A

Complete 4-belts L44 backrest	
Size	Article
35	2996479
40	2991684
45	2991685
50	2991686

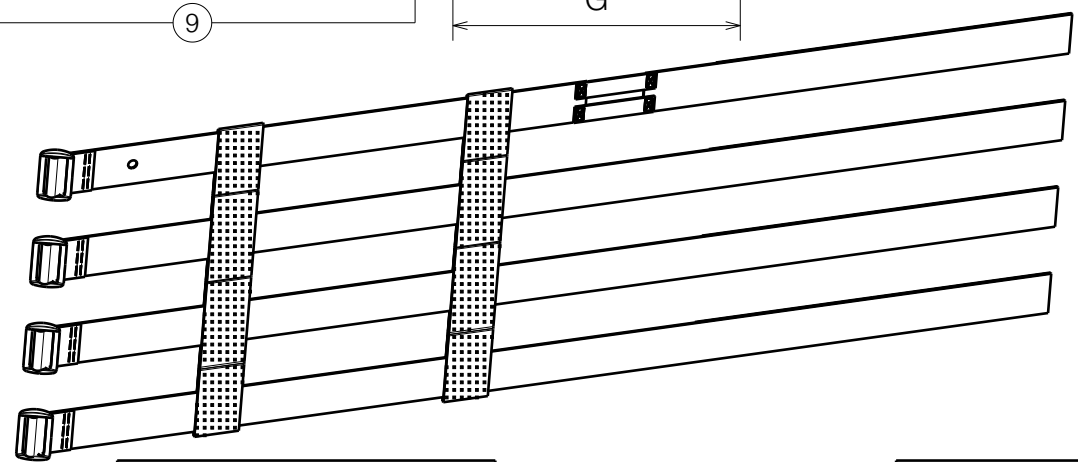

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

L44 / BACKREST 4-BELTS+FLAP/ INOVYS 2			
Scale: 1:5	Drafted	Verified	Confirmed
Date	05.03.2024	07/03/2024	07/03/2024
Initials	AZ	AZ	AS

Plan No. E00077474/A1.1


* - not depicted

4-BELTS BACKREST - material:


1 - 1009511 - Buckle




2 - 1009520 - Belt

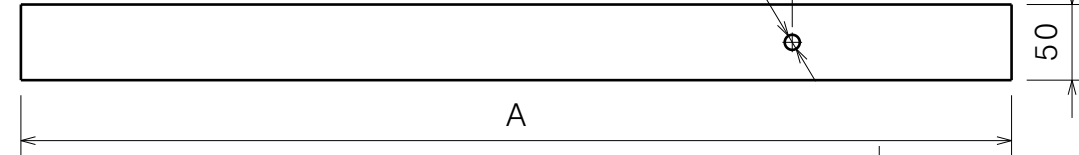


3 - 1009515 - Belt

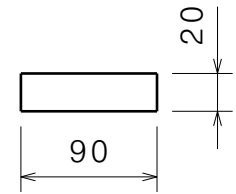


4 - 1009520 - Belt

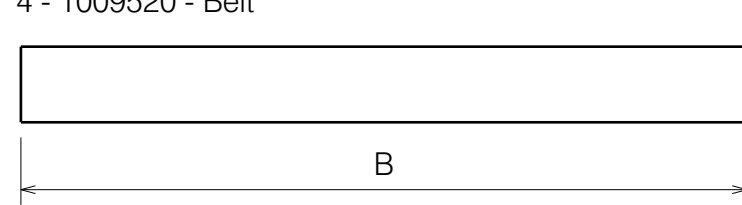

5 - 1009792 - Velcro

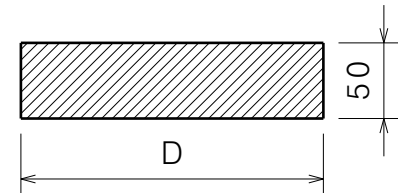


6 - 1009791 - Velcro hook

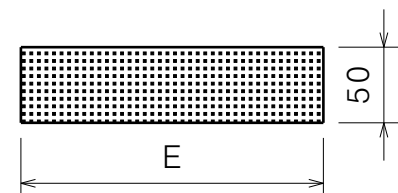


C


7 - 1009520 - Belt

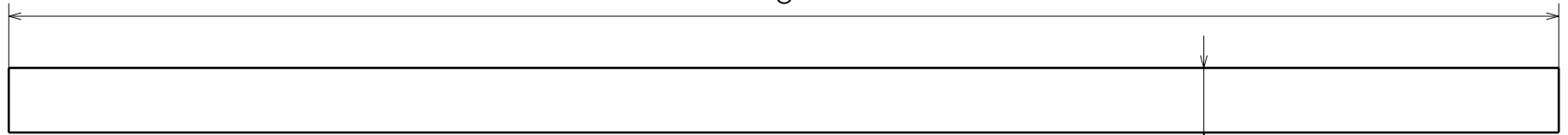

8 - 1009520 - Belt

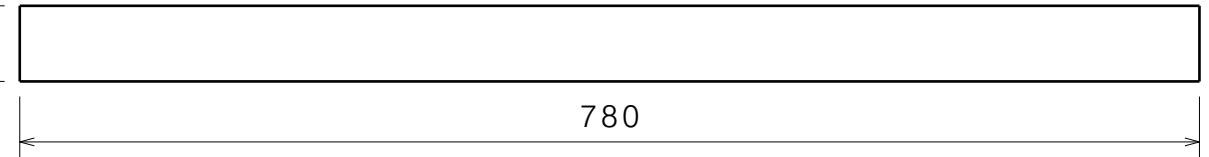

9 - 1009791 - Velcro hook

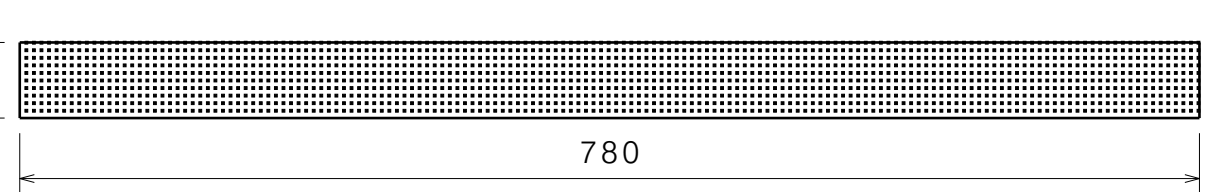

Size	35	40	45	50
A	550	630	660	730
B	400	430	490	620
C	1020	1120	1220	1410
D	160	170	200	270
E	160	170	200	270
F	170	200	230	320
G	180	220	190	190
H	95	120	95	95

* - not depicted

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.


ISO 2768-1 c (coarse)			
⊖ Significant characteristic			
Linear dimensions		Angular dimensions	
0.5 up to 3	±0.2	up to 10	±1°30'
over 3 up to 6	±0.3	over 10 up to 50	±1°
over 6 up to 30	±0.5	over 50 up to 120	±0°30'
over 30 up to 120	±0.8	over 120 up to 400	±0°15'
over 120 up to 400	±1.2	over 400	±0°10'
over 400 up to 1000	±2		
over 1000 up to 2000	±3		
over 2000 up to 4000	±4		

L44 / BACKREST 4-BELTS+FLAP/ INOVYS 2^{2/4}

Scale: 1:5	Drafted	Verified	Confirmed
Date	05.03.2024	07/03/2024	07/03/2024
Initials	AZ	AZ	AS

Plan No.
E00077474/A.1.1

FLAP BACKREST - assembly:

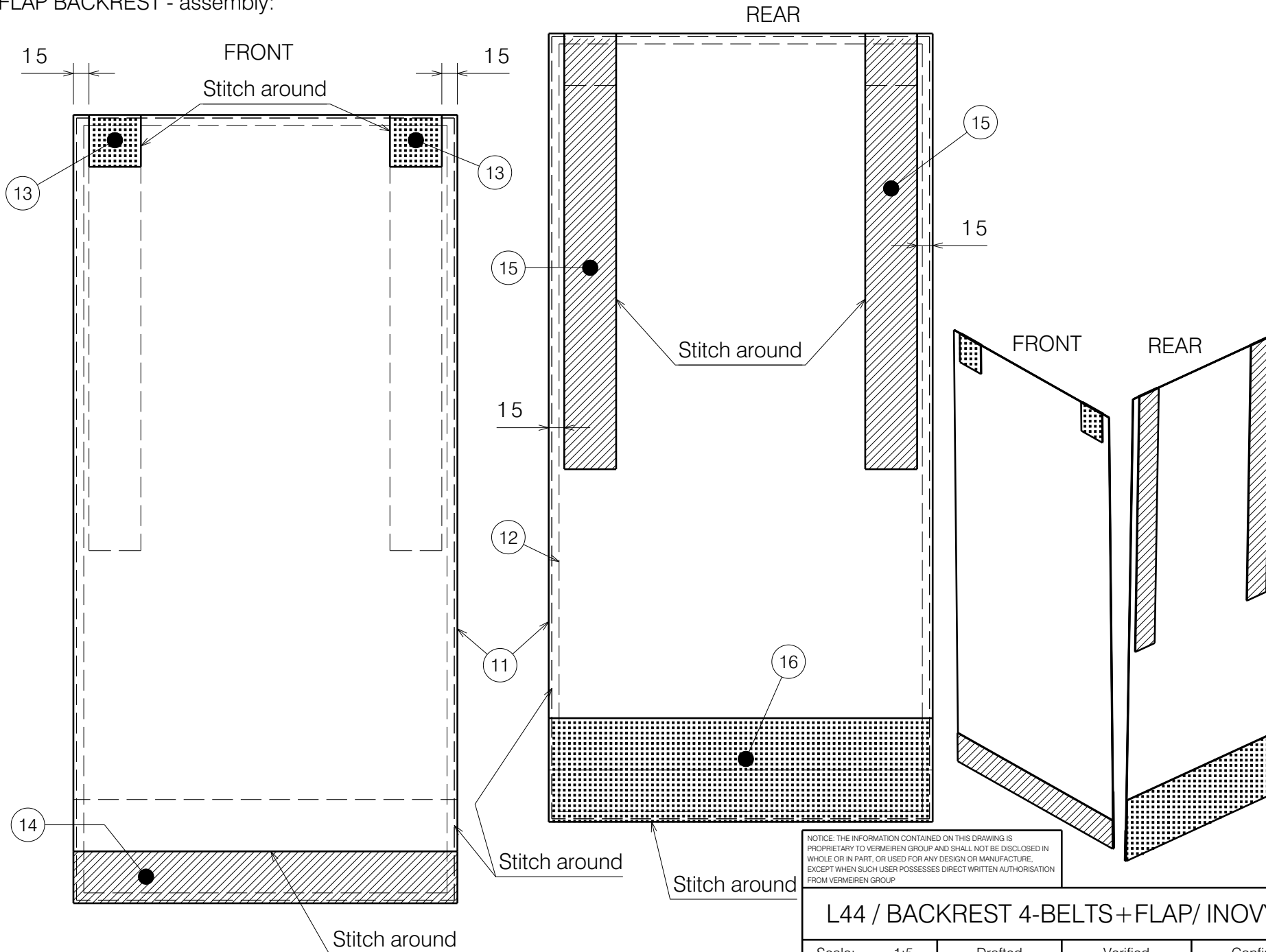

Complete L44 flap (4-belts)	
Size/Colour	Article
35/Black	2998350
35/Grey	2998351
35/Mokka	2998352
40/Black	2998353
40/Grey	2998354
40/Mokka	2998355
45/Black	2998356
45/Grey	2998357
45/Mokka	2998358
50/Black	2998359
50/Grey	2998360
50/Mokka	2998361

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

L44 / BACKREST 4-BELTS + FLAP/ INOVYS 2_{3/4}

Scale: 1:5	Drafted	Verified	Confirmed
Date	05.03.2024	07/03/2024	07/03/2024
Initials	AZ	AZ	AS

Plan No.
E00077474/A1.1


* - not depicted

FLAP BACKREST - material:


11 - Soft (Colour)

12 - 1994137 - Skai Black

14 - 1009792 - Velcro



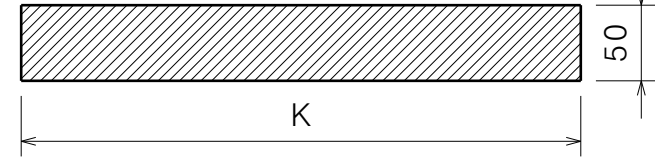


15 - 1009792 - Velcro

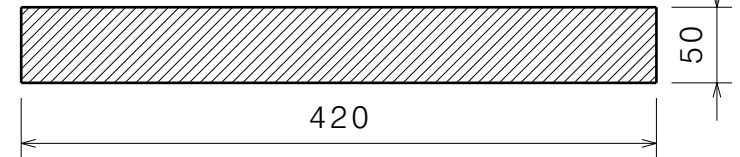

16 - 2991788 - Velcro hook

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13 - 1009791 - Velcro hook

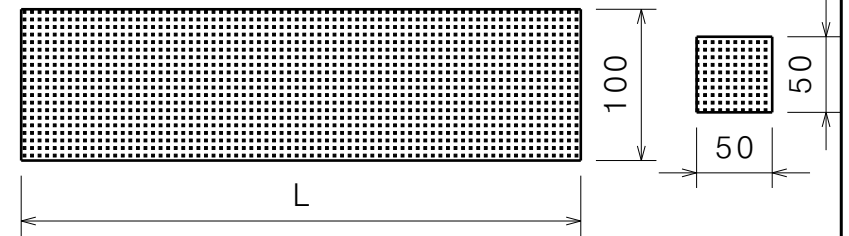

Size	35	40	45	50
I	335	345	395	435
J	300	310	360	410
K	310	320	370	420
L	310	320	370	420

No.	Plan	Article	Qty
11	Soft Black	1918292	0,160 [m]
	Soft Grey	1907554	0,160 [m]
	Soft Mokka	1908434	0,160 [m]
12	Skai Black	1994137	0,213[m]
13	Vecro 50mm Hook	1009791	0,100 [m]
14	Vecro 50mm	1009792	1xK [m]
15	Vecro 50mm	1009792	0,840 [m]
16	Vecro 100mm Hook	1009791	1xL [m]
17*	Thread Black	W200016	N/A
	Thread Grey	W200016	N/A
	Thread Mokka	2993221	N/A

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

L44 / BACKREST 4-BELTS+FLAP/ INOVYS 2_{4/4}

Scale:	1:5	Drafted	Verified	Confirmed
Date	05.03.2024	07/03/2024	07/03/2024	
Initials	AZ	AZ	AS	

Plan No.
E00077474/A1.1

* - not depicted