


Material			
No.	Plan	Article	Quantity
1	X002976	1995343	2
2A	P00065423	2684165	1
2B	P00071838	2684265	1
3	Y004693	1518204	2
4	Y004751	1521604	2
5	M6x40 ISO7380	1006111	4
6	M6 DIN7349	1003321	1
7	M6x25	1903394	1
8	---	1901208	1
9	---	1901745	1
10	---	1994568	4
11	DIN471	1995259	4
12A	P00064457	2683565	1
12B	P00071833	2683665	1
12C	P00071837	2684065	1
13A	X005892	G103100	1
13B	X005891	G103000	1
14	X005893	1915848	1
16	M6	1008492	4
17	XP03058	GP23399	2
18	P00064455	GP29965	2
19	D3x26 DIN 1481	1908821	2
20	5x5x10 DIN6885A	2998153	2

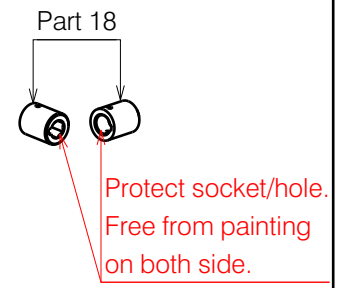

EP00521 /E

2B-12C- 13A Seat widths 38-40  
 2A-12A - 13A Seat widths 42-44  
 2A-12B - 13B Seat widths 46-48-50

DRIVE SYSTEM HEM2 TRIGO				Plan No.
				EP00521 /E
Scale	1:6	Drafted	Verified	Confirmed
Date	03/07/2020	12/12/2023	12/12/2023	
Init.	AS	AZ	TK	

