

B (1:5)
(x2)


A (1:5)
(x2)

1/2

HEAD SIDE WITH WOODEN LATHS / ILLICO			
Scale:	1:10	Drafted	Verified
Date	02/12/2021	06/05/2024	06/05/2024
Initials	DR	MK	AZ

Plan No.
E00026487/H1.1

Material			
No.	Plan	Article	Quantity
1	P00018249	26450...	1
2	P00026641	26451...	2
3	YP05898DP	27893...	1
4#	YP08882	2629303	2
5**	YP05894	2784700	2
6	YP06216	2784803	1
7	YP06217	2795303	1
8	YP06448	2729800	1
9	X001842	G053400	2
10#	XP01795	2993539	2
11#	XP02992	GP22400	2
12#	XP01899	GP12700	4
13	XP01954	2993706	2
14	D5,3 DIN125	1000972	4
15	Nut M5 DIN986	1992804	2
16	X001016	1001682	4
17	Washer M8 DIN125	1003199	8
18#	M6x12	1007000	8
19	M6x16	1008459	4
20	M5x40	2993786	2
21	M8x45	1994473	2
22	Nut M6 DIN985	1001680	2
23	M6x20	1006999	2
24	M8x35	1001444	2
25	Pen D10x50	1998885	2
26	Tube Cap 20x20	1001612	6
27	LA27	1901113	1
28#	DIN471 15,7-1mm	2993606	2
29	Wheel D29	2993761	2
30	XP01421	2993116	6
31	XP01420	2993117	12
32	Cap Rubber 8mm	1001637	2
33	Nut M6 DIN315	1001520	4
34	Washer M6 DIN125	1000973	4
35	Tube Cap 16x16	1005899	2
36*	Motor Cable	1912902	1
37*	Label "2" for cable	1995836	1

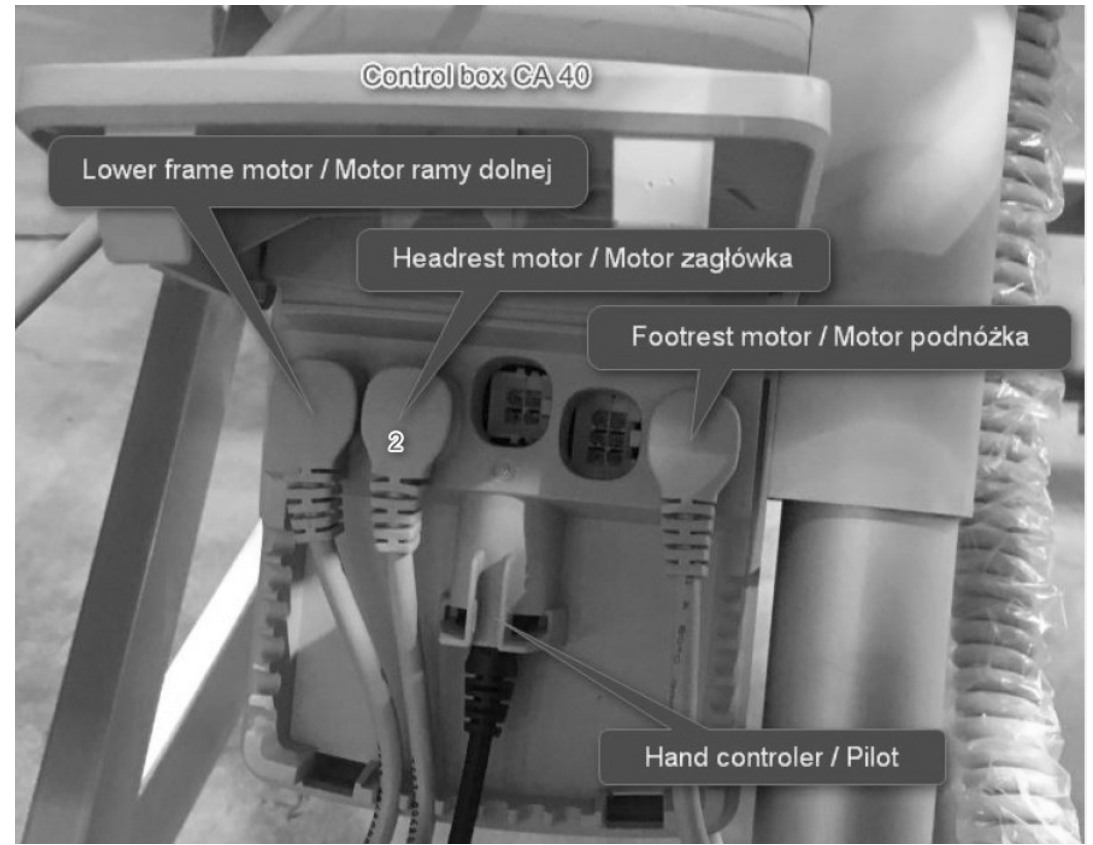

* - Not depicted.

- Use only with metal side rails (EP00256 & EP00257),
Do not use with wooden bars (EP00280) and SSR (EP00493).

** - Use W600111 on 2784700 after assembly to frame. |

2/2

HEAD SIDE WITH WOODEN LATHS / ILLICO				Plan No. E00026487/H1.1
Scale: 1:10	Drafted	Verified	Confirmed	
Date	02/12/2021	06/05/2024	06/05/2024	
Initials	DR	MK	AZ	