

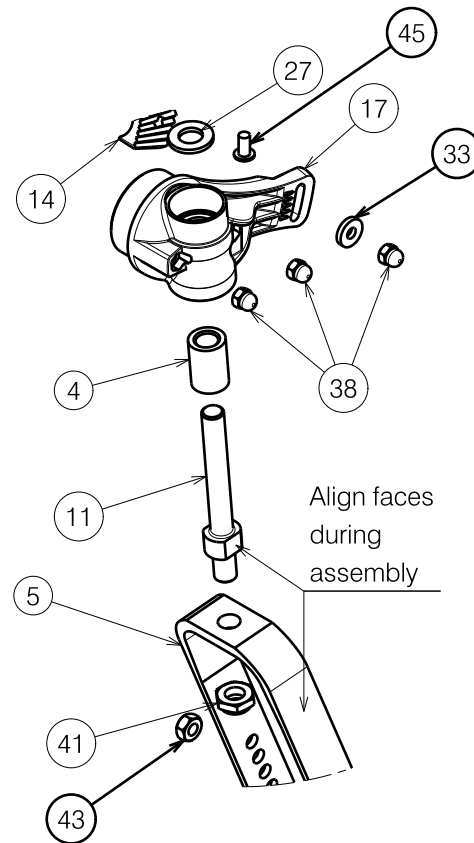


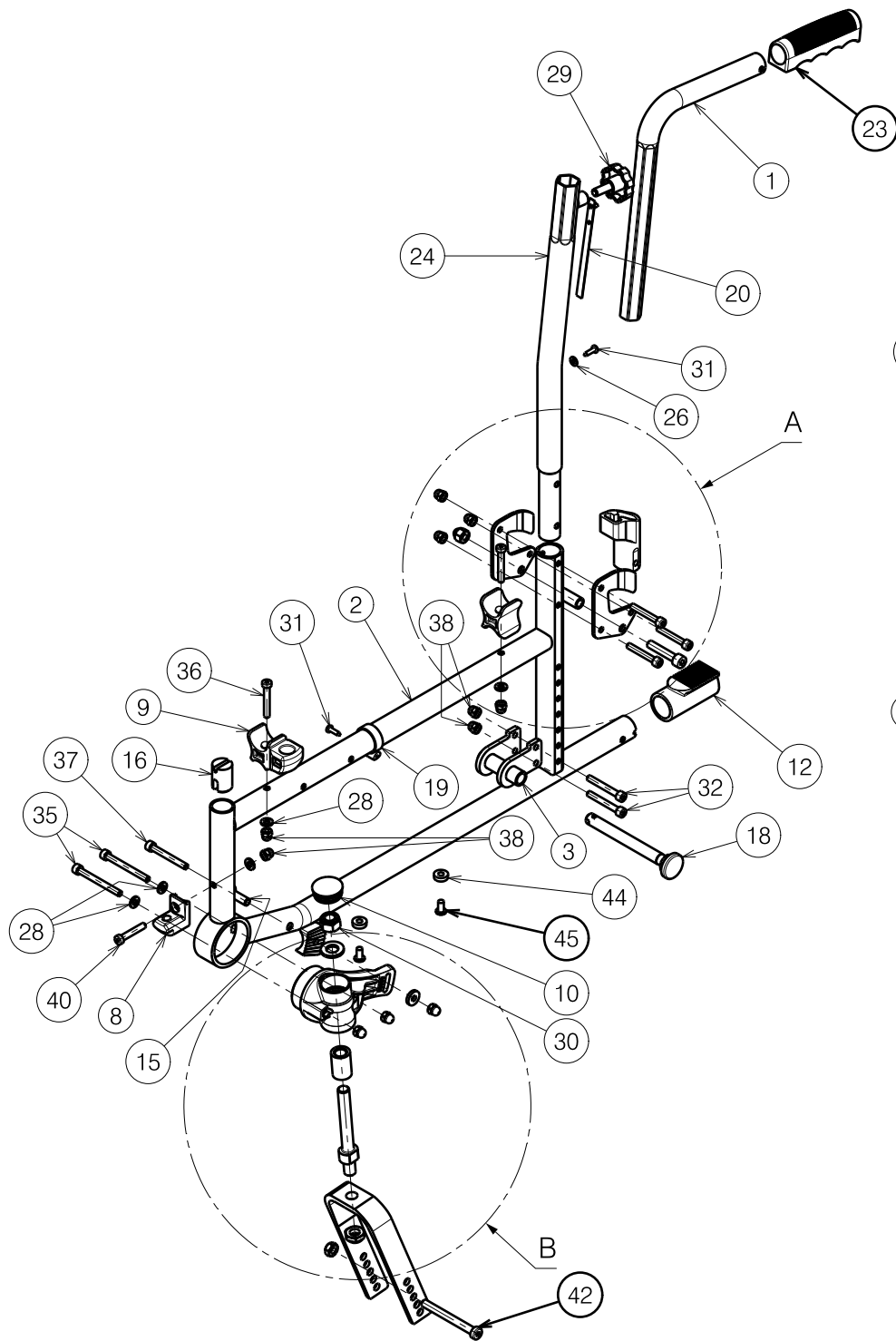

Detail View B (1:4)

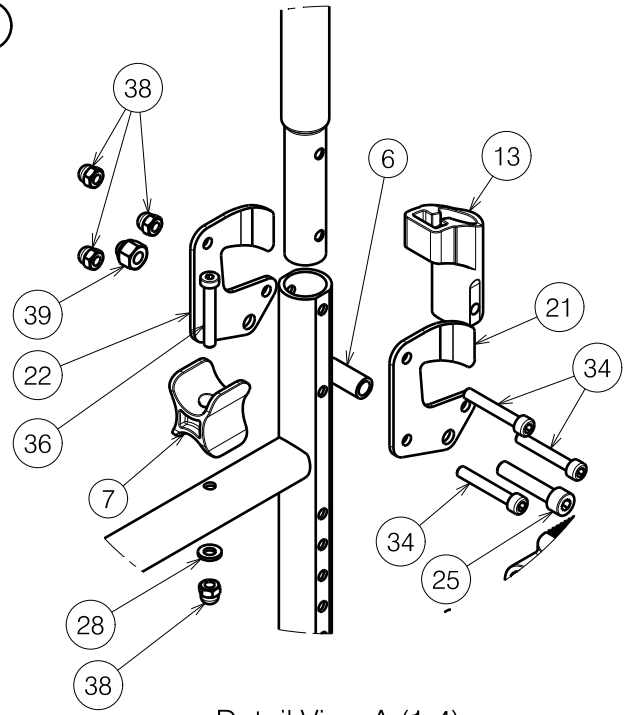

Detail View A (1:4)

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/2

D200N SIDE FRAME

Plan No.

E00075471/B_1.1

Scale: 7:50

Drafted

Verified

Confirmed

Date

30/01/2024

20/02/2024

20/02/2024

Initials

SD

SD

PJ



Material			
No.	Plan	Article	Qty
1	P00058615	11017..	1
2	P00068964	20872.. (L)	1
	P00069813	20873.. (R)	1
3	P00069531	2087454 (L)	1
	P00069532	2087554 (R)	1
4	P00071197	G104854	1
5	X002635	1994437	1
6	X003204	1585100	1
7	X004309	1904133	1
8	X004415	17193	1
9	X004420	1904134	1
10	X004436	1903928	1
11	X004466	1730700	1
12	X004469	1905045	1
13	X004487	1904581	1
14	X004597	1905121	1
15	X004689	1734600	1
16	X004818	1906184	1
17	X005865	1915642 (L)	1
	X005866	1915643 (R)	1
18	XC00037	3990106	1
19	XC00234	3993197	1
20	Y006559	1995662	1
21	Y016062	1736054	1
22	Y016063	1736154	1
23	---	3990041	1
24	YC00503	1C213..	1
25	M8x40 DIN912	1001500	1

Material			
No.	Plan	Article	Qty
26	M5 DIN125	1001550	1
27	M12 DIN125	1001561	1
28	M6 DIN125	1001564	5
29	M8x20	1001580	1
30	M12 DIN985	1001684	1
31	D4x16	1003021	2
32	M6x45 DIN912	1003109	2
33	M6 DIN7349	1003321	1
34	M6x40 DIN912	1003909	3
35	M6x60 DIN912	1005957	2
36	M6x40 DIN7984	1006850	2
37	M6x50 DIN912	1007970	1
38	M6 DIN986	1008492	11
39	M8 DIN986	1902024	1
40	M6x35 DIN7984	1903903	1
41	M12x1.25	1994605	1
42	M8x76 DIN7984	1998993	1
43	M8	1998994	1
44	X004973	1907519	2
45	D6	1992835	2

Quantities are for one side

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/2

Plan No.

E00075471/B_1.1

Scale:	1:1	Drafted	Verified	Confirmed
Date	30/01/2024	20/02/2024	20/02/2024	
Initials	SD	SD	PJ	

