


Detail A

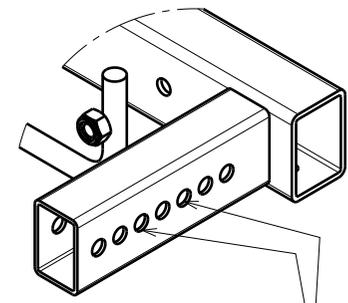

Default mounting holes

A

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

SEAT FRAME L16 FOREST3

Plan No.

E000959/K

Scale:	1:5	Drafted	Verified	Confirmed
Date	26/05/2014	16/10/2025	16/10/2025	
Initials	SD	AS	MK	



Bill of Material			
No.	Plan	Article	Quantity
1	X001016 (M8 DIN985)	1001682	6
2	X003403	1617400	4
3	X004860	1906817 (40)	1
	P00078417	2998394 (42)	1
	X004855	1906741 (45)	1
	P00078004	2998395 (48)	1
	X004887	1906818 (50)	1
4	Y017414	1775254 (40)	1
	P00077353	2686154 (42)	1
	Y017984	1769754 (45)	1
	P00077357	2686254 (48)	1
	Y017889	1776054 (50)	1
5	A00078003	2998396 (40)	1
	---	2998397 (42)	1
	---	2998398 (45)	1
	---	2998399 (48)	1
	---	2998400 (50)	1
6*	Y018411	1778354 (40)	1
	P00077691	2685754 (42)	1
	Y018368	1778454 (45)	1
	P00077360	2685854 (48)	1
	Y018369	1778554 (50)	1

Bill of Material			
No.	Plan	Article	Quantity
7	Y019775	1790554	1
8	Y021021	2007454	1
9	Ø4x11	1001583	4
10	M6 DIN985	1001680	12
11	M6x10	1900641	4
12	M8x40	1008739	2
13	---	1009992	2
14	M12 PA6.6	1993362	2
15	M8x20 DIN7991/8.8	1903136	4
16	M6x30 DIN7984/8.8	1992721	12

* For B06, BZ7 and BZ7E attachment

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Page2/2

2992795

SEAT FRAME L16 FOREST3

Plan No.

E000959/K

Scale:	1:5	Drafted	Verified	Confirmed
Date	26/05/2014	16/10/2025	16/10/2025	
Initials	SD	AS	MK	

