


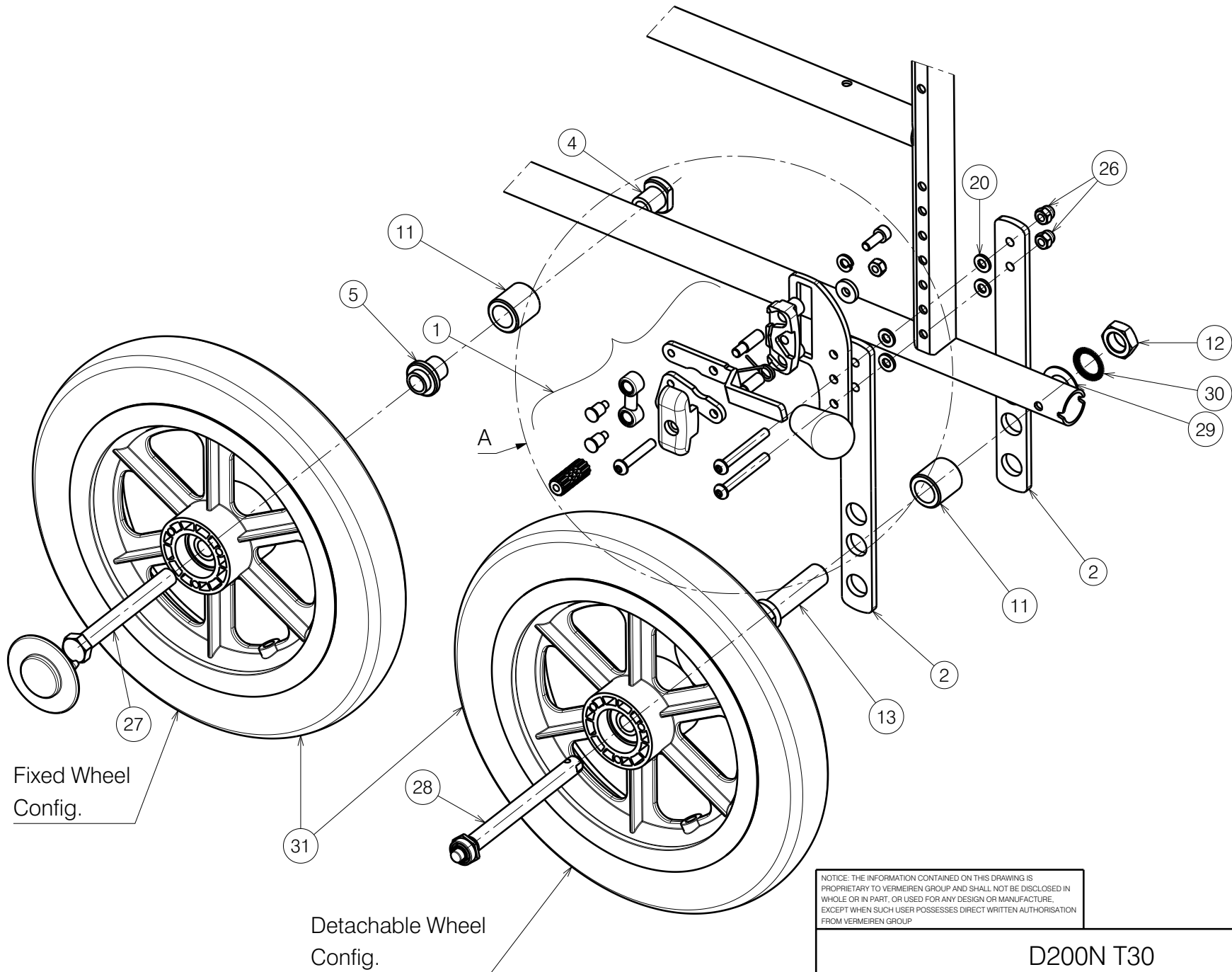

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Sheet 1/2

D200N T30

Plan No.
 E00096803/A.1

Scale: 7:25	Drafted	Verified	Confirmed
Date	16/01/2025	16/01/2025	16/01/2025
Initials	SD	SD	SD




Material			
No.	Plan	Article	Qty
1	EC00247	3991715	1
	EC00248	3991716	1
2	P00095320	2093154	4
3	P00095322	2093254	2
4**	X001308	G051700	2
5**	X001309	G051800	2
6	X002134	X002134	2
7	X002139	1992426	2
8	X002140	1992427	2
9	X002141	1992435	2
10	X002165	1488300	4
11	X002265	1500054	2
12*	X002437	1992699	2
13*	X003662	1998693	2
14	X005274	17937	2
15	X005276	1793900	4
16	X005305	1910032 (L)	1
	X005306	1910033 (R)	1
17	Y016713	1735454 (L)	1
	Y016715	1735554 (R)	1
18	Y020193	1794054	2
19	M6x16 DIN912	1001492	2
20	M6 DIN125	1001564	8
21	M6 DIN985	1001680	2
22	M6 DIN7349	1003321	2
23	M6 DIN127	1004414	2
24	M6x35 ISO7380	1006057	2
25	M6x50 ISO7380	1006997	4
26	M6 DIN986	1008492	4
27**	M12x100 DIN931	1009034	2
28	X005720	1905243	2
29*	M16	1906194	2
30*	M16 DIN17222	1911653	2
31	XC00414	3992836	2

Quantities for left + right side

* Quick release wheel config.

** Fixed wheel config.

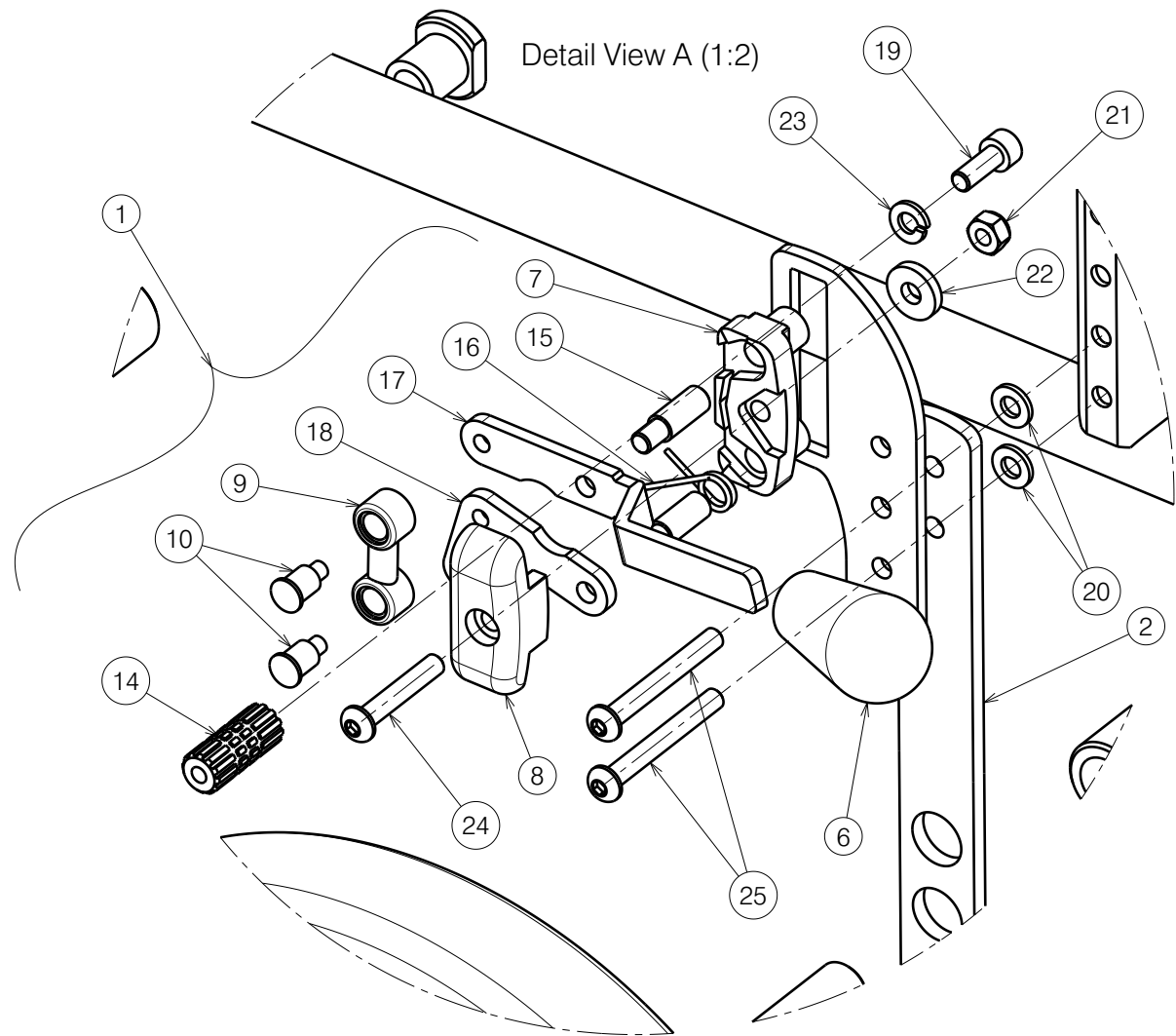

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/2

D200N T30

Plan No.

E00096803/A.1

Scale:	1:2	Drafted	Verified	Confirmed
Date	16/01/2025	16/01/2025	16/01/2025	16/01/2025
Initials	SD	SD	SD	SD

