
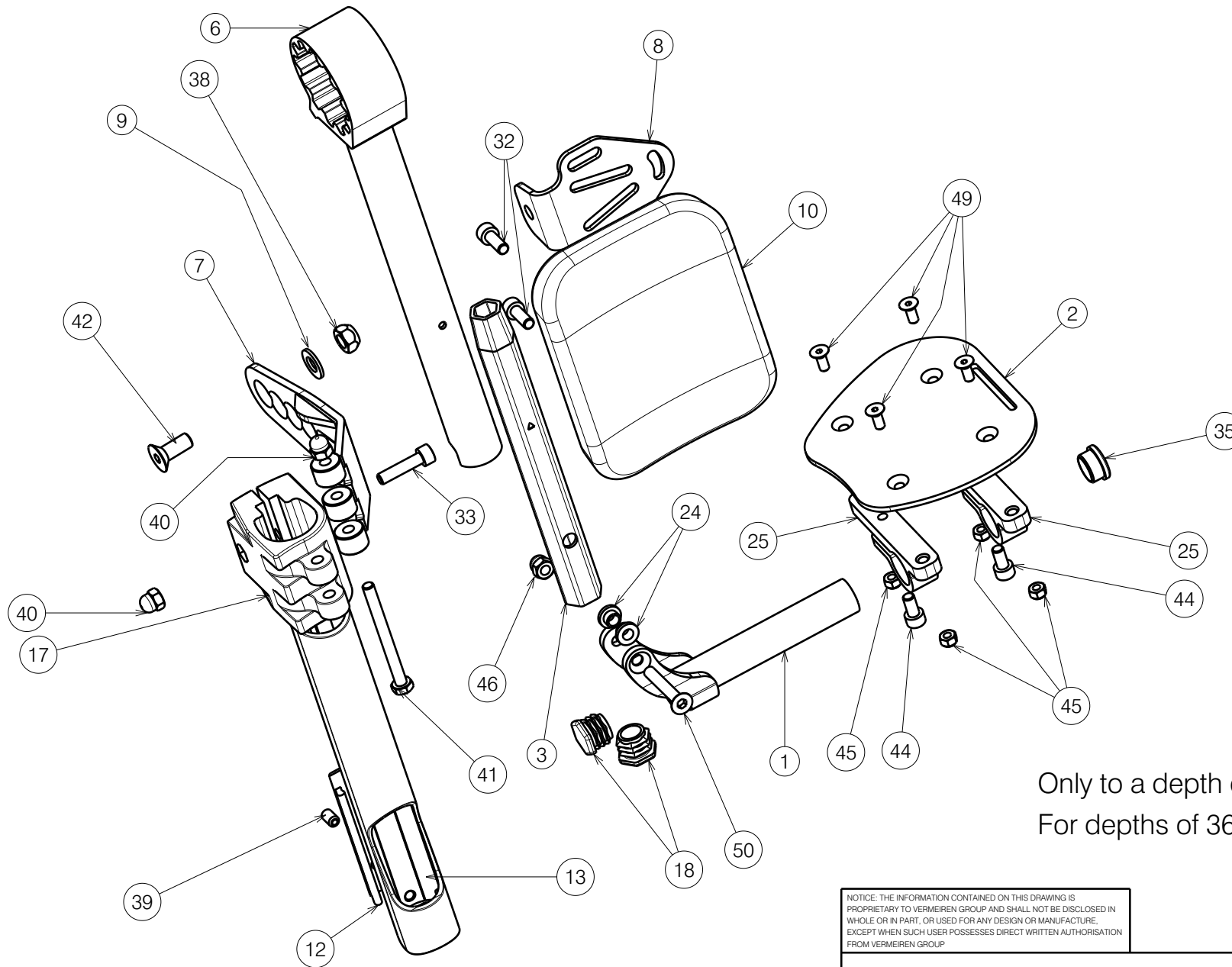


Only to a depth of 32-36 .
 For depths of 36big-50 use EP00538

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

1/3

Legrest BZ8 / Trigo 30-36				Plan No. EP00595/E
Scale: 1:3	Drafted	Verified	Confirmed	
Date	16.07.2021	22.01.2026	22.01.2026	
Initials	AS	PM	AS	




Only to a depth of 32-36 .

For depths of 36big-50 use EP00538

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

<h2 style="margin: 0;">Legrest BZ8 / Trigo 30-36</h2>				2/3
Scale:	1:3	Drafted	Verified	Confirmed
Date	16.07.2021	22.01.2026	22.01.2026	
Initials	AS	PM	AS	

Plan No.
EP00595/E


Material			
No.	Plan	Article	Quantity
1	YP06229(R)	2796365 (30-32)	1
	YP06126 (L)	2729965 (30-32)	1
	YP06254 (R)	2796665 (34-36)	1
	YP06893 (L)	2765665 (34-36)	1
2	YP06231(R)	2796454 (30-32)	1
	YP06232 (L)	2796554 (30-32)	1
	YP06256 (R)	2796754 (34-36)	1
	YP06257 (L)	2796854 (34-36)	1
3	YP06246(Short)	2796099	1
	YP09022(Long)	2631699	1
4	YP08517(R)	2618654	1
	YP08518(L)	2618754	1
5	YP09150(R)	2636454	1
	YP09151(L)	2636554	1
6	YP09153	2636654	1
7	YP09182	2637754	1
8	YP09164	2636854	1
9	X001119	1001554	1
10	X003779	1900228	1
11	X004423	1717300	1
12	X004658	1904968	1
13	X004666	1905731	1
14	X004690	1905735	1
15	X004713	1905734	1
16	X005880	1915832	2
17	X004739	1905739	1
18	X004774	1906056	2
19	X004795	1905970	2
20	X005028	G103900	1
21	X005047	1908420	1
22	X005055	1917169	1
23	X005046	G103800	1

Material			
No.	Plan	Article	Quantity
24	XP01127	2990164	2
25	XP01504	GP07699	2
26	XP01512	2993454	1
27	XP01523	GP08000	1
28	XP01546	2993452	1
29	XP01628	GP09100	1
30	XP02030	GP14499	1
31	XP03096	2996674	1
32	M6x16 DIN912	1001492	2
33	M6x25 DIN912	1001494	1
34	Rivet D4x14	1001584	4
35	Cap D19	1001602	1
36	Cap D25	1001608	1
37	M5 DIN985	1001679	1
38	M8 DIN985	1001682	1
39	M6x12 DIN916	1005949	1
40	M6	1008492	2
41	M6x65 DIN931	1991860	1
42	M8x20	1994677	1
43	M5x10 DIN7991	1999977	2
44	M6x12 DIN912	2994005	2
45	M5 DIN985	2994006	4
46	M6 DIN986	2994059	1
47	M6x25 ISO7380	2994062	1
48	D3.9x9.5	2992349	2
49	M5x12 DIN7991	2994076	4
50	M6x40 DIN7991	2994123	1
51	G0409	G0409	1

Only to a depth of 32-36 .
For depths of 36big-50 use EP00538

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Legrest BZ8 / Trigo 30-36				Plan No.
				EP00595/E
Scale:	1:3	Drafted	Verified	Confirmed
Date	16.07.2021	22.01.2026	22.01.2026	
Initials	AS	PM	AS	

