


Use WEICONLOCK AN 302-50

Use WEICONLOCK AN 302-50

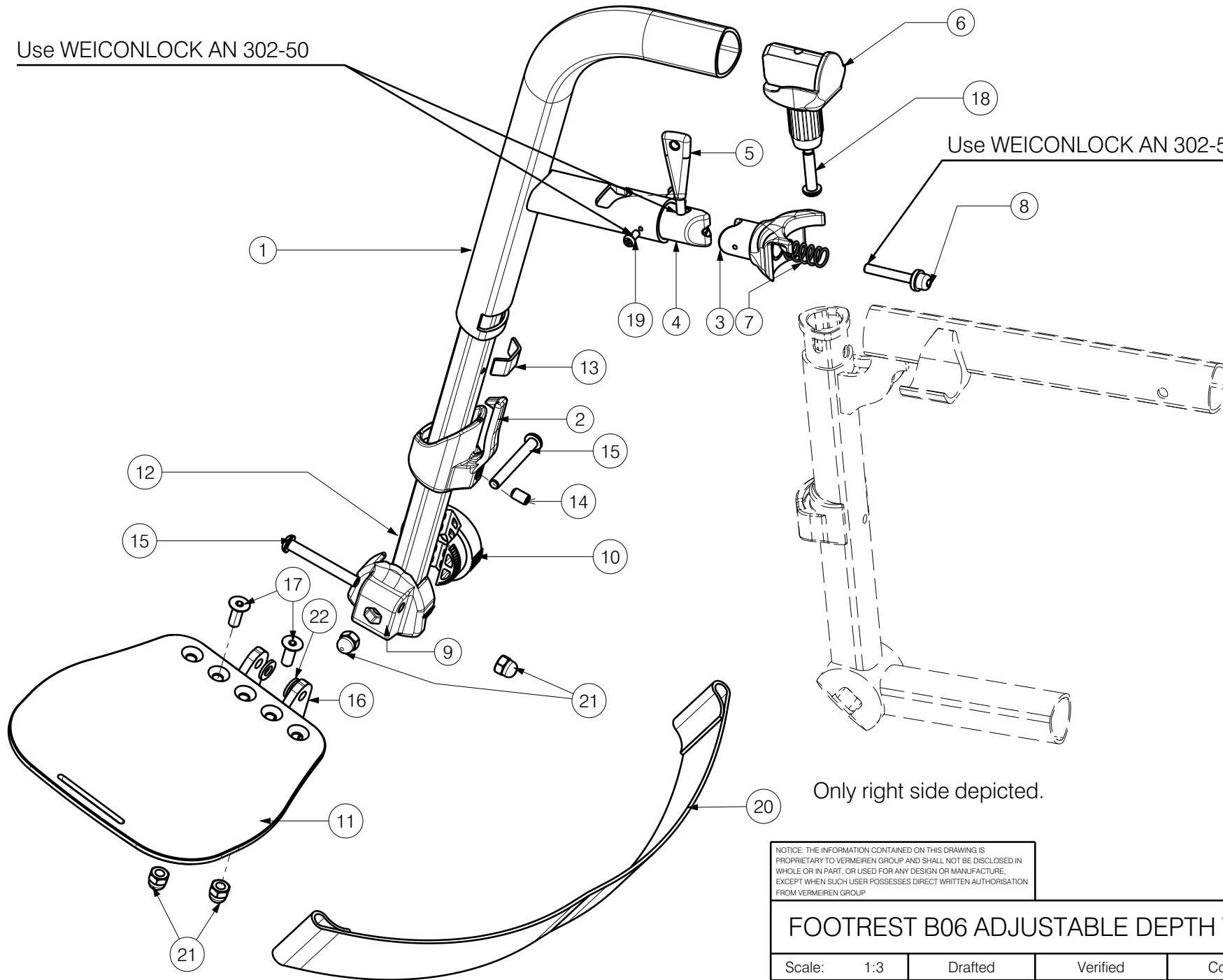


Only right side depicted.

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

FOOTREST B06 ADJUSTABLE DEPTH TRIGO

Scale:	1:3	Drafted	Verified	Confirmed
Date	2023-12-18	22-08-2024	22-08-2024	
Initials	AS	AZ	AS	

--
Plan No. E00071514/A1.1

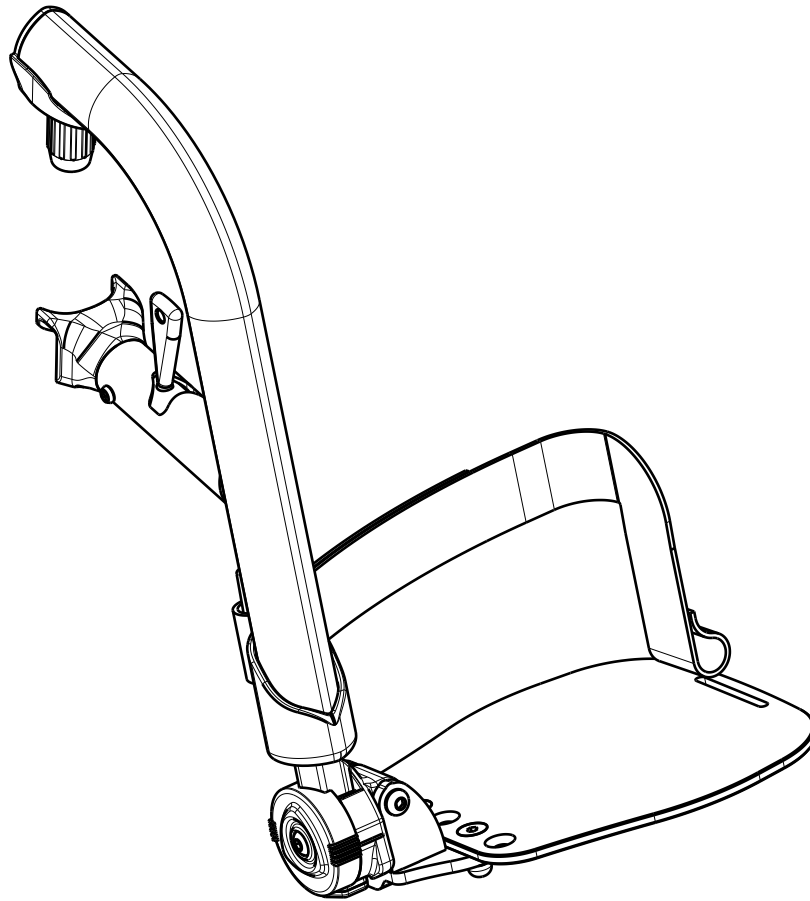

Material			
No.	Plan	Article	Quantity
1	YP05383	27122..(70)	1
	YP05531	27137..(80)	1
	YP05536	27138..(90)	1
2	XP01542	2993471	1
3	XP01546	2993452	1
4	XP01565	2993472	1
5	XP02030	GP14499	1
6	XP01512	2993454	1
7	XP01628	GP09100	1
8	XP01523	GP08000	1
9	XP02026	2993662	1
10	XP02025	2993661	1
11	P00072069	2683054(36-38)	1
	P00070608	2682954(40-42)	1
	P00070607	2682854(44-46)	1
	P00070811	2682754(48-50)	1
12	YP06404	2730126	1
13	Y016501	1733500	1
14	M6x12 DIN916	1005949	1
15	M6x45	2994086	2
16	P00069732	2682654	1
17	M6x16	2994064	2
18	M6X25	2994062	1
19	M3.5x9.5	2994075	2
20	Belt	2994199	1
21	M6 DIN986	2994059	4
22	M6	2994068	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

--

FOOTREST B06 ADJUSTABLE DEPTH TRIGO

Plan No.
E00071514/A1.1

Scale:	1:3	Drafted	Verified	Confirmed
Date	2023-12-18	22-08-2024	22-08-2024	
Initials	AS	AZ	AS	



Only right side depicted.