


after regulation use WEICONLOCK AN 302-50

after regulation use WEICONLOCK AN 302-50


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Only for depth: 38-50  
For depth 30-36 use E00059601

<b>FOOTREST B06F / 38-50 / TRIGO</b>				1/2
Scale:	1:3	Drafted	Verified	Confirmed
Date	15.05.2023	22.01.2026	22.01.2026	
Initials	AZ	AZ	AS	

---
Plan No.
E00059605/C1.1

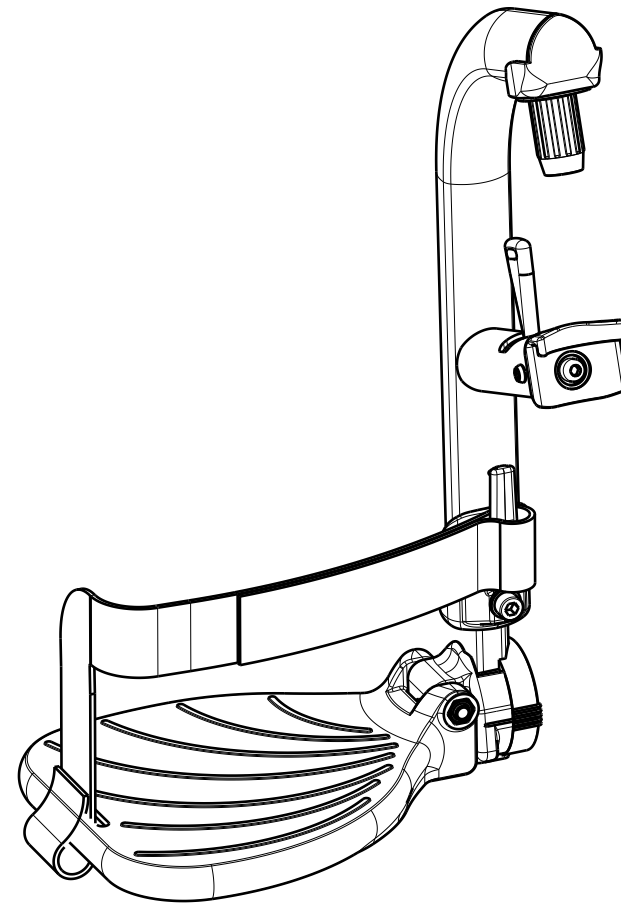

No.	Plan	Article	Qty
1	M6x35 DIN912	2994634	1
2	P00058850	2676600	1
3	P00059600	2676926	1
4	YP05383	27122...(70)	1
	YP05531	27137...(80)	1
	YP05536	27138...(90)	1
5	XP01512	2993454	1
6	XP01523	GP08000	1
7	XP01542	2993471	1
8	XP01546	2993452	1
9	XP01565	2993472	1
10	XP01628	GP09100	1
11	XP01655(L)	2993501(38-40)	1
	XP01656(R)	2993502(38-40)	1
	XP01639(L)	2993475(42-44)	1
	XP01640(R)	2993476(42-44)	1
	XP01644(L)	2993480(46-48)	1
	XP01643(R)	2993479(46-48)	1
	XP01641(L)	2993477(50)	1
	XP01642(R)	2993478(50)	1
12	XP02025	2993661	1
13	XP02026	2993662	1
14	XP02030	GP14499	1
15	M6 DIN985	2994063	2
16	B18 (38-50)	2994199	1
17	M6x25 ISO7380	2994062	1
18	M6x40 ISO7380	2994066	1
19	D6,4 DIN125	2994414	1
20	M6x45 ISO7380	2994086	1
21	Screw D3.9x9.5	2992349	2



NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Only for depth: 38-50  
For depth 30-36 use E00059601

---

<b>FOOTREST B06F / 38-50 / TRIGO</b> 2/2				Plan No.
				E00059605/C1.1
Scale: 1:3	Drafted	Verified	Confirmed	
Date	15.05.2023	22.01.2026	22.01.2026	
Initials	AZ	AZ	AS	