


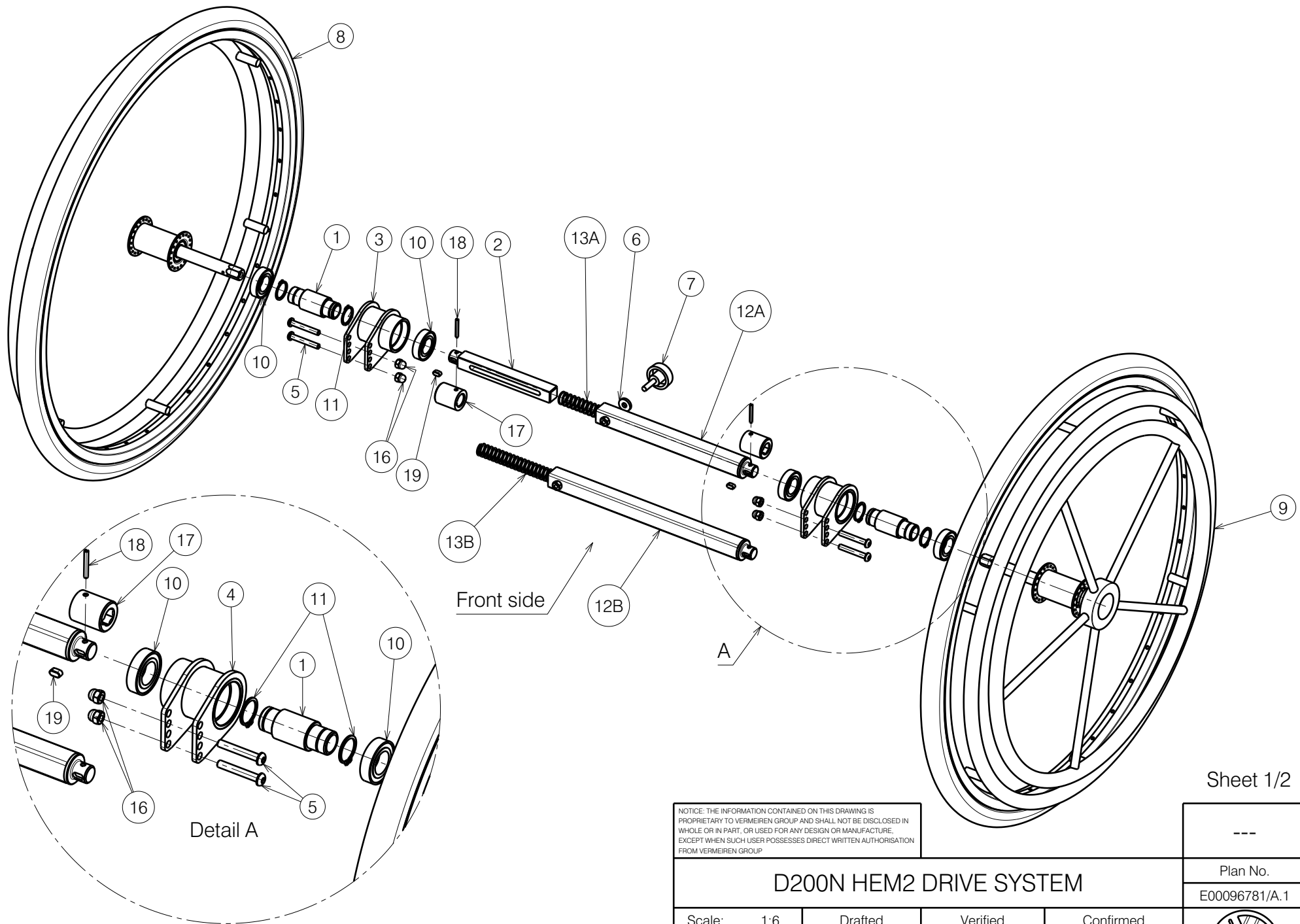

Sheet 1/2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

D200N HEM2 DRIVE SYSTEM

Scale: 1:6	Drafted	Verified	Confirmed
Date	13/01/2025	13/01/2025	13/01/2025
Initials	SD	SD	PJ

Plan No.
E00096781/A.1



Bill of Material			
No.	Plan	Article	Quantity
1	X002976	1995343	2
2	P00065423	2684100	1
3	P00095258	2092954	1
4	P00095256	2093054	1
5	M6x40 ISO7380	1006111	4
6	M6 DIN7349	1003321	1
7	M6x25	1903394	1
8	24" BL	1918198	1
9	24" BL	1918199	1
10	61904-2RS1	1994568	4
11	Ø20 DIN471	1995259	4
12A	P00071835	2683800	1
12B	P00071836	2683900	1
13A	X005892	G103100	1
13B	X005891	G103000	1
16	M6	1008492	4
17	P00064455	GP29900	2
18	D3x26 DIN 1481	1908821	2
19	5x5x10 DIN6885A	2998153	2

12A - 13A Seat widths 39-42-44

12B - 13B Seat widths 46-48-50

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.				---
D200N HEM2 DRIVE SYSTEM				Plan No.
				E00096781/A.1
Scale:	1:6	Drafted	Verified	Confirmed
Date	13/01/2025	13/01/2025	15/01/2025	
Initials	SD	SD	PJ	

